
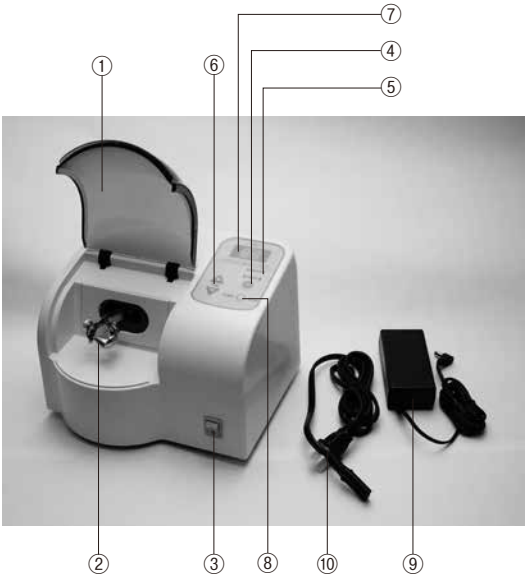
Screws / Tornillos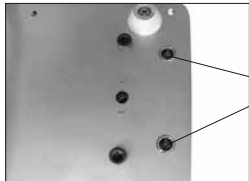

Storage holes / Compartimento

Fig.3

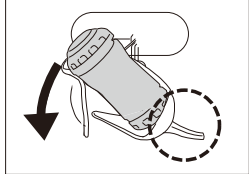
Fig.4

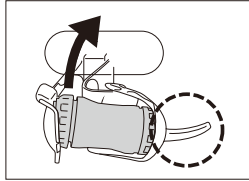
Fig.5

CAPSULE MIXER CM-II

DENTAL CAPSULE MIXER

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

Mixing of glass ionomer cements and other predosed cements in capsules

IDENTIFICATION OF PARTS

1. Safety cover
2. Capsule holder
3. Power switch
4. Mode button
5. Mode indicator
6. Timer selection button
7. Time display
8. Start button
9. Adapter
10. Power cord

SETUP

1. Remove the two black transportation screws from the bottom of the device (Fig. 1). Keep the device as horizontal as possible when unscrewing. Loosen both screws evenly.
2. Store the screws in the storage holes (Fig. 2). The screws will be necessary again if the device has to be transported in the future.
3. Firmly connect the power cord to the device and subsequently to the main power socket (100-240V AC) (Fig. 3).

SAFETY INSTRUCTIONS

1. The unit should only be used by a qualified dental professional according to the instructions.
2. The unit should not be exposed to direct sunlight, dust, a wet environment or placed near a heater.
3. Keep out of the reach of infants or children.
4. Prior to use, make sure the unit is properly connected to the appropriate power source.
5. The unit should be checked on a regular basis.
6. Do not handle the unit with wet hands.
7. Install the mixer on a flat and stable place.
8. In case of breakdown, disconnect the unit and appropriately label the unit with details of possible fault. The repair of the unit may only be carried out by a qualified person.

DIRECTIONS FOR USE

Make sure the safety cover is tightly closed. If the cover is open, OF is shown on the time display.

1. Turn on the power switch. The mode indicator and time display will light up.
2. Set the mixing time according to the manufacturer's instructions of the encapsulated product. Two preferred time settings can be saved as mode 1 or 2. Both modes are preset at the factory at 10 seconds. To change the time setting, follow the below steps.
 - a) Select mode 1 or 2 by pressing the mode button.
 - b) Press either "+" or "-" once to activate (display starts flashing).
 - c) Then press "+" or "-" button to increase or decrease the mixing time by 1 sec. Mixing time in the range of 1-30 seconds may be selected.
 - d) Press the mode button to set/save the selected mixing time (display stops flashing).
3. Open the cover. Hold the capsule with the nozzle facing upward, install it in the swing arm of the capsule holder and press down until it fits in the fixed arm as shown in figure 4.
4. Close the cover and press the start button. Once the mixer starts, the time display starts counting down.

Note:

For safety reasons, the mixer will not start if the cover is open.
5. Once the selected mixing time has finished, the cover can be opened. Remove the mixed capsule from the capsule holder. Make sure the capsule nozzle is facing upward when removing from the holder (Fig. 5).

Note:

 - a) Do not attempt to start mixing without a capsule being installed in the device. This may cause breakage of the device. Breakage caused by improper use is not covered by the warranty.
 - b) In case of emergency, the mixing process can be stopped by opening the safety cover.
 - c) If a capsule leaks and material comes into contact with the device, wipe off immediately.

MAINTENANCE AND CARE

Put the device away from direct sunlight, strong heat sources and dusty areas. Prior to routine cleaning and maintenance of the unit, disconnect the power cord from the mains socket. The device can be cleaned with a soft cloth soaked in warm water or mild detergent. Avoid using volatile solvents.

TROUBLESHOOTING

If any difficulty with the device is experienced, please check below for possible causes of the problem and suggested corrective actions before seeking further advice or repair from an authorized GC Dealer or Service center.

Problem	Check	Corrective action
The power switch is on but time display or mode indicator is off.	Is the power cord correctly connected to the mains socket?	Correctly connect the power cord to the mains socket.
	Is the adapter correctly connected to the device?	Correctly connect the adapter to the device.
The device does not start mixing even though the start button is pressed.	Is the cover closed?	Close the cover tightly.
The device moves during mixing.	Is the device placed on a flat surface?	Place the device on a flat surface or use a rubber mat under the device.

WARRANTY/REPAIR

Warranty: 1 year from the date of purchase.

In case of a breakdown during the term, repair will be carried out free of charge provided that the unit has been used under normal conditions and according to the instructions for use.

DISPOSAL

Dispose of the unit according to the local regulations.

PACKAGING

- | | |
|------------|---|
| Mixer | 1 |
| AC adapter | 1 |
| Power cord | 1 |

SPECIFICATIONS

Equipment class	Class II
Protection from ingress of liquid	IPX0 (Ordinary equipment)
Observe instructions for use	
Power supply voltage	DC24V
Power input	50VA
AC Adapter voltage	INPUT 100-240V AC 50/60Hz OUTPUT DC24V 1.5A
Mixing speed	Approx. 4,000 rpm
Mixing time	Selectable from 1-30 seconds
Noise level	< 70dB(A)
Ambient temperature during use	10°C-40°C
Storage condition	Temperature -10°C-60°C Humidity 10%-85% Pressure 500-1060hPa
Dimensions	(Device) W x L x H 190mm x 18mm x 150mm
	(Adapter) W x L x H 60mm x 120mm x 40mm
Weight	2.9kg (including adapter)
COMPLIANCE	CE
	EN60601-1 EN60601-1-2

The specifications of the unit may be subject to change without notice.

CAUTION

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Last revised : 04/2024

Prior to use, carefully read the instructions for use.

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