G-ænial™ BULK Injectable

UNIVERSAL LIGHT-CURED RADIOPAQUE INJECTABLE COMPOSITE FOR BULK FILL RESTORATION

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

- Direct restorative for Class I, II, III, IV and V cavities
- Fissure sealant
- Sealing hypersensitive areas
- Repair of (in) direct aesthetic restorations, temporary crown & bridge, defect margins when margins are in enamel
- Blocking out undercut
- Liner or base
- Core build-up

CONTRAINDICATIONS


- Direct pulp capping.
- Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

- Preparations
 - Hold the syringe upright and remove the cap by turning counterclockwise.
 - Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
 - Place the light protective cap until use.

Note:

 Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection make sure that the threads are free of remnant material.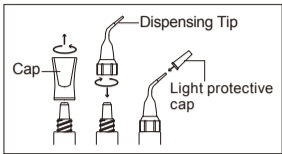

- Shade Selection

Select an appropriate shade of G-ænial BULK Injectable.
- Cavity Preparation

Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Rubber dam is recommended to isolate the prepared tooth from contamination with saliva, blood or sulcus fluid.

Note:

For pulp capping, use calcium hydroxide.

- Bonding Treatment

For bonding G-ænial BULK Injectable to enamel and/or dentin, use a light-cured bonding system such as G-Premio BOND, G-ænial Bond or G-BOND. Follow manufacturer's instructions.
- Placement of G-ænial BULK Injectable

- Remove cap from the dispensing tip.


Note:

Prior to syringe material into the cavity, make sure to check the following:

 - Dispensing tip is securely attached to the syringe.
 - Any possible air from the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material reaches the opening of the tip (Fig. 2).
- Place the dispensing tip as close as possible to the prepared cavity, and slowly push the plunger to extrude material. Alternatively, dispense material onto a mixing pad and transfer to the cavity using a suitable instrument.

Note:

 - If plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
 - Minimize exposure to ambient light. Ambient light can shorten the working time.
 - After use, immediately remove and dispose of the dispensing tip and tightly close the syringe with the cap.

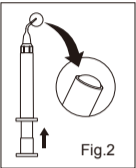Clinical Hints

In order to inject effectively, ensure uniformity across the entire surface of the restoration during build up. Once the required amount has been injected, release the pressure on the plunger and withdraw the syringe in a direction perpendicular to the surface. This will allow the material to separate from the dispensing tip and provides a smooth surface over the restoration.

- Light-curing

Light-cure G-ænial BULK Injectable using a light curing unit with a minimum intensity of 1,200mW/cm² for 20 seconds. G-ænial BULK Injectable can be filled and cured in a single bulk increment depth of up to 4mm. Keep light guide as close as possible to the surface.

Irradiation Time and Effective Depth of Cure		
	Irradiation time	20 sec. (High power LED) (>1200mW/cm ²)
Shade	30 sec. (Halogen/LED) (>700 mW/cm ²)	
A1, A2	4.0mm	

- Finishing and Polishing

Finish and polish using standard techniques.

SHADES

A1, A2

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C/39.2-77.0°F) away from high temperatures or direct sunlight.

PACKAGES

- Syringe 3.4g (2.0mL) x1, DISPENSING TIP III (plastic) x 20, Light-protective cap x1
- Dispensing Tip Refill: 30 DISPENSING TIP III (plastic type), 2 light protective cap 30 DISPENSING TIP III (needle type), 2 light protective cap

CAUTION

- In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Do not mix with other similar products.
- Dispensing tips are for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilized in an autoclave or chemicleave.
- Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- Wear protective eye glasses during light curing.
- When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Avoid getting material on clothing.
- In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light-curing.
- Do not use G-ænial BULK Injectable in combination with eugenol containing materials as eugenol may hinder G-ænial BULK Injectable from setting.
- Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the paste.
- Dispose of all wastes according to local regulations.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gcamerica.com> They can also be obtained from your supplier.

CLEANING AND DISINFECTING: MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERS. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 06/2018

Rx Only

GC

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-265 Toride-cho, Kasugai, Aichi 486-0844, Japan
RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heitorora, 359, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0955
CNPJ: 08.279.999/0001-61
RESP. TÊC: Mayara de Santis Ribeiro - CRO/SP 105.982

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