

G-ænial™ Universal Injectable

Clear Aligner Partner

For clear aligner attachments, every detail counts



The right material can turn a slow, inconsistent process into one that's fast, accurate, and reliable.

G-ænial™ Universal Injectable is a high strength injectable composite resin. This is ideal for creating clear aligner attachments with excellent clinical outcomes (exceptional esthetics, no flash, ideal flow without being runny, high strength and no porosities) and most importantly a significant reduction in application time helping further improve the cost effectiveness of the aligner treatment procedure.

Courtesy of Dr. Bharat Agrawal.



Initial presentation, prior to aligner attachment placement.



Initial presentation, lateral view.



G-ænial™ Universal Injectable enamel shade JE (Junior Enamel), used to fill the attachment space.



After applying resin adhesive, the template is placed in the mouth and the G-ænial™ Universal Injectable is light cured for 20 seconds and the template is removed.



Attachments in position after template removal.



Completed clear aligner attachments after final polishing.

G-ænial™ Universal Injectable

Ordering Information

G-ænial™ Universal Injectable Syringe Refills

Contains: Two 1.7g (1.0mL) syringes, 10 long dispensing tips and 10 short dispensing tips.

012364 A1	012365 A2	012366 A3	012367 A3.5
012368 A4	012369 B1	012370 B2	012371 CV
012372 CVD	012373 AO1	012374 AO2	012375 AO3
012376 JE	012377 AE	012378 XBW	012379 BW

G-ænial™ Universal Injectable Unitip Refills

Contains: 15 0.16mL unitips.

012348 A1	012349 A2	012350 A3	012351 A3.5
012352 A4	012353 B1	012354 B2	012355 CV
012356 CVD	012357 AO1	012358 AO2	012359 AO3
012360 JE	012361 AE	012362 XBW	012363 BW

