




1
Recommended: Apply
CAVITY CONDITIONER
(GC) for **10 sec.**



2
Rinse and gently dry, do
not desiccate.

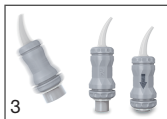

3
Shake or tap. Depress
plunger.



4
Mix for **10 sec.** Working
time is **2 min.** from start
of mix.



5
Insert into Capsule Applier.
Click twice to prime
capsule.



6
Dispense within
10 sec.




7
Pack and contour.
Avoid moisture
contamination and
desiccation.



8
Final finishing after
6 min. from start
of mix.



9
Apply G-COAT PLUS™.
DO NOT AIR BLOW.*
*Alternatively, apply GC Fuji
VARNISH™ (blow dry) or
GC Fuji COAT™ LG (light cure).

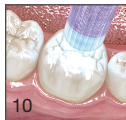

10
Light cure for **20 sec.**