

GC FUJIROCK®

Family of High Quality Dental Stones

Capture every detail



Since 1921

100 years of Quality in Dental



Family of High Quality Dental Stones

GC FUJIROCK® - the Foundation of Success

Superior quality is what makes the GC FUJIROCK® family of gypsums among the most popular in the world. Used in the production of all types of dental prosthetics, GC FUJIROCK® is specifically designed to produce consistently accurate models and dies, fully meeting the requirements of the modern dental laboratory. The line of gypsum materials were developed with your dental laboratory's techniques in mind. From world class, synthetic or natural Type 4 die stones, to fast setting low expansion die stones, even specifically adapted Type 4 implant stones. These are all extremely high-quality products which have been specifically designed to suit the needs of the dental professional. Each product also comes in a range of different colors and sizes so you can choose shade combinations that present your laboratory and work at their very best.



GC FUJIROCK® EP Classic

GC FUJIROCK® EP Classic lives up to its name. One of the world's most popular Type 4 dental die stones, it is "the classic." It is ideal for all kinds of prosthetics, combining high precision, outstanding edge hardness and high-pressure stability. GC FUJIROCK® EP Classic is very fluid yet still thixotropic for excellent handling properties and easy pouring with no porosity. Choose the color that best suits your daily work:

- Golden Brown
- Topaz Beige



| GC FUJIROCK® EP Classic | |
|--|---------------|
| Golden Brown/Topaz Beige | |
| Liquid/Powder Ratio (L/P) | 20mL/100g |
| Manual Mixing | 15 seconds |
| Vacuum Mixing | 45 seconds |
| Setting Time | 10-13 minutes |
| Removal Time | 30-40 minutes |
| Setting Expansion Time (after 2 hours) | ≤ 0.09% |
| Compressive Strength | ≈ 53 MPa |

GC FUJIROCK® EP Premium

GC FUJIROCK® EP Premium has been specially developed for high performance and sensitive technologies such as extended implant cases and complex full-ceramic indications. The improved powder composition of GC FUJIROCK® EP Premium and refined color setting make this special stone the perfect choice for a variety of applications.

Choose the color that best suits your refined works:

- Pastel Yellow
- Polar White
- Inca Brown
- Titanium Grey

| GC FUJIROCK® EP Premium | |
|--|---------------|
| Pastel Yellow/Polar White/Inca Brown/Titanium Gray | |
| Liquid/Powder Ratio (L/P) | 20mL/100g |
| Manual Mixing | 15 seconds |
| Vacuum Mixing | 45 seconds |
| Setting Time | 9-12 minutes |
| Removal Time | 25-35 minutes |
| Setting Expansion Time (after 2 hours) | ≤ 0.09% |
| Compressive Strength | ≈ 60 MPa |



NEW FUJIROCK®

NEW FUJIROCK® is an improved Type 4 die stone in its physical properties and workability. This new die stone is ideal for all kinds of prosthetics, with high precision, outstanding edge hardness and high pressure stability. Thus a precise model as a basis for precise prosthetics is achieved. Available in Golden Brown and Alabaster Ivory.

Benefits

- Easy-to-mix, and flows well yet runs less from the impression when pouring.
- Low setting expansion (0.08) provides accurate models with smooth surfaces.
- Increased compressive strength results in no fractures during removal from impression.



| Physical Properties | |
|--|------------|
| Items | Value |
| Liquid/Powder Ratio (L/P) | 20mL/100g |
| Setting Time | 11 minutes |
| Linear Setting Expansion after 2 hours | 0.08% |
| Compressive Strength after 1 hour | 58MPa |

NEW FUJIROCK® IMP

NEW FUJIROCK® IMP is an extremely precise and hard Type 4 die stone with very low expansion (0.06). Because of these very special properties, NEW FUJIROCK® IMP is an ideal stone for implant cases, CAD/CAM, and long-span superstructures which require a long span of accuracy and detail. A very precise die stone for very precise work. Available in Golden Brown and Alabaster Ivory.

Benefits

- Easy-to-mix, and flows well yet runs less from the impression when pouring. Exhibits superior handling characteristics.
- Low setting expansion (0.06) provides accurate models with smooth surfaces.
- Increased compressive strength results in no fractures during removal from impression.
- Specifically designed for, implant, CAD/CAM, and very precise long span restorations.

| Physical Properties | |
|--|------------|
| Items | Value |
| Liquid/Powder Ratio (L/P) | 20mL/100g |
| Setting Time | 11 minutes |
| Linear Setting Expansion after 2 hours | 0.06% |
| Compressive Strength after 1 hour | 54 MPa |



NEW FUJIROCK® FAST SET

NEW FUJIROCK® FAST SET is everything you would expect from the GC FUJIROCK® line of high quality gypsum materials. NEW FUJIROCK® FAST SET has all of the strong and precise qualities as NEW FUJIROCK®, but sets in only five minutes. Available in Golden Brown.



Benefits

- Sets in five minutes, yet provides proper compressive strength in a shorter time. Enables quality work to be done very quickly.
- Easy-to-mix, flows well yet runs less from the impression when pouring.
- Low setting expansion (0.08) provides accurate models with smooth surfaces.
- Increased compressive strength results in no fractures during removal from impression.
- Provides ample working time with a sharp set.

| Physical Properties | |
|--|-----------|
| Items | Value |
| Liquid/Powder Ratio (L/P) | 20mL/100g |
| Setting Time | 5 minutes |
| Linear Setting Expansion after 2 hours | 0.08% |
| Compressive Strength after 1 hour | 58 MPa |



The Art of Presenting Aesthetics

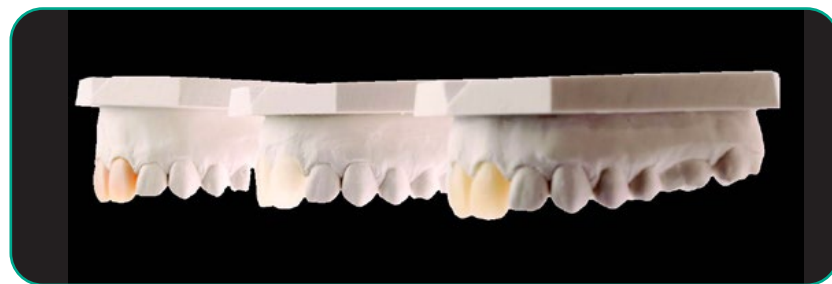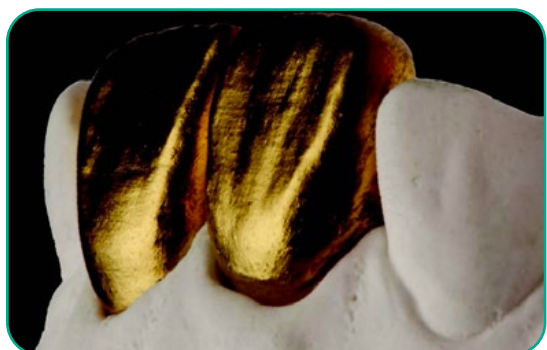

Restorations courtesy of: M. Brusch MDT, S. Maffei MDT, D. Ibraimi MDT, M. Bladen MDT, G. Quini MDT

Ancillary Products

COECAL™ is a hard, quick-setting, accurate Type 3 laboratory stone that's unexcelled for its minimum processing change. Its excellent detail reproduction makes COECAL™ the choice for study models, working models and flasking procedures. Made in two colors: Buff and White.

- Reduces breakage and chipping
- Excellent reproduction of detail
- Low setting expansion (0.13%)
- Great accuracy
- Quick Setting



GC FUJIROCK®

Ordering Information



GC FUJIROCK® EP - Classic

- 890222 Golden Brown (5 kg)
- 890226 Golden Brown (12 kg)
- 890225 Topaz Beige (5 kg)

GC FUJIROCK® EP - Premium

- 890367 Inca Brown (4 kg)
- 890364 Pastel Yellow (4 kg)
- 890368 Titanium Grey (4 kg)
- 890365 Pastel Yellow (11 kg)
- 890225 Polar White (4 kg)

NEW FUJIROCK®

- 315005 Alabaster Ivory (3 kg)
- 315003 Golden Brown (3 kg)
- 315021 Alabaster Ivory (22.7 kg)
- 315019 Golden Brown (22.7 kg)

NEW FUJIROCK® Fast Set

- 315004 Golden Brown (3 kg)
- 315020 Golden Brown (22.7 kg)

NEW FUJIROCK® IMP

- 315007 Alabaster Ivory (3 kg)
- 315006 Golden Brown (3 kg)
- 315023 Alabaster Ivory (22.7 kg)
- 315022 Golden Brown (22.7 kg)

COECAL™

- 310125 Buff 25 lb
- 310225 White 25 lb
- 310150 Buff 50 lb
- 310250 White 50 lb



MULTI-SEP™ - Separating Medium

- 343691 Refill Bottle (2 oz; 59 mL)
- 343694 Refill Bottle (8 oz; 300 mL)



COE-SEP™ - Tinfoil Substitute

- 343591 Ounce Bottle (2 oz)
- 343592 Quart Bottle (1 qt)
- 343592 Gallon Bottle (1 gallon)



GC America Inc.

3737 West 127th Street
 Alsip, IL 60803
 800.323.7063
 www.gc.dental/america
 © 2026 GC America Inc.