Prior to use, carefully read the instructions for use. **EN**

GC Fuji TRIAGE™ CAPSULE

RADIOPAQUE GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL IN CAPSULES

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

1. Fissure protection
2. Root surface protection.
3. Hypersensitivity prevention and control.
4. Protection for immature enamel.
5. Temporary filling of endodontic access.
6. Intermediate restorations.

CONTRAINDICATIONS

1. Direct pulp capping.
2. Avoid use of this product in patients with known glass ionomer cement allergies.

COMPOSITION

Powder: Fluoro-alumino-silicate glass, pigment
Liquid: Water, polyacrylic acid, carboxylic acid

DIRECTIONS FOR USE

PINK Shade is command set, WHITE Shade is chemical set only.

Powder / Liquid Ratio (g/g)	0.30 / 0.15
Mixing Time (sec.)	10"
Working Time (23°C/73°F) (min., sec.)	140"
Net Setting Time (37°C/99°F) (min., sec.)	2'30"
Finishing Time (37°C/99°F, from start of mixing) (min., sec.)	6'00"
Finishing Time (if light cured (PINK Command set) (37°C/99°F, from start of mixing) (min., sec.)	4'00"

ISO 9917-1 Glass polyalkenoate cement, Restoration

A. FISSURE PROTECTION

ROOT SURFACE PROTECTION

HYPERSENSITIVITY PREVENTION AND CONTROL

PROTECTION FOR IMMATURE ENAMEL

1. Preparation of the tooth surfaces (e.g. for fissure protection or root surface protection)
- a) After cleaning the tooth surfaces (prophylaxis with pumice and water) in a usual manner, rinse thoroughly with water. Avoid aggravating the operculum.

Note:
If extra retention is desired, application of CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) is recommended. The conditioner should be rinsed off thoroughly after application.
Do dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. A-1). DO NOT DESICCAT. Best results are obtained when prepared surfaces appear moist (glistening).

2. Mixing

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. A-2).
- b) To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig. A-3).

- Note:
1) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
- 2) The capsule should be activated just before mixing and used immediately.
- c) Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (≈ 4.000 RPM) (Fig. A-4).

3. Placement

- a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- b) Click twice to prime the capsule, then dispense (Fig. A-5). The working time is 1 minute 40 seconds from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
- c) Extrude the mixture onto the tooth surface (Fig. A-6) then use a brush to spread a thin film of GC Fuji TRIAGE CAPSULE directly over the root surface or hypersensitive area or over the occlusal surface and into the pits and fissures.

- Note:
1) If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.
- 2) To adjust the direction of the light curing device, turn the capsule towards you and turn the capsule body.
- 3) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.

- d) After placement, when the material starts to lose the glossy appearance (or after curing with the light curing device), apply GC Fuji VARNISH (flow dry) or GC Fuji COAT LC (light cure) to the sealed area and dry the margins using a cotton pellet or sponge (Fig. A-7).
- e) Finishing under air/water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cured. Use a superfine diamond bur or a silicone finishing point (Fig. A-8).

- f) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig. A-9).

B. TEMPORARY FILLING OF ENDODONTIC ACCESS

1. Cleaning the pulp chamber
- a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).

- b) Fill the chamber with a cotton pellet (Fig. B-2).

2. Mixing

- See directions in A (above), Section 2.

3. Placement

- See directions in A, Section 3, a and b.

- b) Extrude the mixture directly over the cotton pellet (Fig. B-3).

- Note:
If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

- c) Moisture protection and Finishing
- See directions in A, Section 3, d, e and f.

C. INTERMEDIATE RESTORATIONS

1. Caries removal
- a) Remove caries and loose debris with hand instruments.
- b) For better retention, it is recommended to gently clean the carious surface with CAVITY CONDITIONER for 10 seconds or DENTIN CONDITIONER for 20 seconds.

- c) Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1). DO NOT DESICCAT. Best results are obtained when prepared surfaces appear moist (glistening).

2. Mixing

- See directions in A (above), Section 2.

3. Placement

- See directions in A, Section 3, a and b.

- b) Extrude the mixture directly over the prepared lesion or both surface (Fig. C-2). Form the contour and if possible cover with a matrix.

- Note:
If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

- c) See directions in A, Section 3, d, e and f.

- NOTE:
The initial set of GC Fuji TRIAGE CAPSULE PINK can be accelerated using the energy from a visible light curing device (LED / Halogen > 700 mW/cm²).

SHADE

PINK, White

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

PACKAGING

50 capsules
Average contents per capsule: 0.30 g powder and 0.15 g (0.12 mL) liquid
Minimum net volume of mixed cement per capsule: 0.13 mL

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

4. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<https://www.gc.dental/europe> or for the Americas

<https://www.gc.dental/americas>

They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gcc@gc.dental

Undesired effects-Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by you or your patients, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 10/2024

Rx Only

MANUFACTURED BY
GC CORPORATION
70-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Asip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
70-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interhavenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 79 14 00
GC AMERICA INC.
3737 West 127th Street, Asip, IL 60803 U.S.A.
TEL: +1 708 267 0800
www.gc.dental/america

GC AUSTRALIA PTY LTD
Researchpark Haasrode-Leuven 1240, Interhavenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 79 14 00
GC AMERICA INC.
3737 West 127th Street, Asip, IL 60803 U.S.A.
TEL: +1 708 267 0800
www.gc.dental/america

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