GCLT LABORATORY PUTTY

VINYL POLYSILOXANE IMPRESSION/DUPLICATING MATERIAL
PUTTY-TYPE O

APPLICATIONS

- Matrix for denture and partial denture repairs.
- Duplicating gypsum models.
- Index for Gradia Micro-Ceramic composite system.
- Implant template matrix.
- Incisal matrix guides for provisional and final restorations.

DIRECTIONS FOR USE

1. Clean and dry hands thoroughly.
2. Using the color-coded scoops, take equal amounts of Base and Catalyst.
3. To mix, knead with dry fingers for 30 seconds, until a uniform color is obtained.
4. Using finger pressure, apply putty to desired area (Gypsum model, wax set-up for a denture or partial denture, crown wax-up, or when indexing for Gradia) and press into place.
5. Allow material to set.
6. Remove carefully.

NOTES

1. When mixing putty, avoid contact with the following materials. They may interfere with setting.
 - Latex Gloves
 - Sulfur
 - Moisture
 - Oil
 - Acrylates
 - Glycerol
2. Close containers after each use with their original lids to prevent cross-contamination.
3. Store in a dry cool place.
4. Avoid getting GCLT Putty on clothing.
5. Shelf life is 24 months from the manufacturing date.

PHYSICAL PROPERTIES (Average)

Mixing Time	30"
Total Working Time	1'00"
Setting Times from End of Mix:	
Recommended Removal From Model	2'30" - 3'30"
Fully Rigid	6'00"
Recovery from Deformation (%)	98.8
Maximum Strain in Compression(%)	≤1.5
Linear Dimensional Change After 24 Hrs. (%)	≤0.2

CONTENTS	STANDARD PACKAGE
Putty	1 Base - 1 Catalyst (1.8kg/1.0L each)
Accessories	1 + 1 Measuring Spoons

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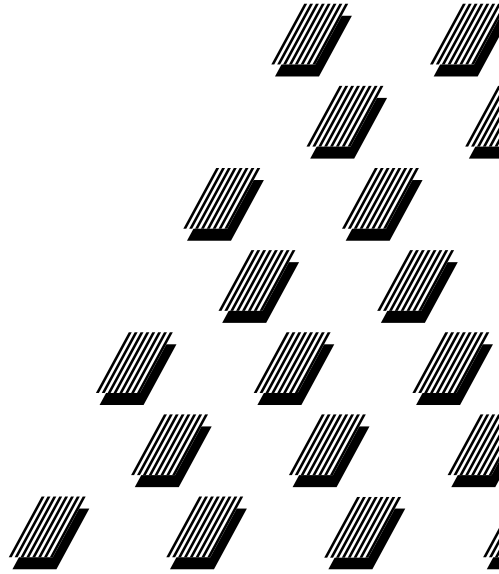
GCLT LABORATORY PUTTY

VINYL POLYSILOXANE IMPRESSION/
DUPLICATING MATERIAL

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GC AMERICA INC.
Alsip, IL 60803 U.S.A.

DISTRIBUTED by
GC CORPORATION
76-1 HASUNUMA-CHO, ITABASHI-KU, TOKYO 174-8585, JAPAN
GC AMERICA INC.
3737 WEST 127TH STREET, ALSIP IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.
19 LOYANG WAY, #06-27 SINGAPORE 508724