## G-Premio BOND TECHNIQUE GUIDE

Prior to use, carefully read the instructions for use.

### SELECT ETCHING MODE



Etch for 10-15 sec., rinse for 5 sec. and dry.

### BONDING PROCEDURE



Apply G-Premio BOND and wait 10 sec.



Start placing composite.

# MAX Air 5 Sec

Dry.



,'GC.'



Light cure.

Light	curing	time

Light curing unit	Distance from light guide tip		
	<10mm	>10mm	
Halogen / LED (700 mW/cm²)	10 sec.	20 sec.	
High power LED (>1200 mW/cm <sup>2</sup> )	5 sec.	10 sec.	

## G-Premio BOND TECHNIQUE GUIDE

#### INTRAORAL REPAIR











Light cure.

Roughen surface. Rinse and dry.

Apply Ceramic Primer II on ceramic BOND to all surfaces. Dry.

Apply G-Premio surfaces and wait 10 sec

### HYPERSENSITIVITY TREATMENT



Apply G-Premio BOND Wait 10 sec.







Light cure.



Remove unpolymerized layer.

13009178 270451DK

Dry.