

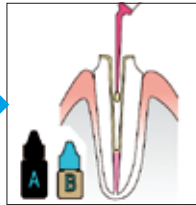
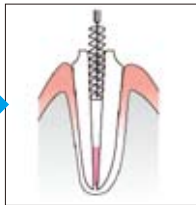
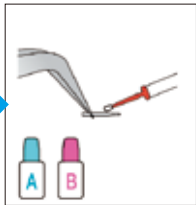
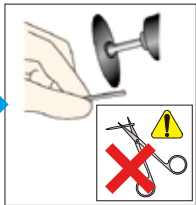


GRADIA CORE & GC FIBER POST Technique Guide

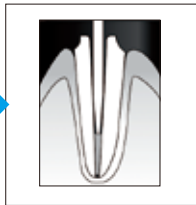

Carefully read the instructions for use.

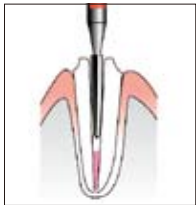

1. Prepare post space according to the standard technique.

2. Trial fit the post.

3. Cut the post to the desired length.







4. Treat the post with silane agent such as GC Ceramic Primer.

* Mix A and B for 5 sec. apply and dry.

5. Rinse and dry with air syringe and paper point.

6. Treat the surface to be bonded to with GRADIA CORE SELF-ETCHING BOND.

* Mix A and B for 5 sec. apply and wait 30 sec.

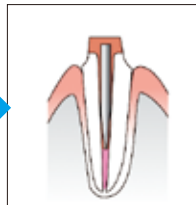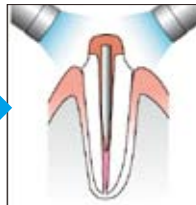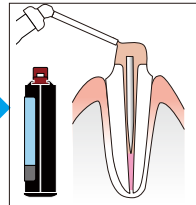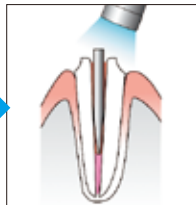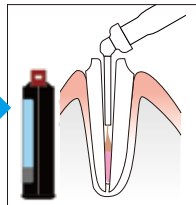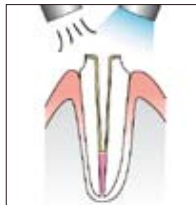

7. Dry with medium air pressure for 10 sec. Light cure for 10 sec. See tables in instructions for use for the light curing times.

8. Dispense GRADIA CORE into the post space.

9. Seat the post. Light cure for 3 sec.

10. Build up with GRADIA CORE.

11. Light cure each surface for 10 sec.

12. Self cure for 5 minutes if the light cannot reach the material.