


## TEMPSMART Technique Guide (Chair side)

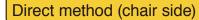




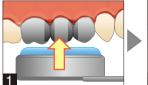

Take impression before preparation of abutment tooth.

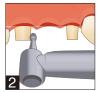

Preparation of abutment tooth

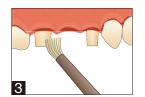


Block out and apply separating medium



Bleed cartridge a pea-sized amount. Place TEMPSMART


into the impression.

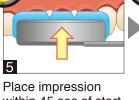

within 45 sec of start of dispensing material

between 2 and 2.5 mins after dispensing material. Remove cured material if residue remains.

Remove impression

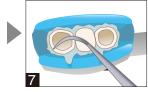


Trim excess material



Remove temporary from impression, light cure on occlusal surface for 5 sec. per tooth1 (Self cure : wait 5 min after dispensing material).



Finish and polish surface. Control bite to minimize occlusal stress and load.

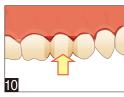

Seat temporary


Material for margin adaptation and repairing


Margin adaptation: TEMPSMART (Light curing is necessary), self curing acrylic resin, composite resin Repair: TEMPSMART (Light curing is necessary), self curing acrylic resin, flowable / injectable composite resin

Expose new surface to prevent low bond strength due to saliva contamination. CERAMIC PRIMER II enhances bond strength.


<sup>1</sup>Do not perform final cure on tooth in order to minimize heat transfer.

## TEMPSMART Technique Guide (Laboratory)

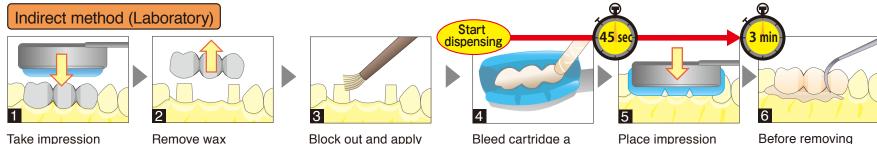

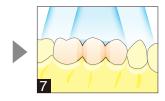


Light cure for 1 min with lab curing unit, keeping the temporary on the model. (Self cure: wait 5 min from start of dispensing.

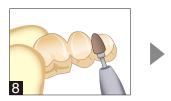

separating medium

Finish and polish. Control bite to minimize occlusal stress and load.

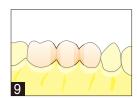

pea-sized amount.

Place TEMPSMART

into the impression.

Complete restoration

Place impression within 45 sec of dispensing material, and remove it 3 min after start of dispensing. Before removing excess material, remove the cured temporary. This helps retrieval from model.





after wax up.

Recommended for optimal performance, store in a cool and dark place (4 - 25°C / 39.2 - 77.0°F) away from high temperatures or direct sunlight.



GC CORPORATION TOKYO, JAPAN