

TRECLEAN

TRAY CLEANER FOR REMOVING ALGINATE

FEATURE

Effective in removing residual alginate impression material from impression trays.

USE

- a. Add 2 scoops of powder to 1 liter of water. Let it dissolve.
- b. Remove the bulk of residual alginate from the tray. Place it in the solution (within 2 hours).
Avoid immersing the tray for a long time, because there is a risk of discoloration of the tray.
- c. After immersing the tray, brush it under running water for additional removal of the residue if necessary.
- d. Rinse the tray well with water and sterilize for reuse.

NOTES

1. If the solution should get into eyes, immediately flush with water and seek medical attention.
2. Take care not to inhale powder dust. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
3. As the solution contains very weak alkali, it may cause skin irritation to some persons. It is advisable to wear latex or plastic gloves. In use of the solution for a long time, also wear such gloves.
4. Do not mix with any other product.
5. After scooping the powder, be sure to close the powder container tightly.
6. Use TRECLEAN at room temperature and store it in a cool place.
7. Do not use TRECLEAN for any purpose other than removing residual alginate impression material from impression trays.