

G-ÆNIAL UNIVERSAL INJECTABLE

CLEAR ALIGNER PARTNER



G-aenial Universal Injectable is a high strength injectable composite resin. This is ideal for creating clear aligner attachments with excellent clinical outcomes (exceptional aesthetics, no flash, ideal flow without being runny, high strength and no porosities) and most importantly a significant reduction in application time helping further improve the cost effectiveness of the aligner treatment procedure.



Initial presentation, prior to aligner attachment placement.



G-aenial Universal Injectable enamel shade JE (Junior Enamel), used to fill the attachment space.



After applying resin adhesive, the template is placed in the mouth and the G-aenial Universal Injectable is light cured for 20 seconds and the template is removed.



Attachments in position after template removal.



Completed clear aligner attachments after final polishing.

[View products online](#)

[VIEW PRODUCTS](#)

Images courtesy of Dr Bharat Agrawal, Gold Coast.

DR BHARAT AGRAWAL
Gold Coast

