# MAKE A BETTER IMPRESSION




,'GC,'

## **AROMA FINE PLUS**

FROM POWDER TO PERFECT PASTE IN JUST 20 SECONDS



## **COE Spacer Trays**

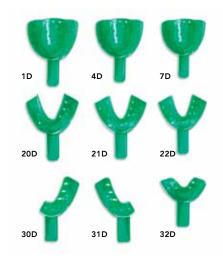

1D Large Upper (Bag of 12) 4D Medium Upper (Bag of 12) 7D Small Upper (Bag of 12) 20D Large Lower (Bag of 12) 21D Medium Lower (Bag of 12) 22D Small Lower (Bag of 12) Partial 30D U/Lft, L/Rt, (Bag of 12) Partial 31D U/Rt, L/Lft, (Bag of 12) Partial 32D Anterior, (Bag of 12)

#### **AROMA FINE PLUS**



- 1kg Fast Set
- 1kg Normal Set



GC Asia Dental Pte Ltd Changi Logistics Centre 19 Loyang Way #06-27 Singapore 508724 Tel +65 6546 7588 Fax +65 6546 7577 www.gcasia.info

> Copyright © 2010 oa AFineP eX 08/10

The new AROMA FINE PLUS powder is incorporated rapidly into water through enhanced powder-water affinity, allowing more spatulation time to obtain a smooth bubble-free mix within 20 seconds.


#### AROMA FINE PLUS





## The reliable properties of AROMA FINE remain unchanged.

#### Accurate impression

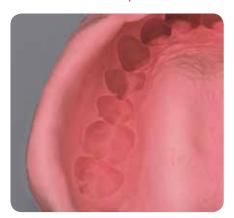

Easy-mixing of AROMA FINE PLUS results in a creamy, bubble-free consistency that captures details accurately.

#### Optimal thixotropic property



AROMA FINE PLUS provides a balance of good drip resistance and fluidity to effectively provide coverage enabling detailed impression taking with minimal discomfort to patient.

#### Smooth model surface




AROMA FINE PLUS has good compatibility with hard stones producing smooth and precise detail reproduction on the gypsum model.

## Aroma Fine Plus Physical Properties

Setting types	Setting time from start of mix (room temp 23°C water temp 23°C)	Holding time in mouth	Compressive strength	Strain in compression	Recovery from deformation
Fast	2 min. 30 sec	1 min. 30 sec	0.7MPa	14.6%	97.4%
Normal	3 min.	2 min.			

## **Working Time**

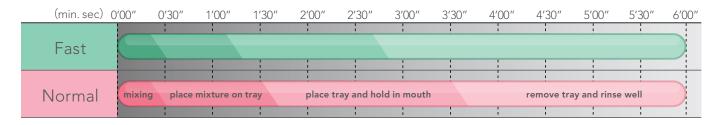

### Directions for use

