Prior to use, carefully read the instructions for use.

CERASMART™ 270

KRAFTABSORBIEREND HYBRID CERAMIC CAD/CAM BLOCK

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

1. Metal free indirect restorations: full crown, inlay, onlay, laminated veneer
2. Implant supported crown

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate polymer.

COMPOSITION

Barium glass, dimethacrylate, pigment, silicon dioxide

PREPARATION DESIGN

When designing restorations, the following minimum dimensions should be maintained:

- Minimum wall thickness should be 1.5 mm, 1.0 mm at margins.
- Prepare margins with deep chamfer or rounded shoulder. (Figure 1)
- For inlays and onlays, all internal edges and angles should be rounded. Avoid having margins in direct occlusal contact with the opposing tooth.
- Minimum thickness of the restoration should be 1.5 mm in pit and fissure areas and 1.5 mm in cusp areas.

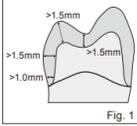
Fig. 1

MILLING

Please refer to respective instructions of the milling unit. Select the milling program for CERASMART270 or composite resin block materials.

- After milling, check the restoration for:
- Discolorations linked to the milling process
 - Protruberances
 - Cracks or chips
- If the restoration is defective, discontinue the manufacture of restoration.

FINISHING AND POLISHING

Finish and polish with coarse and fine silicone point. DIAPOLISHER PASTE can be used for final polishing to obtain high gloss.

NOTE: When adjusting or polishing the restorative, use a dust collector and wear a dust mask to avoid inhaling dust. Wear protective glasses.

CEMENTATION

With Sandblasting technique

- 1) Sandblasting with 25-50 µm alumina 0.15 MPa / 1.5 bar is recommended.

- 2) Blow the restoration with an oil-free air syringe or clean with an ultrasonic cleaner and dry.
- 3) Clean with alcohol to remove oil residue.
- 4) Treat the surface with a silane coupling agent such as CERAMIC PRIMER II or G-Multi PRIMER.
- 5) Cement with an adhesive resin cement such as G-CEM LinkForce.

NOTE: Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-CEM LinkForce, refer to the respective instructions for use of the manufacturer.

With hydrofluoric acid etching

- 1) Treat with hydrofluoric acid (5%) for 60 seconds.
- 2) Wash with water spray or an ultrasonic cleaner and dry.
- 3) Treat the surface with a silane coupling agent such as CERAMIC PRIMER II or G-Multi PRIMER.
- 4) Cement with an adhesive resin cement such as G-CEM LinkForce.

NOTE: (1) Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-CEM LinkForce, refer to the respective instructions for use of the manufacturer. (2) Hydrofluoric acid is for extraoral use only and must not be applied in the oral cavity. (3) Phosphoric acid (35-37%) can be also used for the purpose of cleaning the surface, preferably scrubbing with a microbrush for 10-15 seconds.

ADDITIONAL APPLICATION AND REPAIR

Roughen the bonding area using diamond point or carbide bur. Apply CERAMIC PRIMER II or G-Multi PRIMER to improve adhesion and dry the surface. Apply a bonding agent following its manufacturer's instruction. Proceed with the application of the chosen composite restorative material. G-anial Universal Flo or G-anial Universal Injectable is recommended in order to make the surface of restorative homogeneous as it consists of the same flexible nano ceramic technology as CERASMART270. In this case, the intermediate use of a bonding agent is not necessary.

CHARACTERIZATION

When appropriate, OPTIGLAZE color or GRADIA PLUS Lustre Paint is recommended as surface characterization and glazing technique. Sandblast with 25-50 µm alumina (0.15 MPa / 1.5 bar), clean and dry. Apply CERAMIC PRIMER II or G-Multi PRIMER, dry and apply OPTIGLAZE color or GRADIA PLUS Lustre Paint. OPTIGLAZE color and GRADIA PLUS Lustre Paint can only be cured with a light-curing device with light source having wavelength lower than 430 nm. Refer to the respective instructions for use.

SHADE

5 Vita shades* available in two translucencies: High Translucency (HT): A1 HT, A2 HT, A3 HT, A3.5 HT, B1 HT - Low Translucency (LT): A1 LT, A2 LT, A3 LT, A3.5 LT, B1 LT - One bleach shade (BL)

*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

SIZE

Available in 3 sizes: 12, 14, 14L.

PACKAGE

5 blocks per box

STORAGE

Recommended for optimal performance, store at room temperature (4-25°C / 39.2-77.0°F) away from direct sunlight and high humidity.

CAUTION

1. In rare cases this product may cause sensitivity in some people. If such reactions are experienced, discontinue the use of product and consult a physician.
2. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
3. Do not re-use. Milling partially or fully processed blocks will result in an inaccurate restoration.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for the Americas <http://www.gcamerica.com> They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those noted in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person GC UNITED KINGDOM Ltd. Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS United Kingdom

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