EN
 Please to use, carefully read the instructions for use.

DIAPOLISHER PASTE

DIAMOND POLISHING PASTE FOR COMPOSITE AND PORCELAIN

DESCRIPTION
 DIAPOLISHER PASTE is a diamond polishing paste for direct and indirect use.

INTENDED PURPOSE
 Final high level polishing of composite resin and porcelain restorations.

INDICATIONS FOR USE
 Natural teeth or restoration lacking smoothness and aesthetics. Areas of application:
 - Extraoral and intra-oral polishing of direct and indirect restorations.

CLINICAL BENEFITS
 Restoration of aesthetics and gloss

CONTRAINDICATIONS
 Patients with known allergies to benzoyate preservatives.

INTENDED USERS/SPECIAL TRAINING
 - Dentists
 - No special training required

PATIENT TARGET GROUP
 - Patient with direct or indirect restoration.

PERFORMANCE CHARACTERISTICS
 The product's performance characteristics satisfy the requirements of the intended use.

DIRECTIONS FOR USE
 1. Preparation
 a) Finish and polish restoration using standard techniques.
 b) Hold the syringe upright and remove the wing cap by turning counterclockwise.
 c) Attach the dispensing tip correctly to the syringe by turning clockwise.
 Note:
 Take care not to thread the syringe too tightly. This may damage the screw. Use the syringe to dispense the product, make sure that the screws are free of remnant material and that the application of the product is uniform.

2. After attaching the dispensing tip, protect it with the protective cap and use it in order to avoid contamination or drying out.

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4. Apply a small amount of paste onto an appropriate composite / porcelain polishing brush or cup. Polish all areas using moderate speed and pressure (up to 10,000 rpm). Refer to the respective manufacturer's instructions for use of the brush or cup. After polishing, rinse with water to remove the paste.

Note:
 In case of contact with adjacent natural tooth structure, use moderate speed and pressure up to 5,000 rpm.

DISPOSAL
 Dispose of all waste according to local regulations.

STORAGE
 Recommended for optimal performance, store in a cool and dry place.

CLEANING AND DISINFECTING
 MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients, the device requires mid-level disinfection. Immediately after use, inspect device and label for deterioration. Discard device if damaged.

Note:
 Do NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a low-level registered healthcare grade infection control product according to regional / national guidelines.

SIDE EFFECTS
 There are no known side effects to date.

RESIDUAL RISKS
 There are no known residual risks to date, other than those specified as part of contraindications, limitations, precautions, warnings and side effects.

PRECAUTION, WARNINGS AND LIMITATION OF USE
 Do not use if the prescribed application technique cannot be applied.

1. To prevent splatter of paste and / or excessive heat, generate always use rotary instruments at speeds below 5,000 rpm for direct and indirect restoration, and below 5,000 rpm for natural tooth. Wear protective eye glasses during use.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Use of plastic or rubber gloves during use to prevent possible sensitivity.

4. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid irritation of the eyes.

5. The dispensing tip cannot be sterilized in an autoclave or chemically.

6. In rare cases the product may cause sensitivity in some people. In any such reactions are experienced, discontinue the use of the product and refer to the physician. To access full personal protective equipment (PPE) such as gloves, face masks and safety eyewear, please refer to the instructions.

7. In case of inhalation, take the affected person into fresh air and keep them calm.

8. In case of swallowing, rinse out mouth and then drink plenty of water. If symptoms persist, consult a doctor.

9. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always use appropriate familiarization with the safety data sheets available at: <https://www.gdentalamerica.com> or for the Americas: <https://www.gdentalamerica.com>

10. This can also be obtained to your supplier.

PACKAGES
 Kit contains: DIAPOLISHER PASTE 2 g x 1
 Dispensing tip (needle type) 20 pieces
 Refill: Dispensing tip (needle type) 20 pieces

Reporting obligation according to EU MDR
 In the case of a serious incident related to the device, please report it to GC Europe N.V. (vigilance@gdental.eu) and to the competent authority of the Member State in which the incident occurred.

Note:
 The current Instructions for Use are available in the download section of the <https://www.gdentalamerica.com> website. To access full appropriate version, please select your region from the footer.

UK RESPONSIBLE PARTY
 UK UNITED KINGDOM Ltd
 Copsey Court Newport Pagnell Buckinghamshire
 MK16 8J5 United Kingdom
 Last revised: 12/2025

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