


DISPENSING INTO DISPOSABLE DISPENSING DISH (BOTTLE)

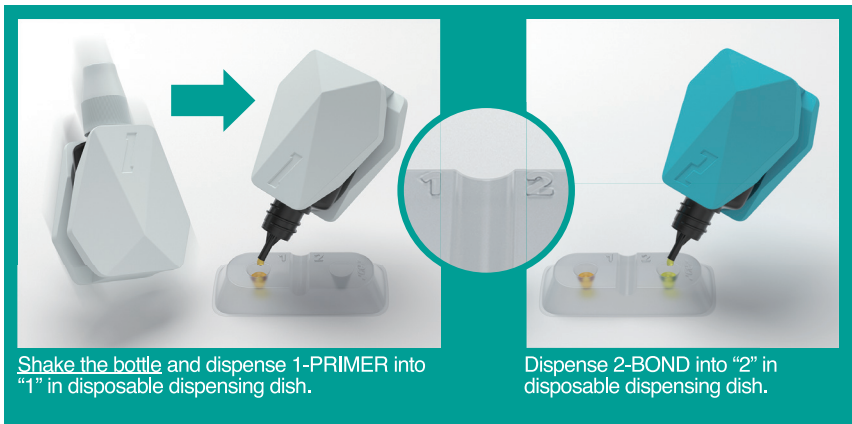

Shake the bottle and dispense 1-PRIMER into "1" in disposable dispensing dish.

Dispense 2-BOND into "2" in disposable dispensing dish.



Bottle cover "1" and "2" colors correspond to product bottle caps and labels. Please ensure bottle covers are applied correctly prior to use.

Bottle covers help insulate G2-BOND Universal from excess heat and give greater control of dispensing drops to minimize wastage. Their use is highly recommended. Bottle covers may be reused with subsequent refills. Clean with water or wipe with alcohol. Do not sterilize, autoclave, or clean with chemical disinfectants.

ETCHING MODE SELECTION



Etch for 10-15 sec., rinse for 5 sec. and dry.

Light curing time

Light curing unit	Distance from light guide tip	
	<10mm	>10mm
Halogen / LED (700-1200 mW/cm ²)	10 sec.	20 sec.
High power LED (more than 1200 mW/cm ²)	5 sec.	10 sec.

BONDING PROCEDURE (INDICATIONS FOR USE [1])



Apply 1-PRIMER and wait 10 sec.



Dry.



Apply 2-BOND.



Make a uniform bond film.



Light cure.


INTRAORAL REPAIR (INDICATIONS FOR USE [3] [4])



Roughen surface. Rinse and dry.



Apply G-Multi PRIMER on ceramic surface Dry

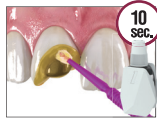

Apply 1-PRIMER to all surface and wait 10 sec.



Dry.



Apply 2-BOND to all surface.



Make a uniform bond film.



Light cure.

SEALING OF TOOTH PREPARATION (INDICATIONS FOR USE [6])



Apply 1-PRIMER and wait 10 sec.



Dry.



Apply 2-BOND.

Option



Make a uniform bond film.



Light cure.



Remove unpolymerized layer.



Since 1921
Towards Century of Health

GC Australasia Dental Pty Ltd
1753 Botany Road Banksmeadow
NSW 2019 Australia

T: +61 2 9301 8200
E: info.australasia@gc.dental
www.gcaustralasia.com

13010519
241102SK