EN Prior to use, carefully read the instructions for use.

G-Multi PRIMER

PRIMER FOR GLASSKERAMIC, HYBRID CERAMICS, ZIRCONIA, ALUMINA, COMPOSITES AND METAL BONDING

G-Multi PRIMER is a priming agent designed to treat the fractured surfaces of glass and glass-ceramics (e.g. CERASMART), zirconia, alumina, composite and metal restorations. This is to promote a strong bond to non-toxic materials and to avoid delamination in case of luting procedures or direct composites in case of repairs.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Priming of adherent surface of indirect restorations made of glass ceramic, hybrid ceramics (e.g. CERASMART), zirconia, alumina, composites and metal before luting procedures.
- Intra-oral and extra-oral repair of fractured indirect restorations made of ceramic, hybrid ceramics, zirconia, alumina, and composites.

CONTRAINDICATIONS

Avoid use of the product in patients with known allergies to methacrylic monomers, methacrylates polymer or ethanol.

DIRECTIONS FOR USE

- For luting procedure.
 - Treat adherent surface of restoration according to the instructions of the manufacturer's instructions and then clean it using a suitable cleaning method. We recommend cleaning in an ultrasonic bath. After rinsing with distilled water, then rinse and dry with an oil-free air syringe.
 - Dispense one drop of G-Multi PRIMER into a dispensing device.
 - Apply a thin layer to the adherent surface of the restoration using a micro-pip applicator and dry with an air syringe.
 - Apply suitable resin-bond luting cement to the primed surface of the restoration. For these steps, we refer to the manufacturer's instructions for use of the corresponding luting cement.
- Intra-oral Repair.
 - Roughen adherent surface using a coarse diamond bur in carbundium oil and dry.
 - Treat adherent surface of the restoration according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.
 - Dispense one drop of G-Multi PRIMER into a dispensing dish.
 - Apply a thin layer to the fractured surface of the restoration using a micro-pip applicator and dry with an oil-free air syringe.
 - Apply the repair by using a light-cured adhesive (e.g. G-Premio BOND) and a light-cured composite (e.g. G-anial, Essentia), referring to their respective instructions for use.
- Intra-oral and extra-oral repair of fractured indirect restorations made of ceramic, hybrid ceramics, zirconia, alumina, and composites.
 - Treat the surface of the restoration with a coarse diamond bur and dry.
 - Retreat the surface of the restoration with the instructions of the manufacturer's instructions and then clean it using a suitable cleaning method.
 - Dispense one drop of G-Multi PRIMER into a dispensing dish.
 - Apply a thin layer to the fractured surface of the restoration using a micro-pip applicator and dry with an oil-free air syringe.
 - Apply the repair by using a light-cured adhesive (e.g. G-Premio BOND) and a light-cured composite (e.g. G-anial, Essentia), referring to their respective instructions for use.

STORAGE

Recommended for optimal performance, store at room temperature (4-25°C / 39.2-77.0°F).

CAUTION

- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
 - G-Multi PRIMER is flammable and volatile. Avoid fire, near open flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
- In case of contact with oral tissues, immediately remove with a cotton swab or gauze. Rinse immediately after luting or repairing.
- In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with water and water. In case of contact with eyes, flush immediately with water and seek medical attention.
- Replace the cap of bottle immediately after use. If condensation (water) is present on the bottle of G-Multi PRIMER, immediately wipe it.
- Avoid inhalation or ingestion of material.
- Do not mix with any other product.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at <http://www.gepore.com> or The Americas <http://www.gamcerica.com>. They can also be obtained from your supplier.

CLEANING AND DESINFECTING

USE OF DELIVERY SYSTEMS is advised to avoid cross-contamination between patients. This device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Do not use if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered disinfectant. Rinse thoroughly with product according to regional/national guidelines.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, or reaction or similar event experienced by use of this product, including those not listed in this instruction for use, please report them directly to the manufacturer's support system, by selecting the proper authority of your country accessible through the following link: <https://ec.europa.eu/growth/smds/media-devices/contacts>, as well as to our internal vigilance system: vigilance@gepore.com. In this way you will contribute to improve the safety of this product.

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- AB do inwazyjnej naprawy
- AG do inwazyjnej naprawy