




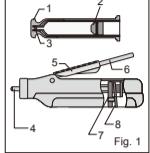
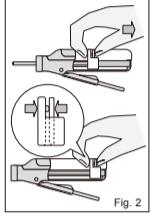
IDENTIFICATION OF PARTS
BEZEICHNUNG DER TEILE
IDENTIFICATION DES PARTIES
I COMPONENTI
IDENTIFICACIÓN DE LAS PARTES

DIRECTIONS FOR USE
GEBRÄUCHSANWEISUNG
MODE D'EMPLOI
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO

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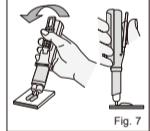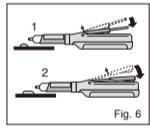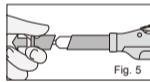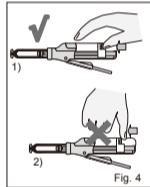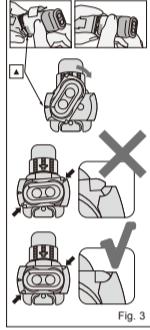Prior to use, carefully read
the instructions for use.
EN

Paste Pak Dispenser

For use only by a dental professional in the recommended indications.

Paste Pak Delivery System consists of Paste Pak Dispenser and the special Paste Pak cartridge. Paste Pak Dispenser enables you to dispense material easily and correctly from the cartridge.

RECOMMENDED INDICATIONS
Dispensing material from GC special Paste Pak cartridges.

IDENTIFICATION OF PARTS (Paste Pak Cartridge)

1. Cover 2. Inner plunger 3. Cartridge tip
(Paste Pak Dispenser)
4. Piston 5. Adjusting guide 6. Lever
7. Sliding block 8. Piston release (Fig. 1)

Directions for use

1. Remove the fixing tape placed between the main part and lever of the dispenser.
2. Remove the V shaped board from the underside of the dispenser.

DIRECTIONS FOR USE

1. Make sure that the piston is completely retracted into the dispenser. (Fig. 2)

Note:
To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them back together.

2. Paste Pak Cartridge Loading (Fig. 3)

- a. Ensure the mark on the cartridge lines up with the mark on the Paste Pak Dispenser as illustrated.

b. Then turn the cartridge fully to the right position.

Note:

To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.

3. After cartridge loading, push the piston release forward until it stops. (Fig. 4)

- a. Push the piston release forward.

b. Do not push the upper side.

4. Remove the cover. (Fig. 5)

5. a) Depress the lever to dispense the required amounts of pastes onto the mixing pad.

Note:

- 1) Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the pastes in order to prevent the incorporation of air bubbles into the material.

- 2) The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed. (Fig. 6)

1. Position of adjusting guide for a smaller quantity

2. Position of adjusting guide for a larger quantity

- b) While depressing the lever, level and cut off the extruded material by moving the cartridge and dispenser to an upright position on the mixing pad. (Fig. 7) When the lever is released after dispensing, residual pastes at the tips are retracted into the cartridge. Replace the cartridge cover.

Note:

- 1) The amount of residual material in the cartridge can be confirmed by the position of the sliding block on the underside of the dispenser. If the cartridge has translucent tips, the residual amount can also be confirmed by the position of the inner plunger seen through the cartridge tips.

- 2) Do not drop the dispenser with the cartridge. Otherwise the cartridge may be damaged.

Cleaning / Sterilization of Paste Pak Dispenser

- 1) Clean the dispenser and adjusting guide with gauze soaked in alcohol.

- 2) The dispenser may be steam autoclaved at 121-135°C (250-275°F) for 10 to 30 minutes or disinfected using alcohol. (Please remove the adjusting guide and the cartridge before autoclaving the dispenser). The use of various chemical disinfectants / sterilants may cause damage to the dispenser and therefore is not recommended.

Note:

- After autoclaving, the piston release of the dispenser may not easily be moved forwards and backwards by fingers. In such a case, slide the piston release forward until it stops. Apply a very small quantity of petroleum jelly onto the entire surface of the piston. Do not apply excess petroleum jelly. Then move the piston release back and forth a few times to ensure smooth movement.

PACKAGE

1 piece

CAUTION

1. In rare cases metal may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product (nickel-plated aluminum) and refer to a physician.

2. Store Paste Pak Dispenser away from chemicals such as iodine tincture containing halogen element. Otherwise the dispenser may discolor.

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
as well as to our internal vigilance system:
vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

Last revised: 10/2019

Rx Only

DISTRIBUTED by
GC CORPORATION
76-1 Hasumuna-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
9797 West 127th Street, Alsip, IL 60603 U.S.A.
TEL: +1-708-537-0900

www.gcamerica.com

GC SOUTH AMERICA
Rua Haddock, 399, São Paulo, SP, BRASIL
CEP 04520-000 - São Paulo, SP, Brazil
CNPJ 08.278.999/0001-61

RESP. TEC: Mayara de Santos Ribeiro - CROISP 105.982

GC ASIA DENTAL PTE. LTD.
5 Tamplines Central 1, #08-01 Tamplines Plaza, Singapore 529541

TEL: +65 6546 7575

GC DENTAL ALASKA DENTAL PTY. LTD.
1703 Kotany Rd, Bankameadow, NSW 2019, Australia

TEL: +61 2 9301 8200

MANUFACTURED by
SHIODA DENTAL CORPORATION
53 Tohbara, Nasu-karasuyama, Tochigi, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60603 U.S.A.
PRINTED IN JAPAN

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