




GC SMART GUIDE


CLICK TO SELECT A PRODUCT CATEGORY




Preventive &
Remineralisation

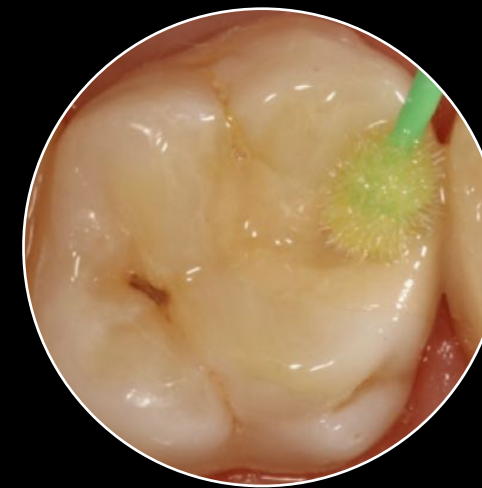

Glass Hybrid
Restorative System

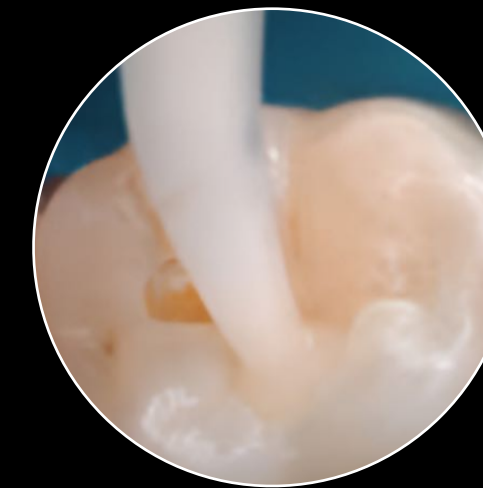

Conventional & Resin-
Modified Glass Ionomers

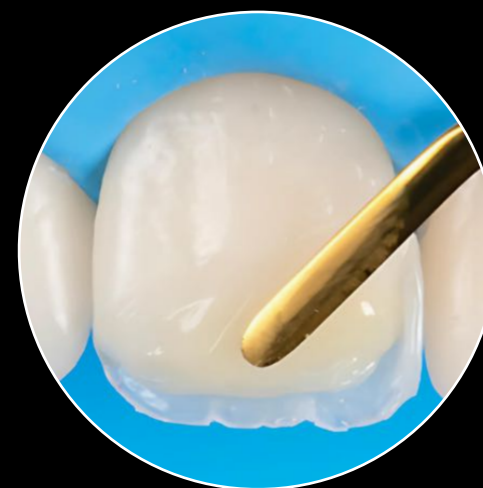

Composites, Bonds
Finishing & Polishing



Luting
Cements



Impression &
Reline

Follow Us



SmartGuide
Instructions



Contact a GC
Product Specialist



CPD
Newsletter




GC Australasia
Website



About GC

GC PREVENTIVE PRODUCTS

Click a product image to view product information

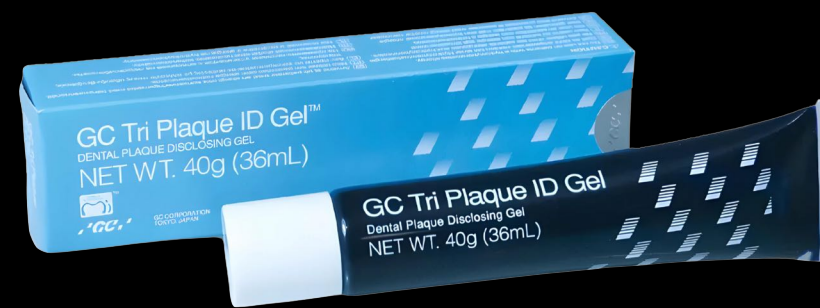

Tri Plaque ID Gel



Dry Mouth Gel



MI Varnish



RECALDENT® Gum



GC Tooth Mousse & Tooth Mousse Plus

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE




GC Tri Plaque ID Gel



Clinical Indications

- Differentiate between new, old and mature plaque within the biofilm
- Identify active **high risk** acid producing bacteria where pH is below 4.5
- Improve patient motivation and education
- Compatible with guided biofilm removal protocols
- Identify high risk acid producing bacteria where pH is below 4.5
- Enables more targeted oral hygiene instruction
- Easy to apply and wash off

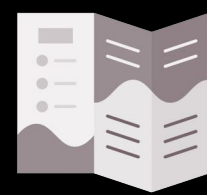


[View product brochure](#)



[View instructions for use](#)

How does Tri Plaque ID Gel work?

GC Tri Plaque ID Gel contains sucrose and pigments that are able to penetrate and stain the plaque biofilm

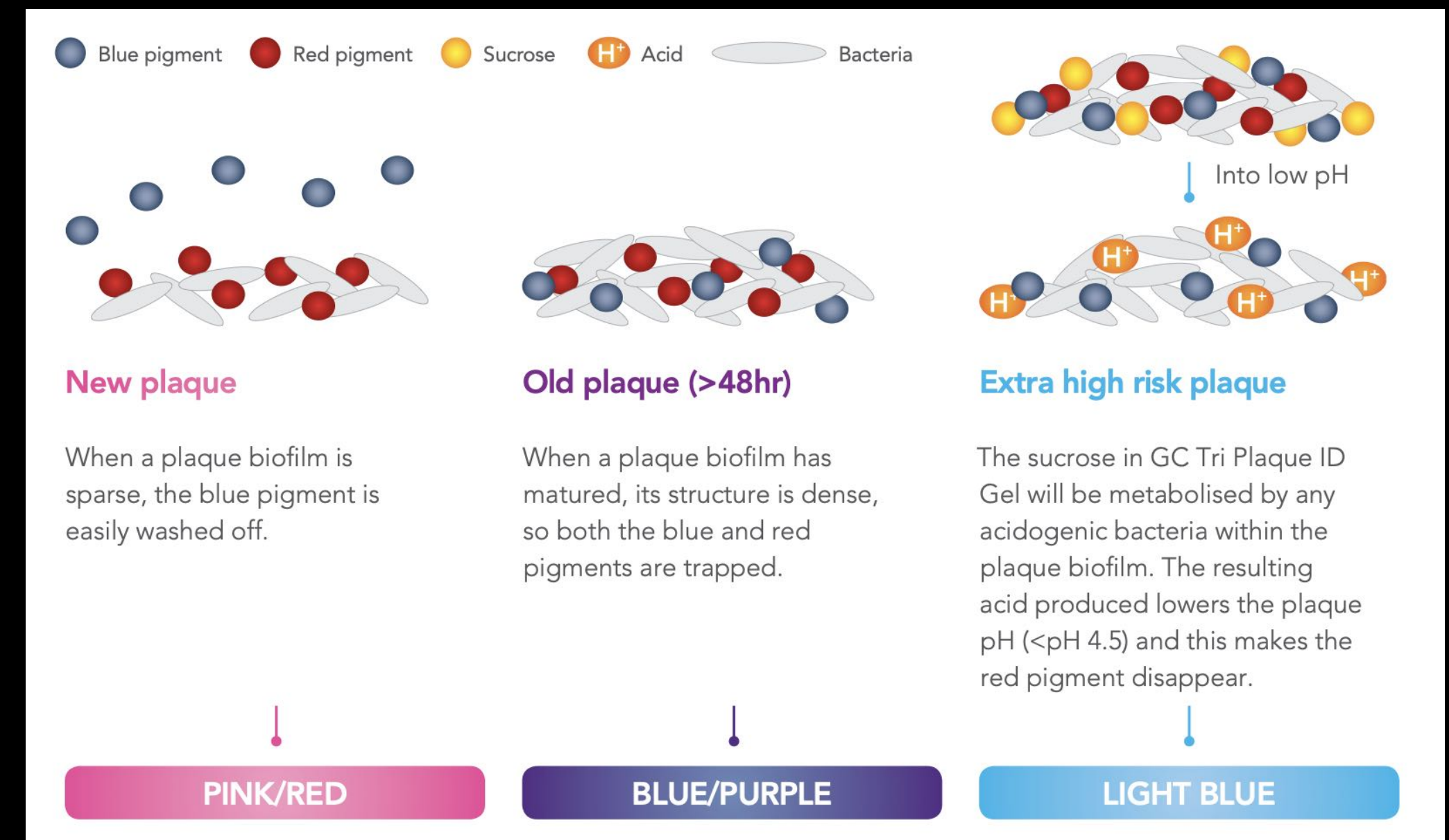

Prof. Ian Meyers



GC Dry Mouth Gel

Dry Mouth Gel is a unique, sugar free product designed to help relieve dry mouths and provide long lasting comfort and a soothing effect for patients.

Uniquely, unlike most saliva substitutes, Dry Mouth Gel has a neutral pH, meaning that effective symptomatic relief can be provided, whilst preserving oral pH within the safe range to prevent demineralisation.

Clinical Indications

Provision of comfort and relief for dry mouth conditions

Provides a protective coating for teeth and oral tissues

Control of halitosis

Up to 4 hours relief from symptoms of dry mouth

Sugar free & near neutral pH

Five flavours - Raspberry, Mint, Orange, Lemon and Fruit Salad

Doesn't interfere with fluoride products or GC Tooth Mousse



Dry Mouth Risk Factors Examples


Dehydration, ageing, anxiety / stress, mouth breathing, snoring, tobacco

Systemic Conditions Examples

Diabetes, stroke, Alzheimer's disease, Sjogren's syndrome, Salivary gland hypofunction, rheumatoid arthritis, nerve damage to your head or neck

Medication Related Examples

Chemotherapy or radiotherapy, antidepressants, antihistamines, decongestants, diuretics

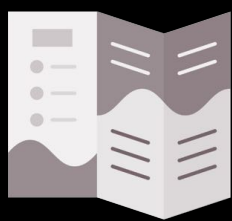

[View product brochure](#)




[View instructions for use](#)



[View On-Demand Webinar \(1 CPD\)](#)



[View Dry Mouth Gel Patient Video](#)

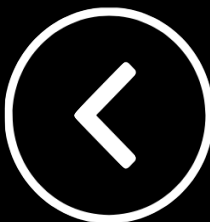


MI Varnish™



Clinical Indications

- Contains 5% NaF enhanced with 2% RECALDENT (CPP-ACP)
- CPP-ACP is derived from milk casein
- Releases bioavailable fluoride, calcium and phosphate
- Adheres to teeth & seals exposed dentinal tubules | Sets on contact with saliva
- Suitable for 3-6 monthly application in children, adolescents and adults
- Minimises tooth sensitivity and strengthens enamel
- Neutral pH of 6.6 enhances enamel acid resistance and inhibits demineralisation
- Simple application applicator brushes and flows easily in hard to reach areas
- Does not clump or coagulate when exposed to saliva
- Neutral shade with natural translucency

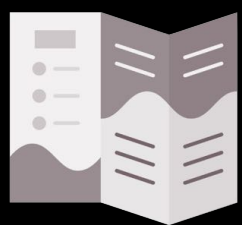

[View product brochure](#)



[View instructions for use](#)



[View Technique Guide](#)

Less is more


Apply a thin uniform layer for maximum benefit and to minimise clumping.



Correct: Uniform thin layer



Incorrect: Too much, too thick

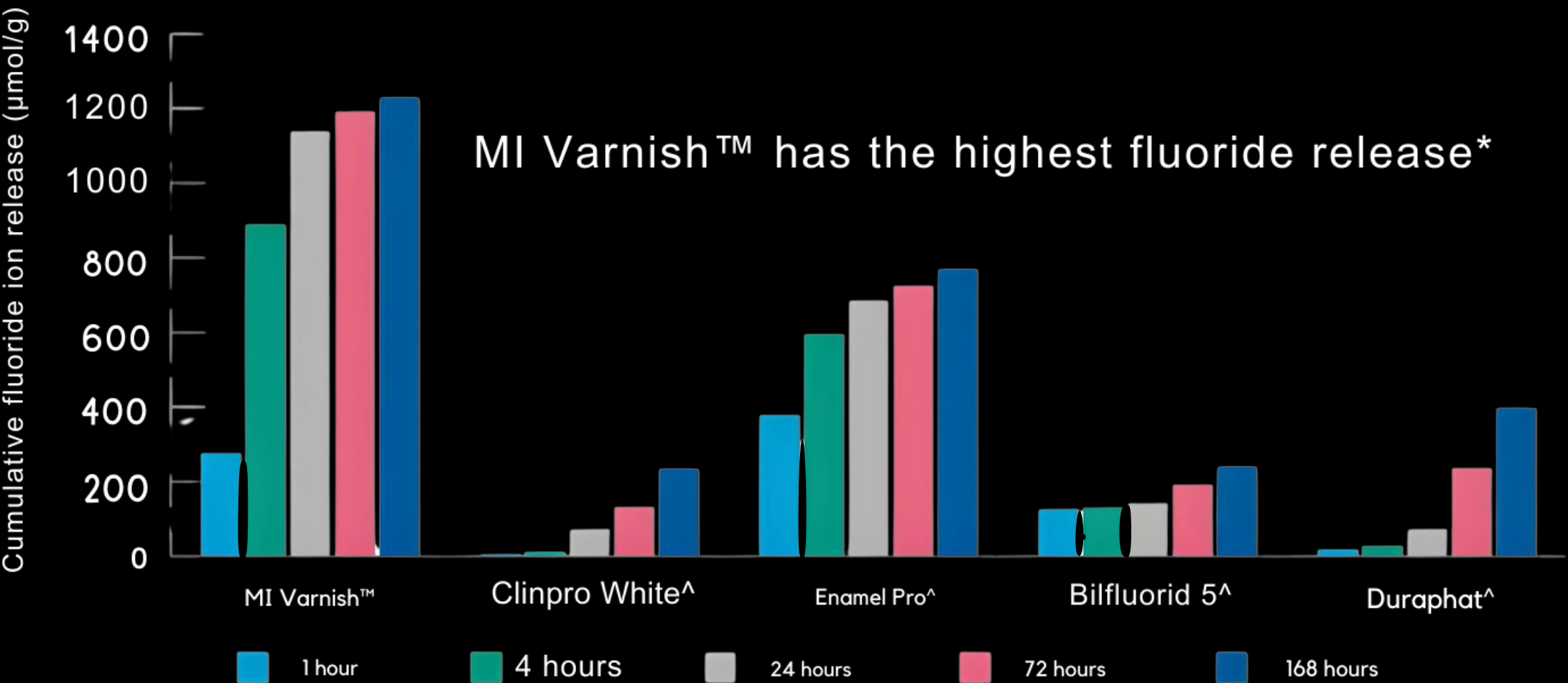

Do not use MI Varnish™ on patients with proven or suspected milk protein allergies

Do not use MI Varnish™ on patients with ulcerative gingivitis and stomatitis

RECALDENT and RECALDENT logo are trademarks of Mondelēz International group used under license.



GC Tooth Mousse™ & GC Tooth™ Mousse Plus



RECALDENT and RECALDENT logo are trademarks of Mondelēz International group used under license.

Clinical Indications, Features and Benefits

- Rapidly acid neutralisation & buffering from plaque bacteria, reflux, food and drinks
- Increases salivary concentration of bioavailable calcium, phosphate and fluoride
- Promotes remineralisation to protect, strengthen and brighten enamel
- Prevents tooth decay & erosion
- Prevents dentine sensitivity from erosion, gingival recession and tooth whitening
- Prevents and reverses white spot lesions during *and* after orthodontic treatment
- Contains 10% RECALDENT™ (CPP-ACP) - derived from milk
- Tooth Mousse is suitable for children and adults of all ages
- Tooth Mousse Plus contains 900 ppm fluoride (suitable for adults and children over the age of 6)

Seven delicious flavours and up to 160 applications per tube provides patients with an economical and tasty way to protect their oral health



View Tooth Mousse patient brochure



View Tooth Mousse Plus patient brochure



View Tooth Mousse instructions for use



View Tooth Mousse Plus instructions for use



View Tooth Mousse FAQ's



View Tooth Mousse Patient Video



View Tooth Mousse Plus Patient Video



View Tooth Mousse Plus For Ortho Patient Video



View On-Demand Clinical Presentation (1 CPD)

The higher the risk, the greater the protection that is required



Min. once a day after brushing
e.g. prevention, children, adolescents, adults, elderly,



Twice a day after brushing
e.g. orthodontics, poor diet, poor oral hygiene, pregnancy, tooth whitening, caries risk, WSL



As often as required
e.g. altered saliva flow, xerostomia, medications, erosion, poly-pharmacy, radiotherapy, chemotherapy, periodontal disease, high caries risk, hypomineralisation

Contraindications - not to be used by patients with known or suspected allergy to

- Milk Protein (casein only)
- Benzoates (preservative)
- Soy bean (doesn't apply to Milkshake Flavour)

www.toothmousse.com.au



Recaldent® Gum

Chewing Recaldent® Advanced sugar-free gum with long lasting flavours helps stimulate saliva flow. Saliva keeps our mouths healthy and is nature's way of protecting our teeth.

Benefits


Stimulates saliva flow

Sugar-free

With RECALDENT (CPP-ACP) casein phosphopeptide-amorphous calcium phosphate




Do not use if you have allergy to cow's milk proteins or soybean derived products.
Recaldent® Advanced chewing gum contains PHENYLALANINE.
Excess consumption may have a laxative effect.

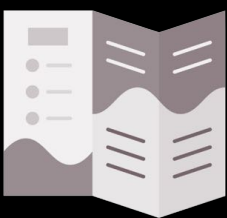

[View product brochure](#)



RECALDENT and RECALDENT logo are trademarks of Mondelēz International group used under license.

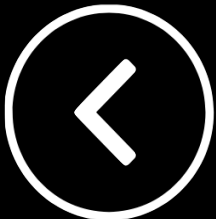

GC GLASS HYBRID RESTORATIVE SYSTEM

Click a product image to view product information



EQUIA Forte HT Fil



EQUIA Forte Coat

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE



EQUIA Forte HT Fil




Clinical Indications

- Class I restorations
- Stress bearing Class II restorations
- Non-stress bearing Class II restorations
- Intermediate restorative
- Class V and root surface restorations
- Core build up
- Amalgam replacement
- Molar Hypomineralisation
- High caries risk patients

EQUIA Forte® HT Fil is a long-term bulk fill glass-hybrid restorative system with enhanced mechanical properties, superior fluoride release, sound marginal seal and excellent handling

Glass hybrid systems offer the strength of the latest innovative glasses combined with the wear resistance of a nano-filled coating (Equia Forte Coat). The unique synergy between the coating and the restorative material improves the durability and wear resistance of the restorations.

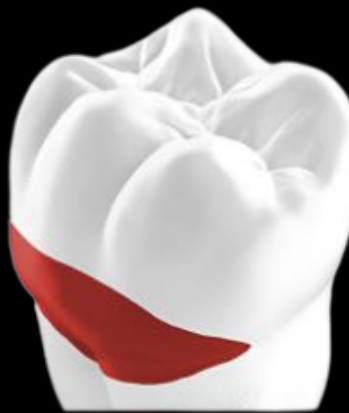

Dr. Zeynep Bilge Kütük, Turkey

Clinical Features & Benefits

- Packable, non-sticky and moisture-tolerant - fast and easy bulk placement
- Enables posterior restoration placement with more natural aesthetics
- Excellent biocompatibility & biomimetic properties
- Hydrophilic - withstand the humidity of the oral environment and the pulpal fluid flow
- Chemically bonds to dentin, enamel and cementum - strong, stable and chemically-fused seal
- Pulp-friendliness - suitable for use in deep cavities
- Ion exchange - uptake from saliva & aqueous solutions applied during brushing and tooth paste
- Co-efficient of thermal expansion close to tooth structure and the low shrinkage behaviour
- Good marginal adaptation of glass hybrids - virtually no post-operative tooth sensitivity
- Use in conjunction with Equia Forte Coat to increases surface hardness and wear resistance



[View product brochure](#)



[View comprehensive guide](#)



[View instructions for use](#)



[View Technique Guide](#)



[View Schwendicke Paper](#)




[View Schwendicke Article](#)





EQUIA Forte Coat®

EQUIA Forte® Coat incorporates a new multi-functional monomer that produces a tougher resin matrix for better protection. Used together with EQUIA Forte® HT Fil, the result is a strong bulk fill glass hybrid restorative system with exceptional physical properties and aesthetics.

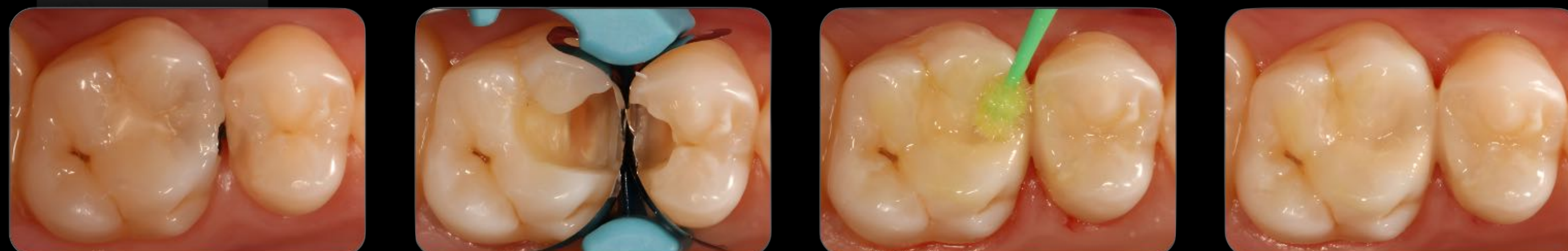

High caries risk patient - restored with Equia Forte HT and Equia Forte Coat®

Courtesy of Dr. Zeynep Bilge Kütük, Turkey

Features and Benefits


Nano-filled, self-adhesive resin coat

Single dispersion nano-filler and reinforced resin technologies

Improves strength and hardness of glass hybrid restoratives

Provides smooth glossy surface


Helps to maintain high wear resistance over time

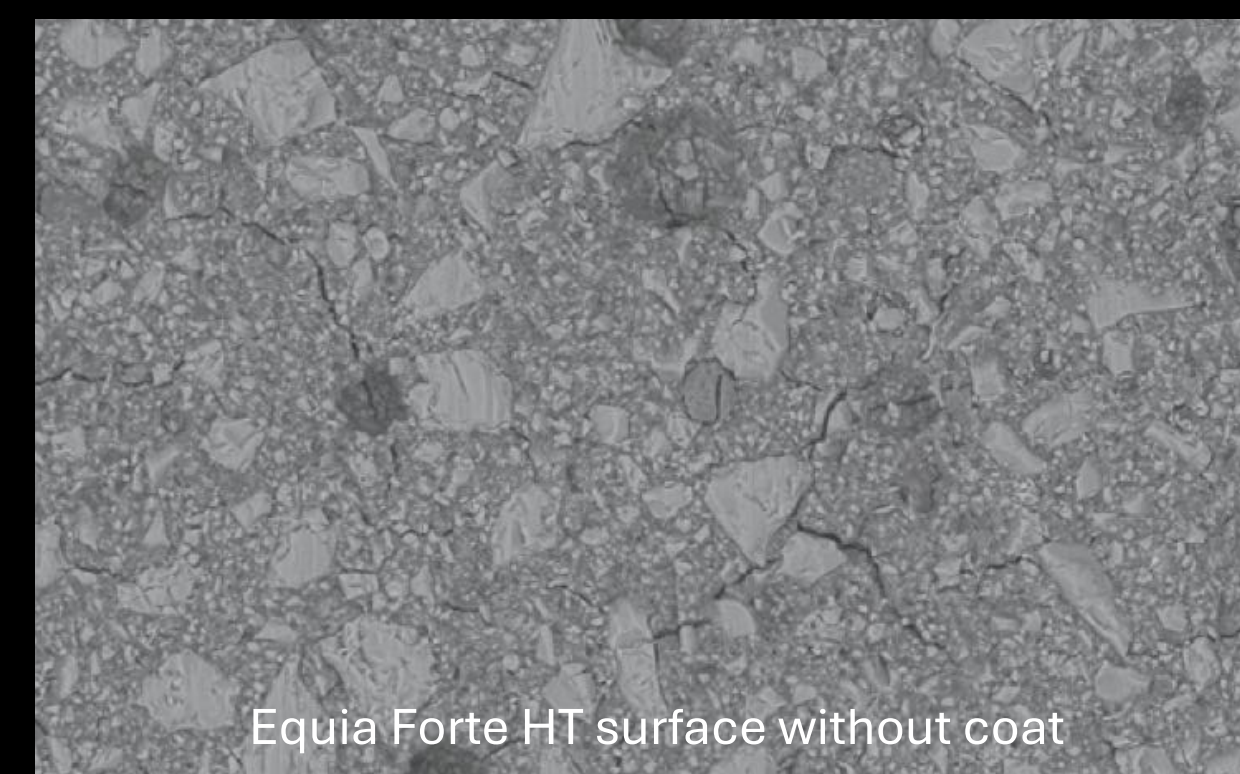

Equia Forte HT surface without coat



With Equia Forte Coat

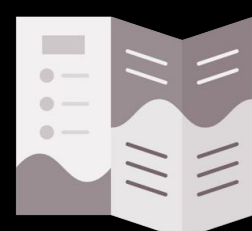


[View product brochure](#)



[View instructions for use](#)



[View Technique Guide](#)

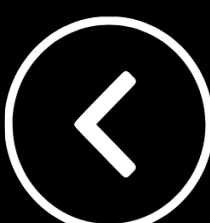

GC CONVENTIONAL & RESIN MODIFIED GLASS IONOMERS

Click a product image to view product information



DENTIN CONDITIONER



CAVITY CONDITIONER



EQUIA Forte Coat



GC Fuji II LC



GC Fuji VII



GC Fuji VII EP



GC Fuji BULK



GC Fuji LINING LC



View cement
usage guide

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE



DENTIN CONDITIONER

Dentin Conditioner is a mild polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentine and enamel before the application of your glass ionomer restorative.

It will increase the bond between your glass ionomer cement and tooth structure for added longevity.

The 10% polyacrylic acid solution cleans tooth surfaces to improve adhesion of glass ionomer cements

Applied on the tooth for 20 seconds



Clinical Indications

Improves the marginal seal

Longevity of the restoration


Leaves smear plugs in the tubules

Less risk of post-operative sensitivity

Blue tinting



Dentine smear layer

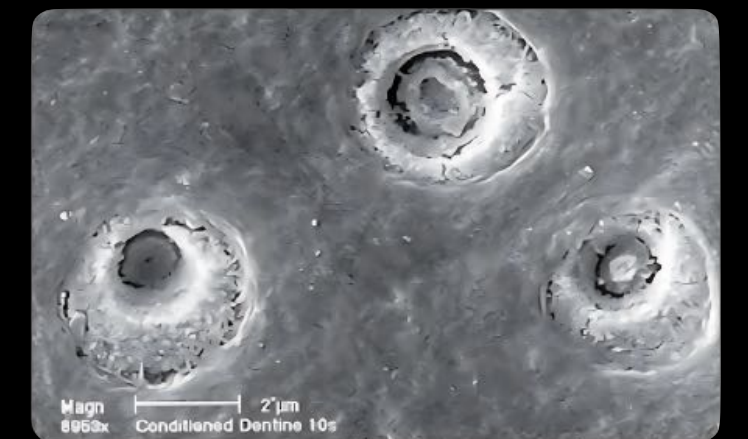


Conditioned dentine

Step-by-step

1. Using a cotton pellet or sponge, apply dentin conditioner to the bonding surfaces for 20 seconds
2. Rinse thoroughly with water
3. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE
4. Best results are obtained when prepared surfaces appear moist (glistening)
5. Apply glass ionomer as a bonding agent, base/liner or restorative material
6. Close the bottle immediately after use



[View instructions for use](#)

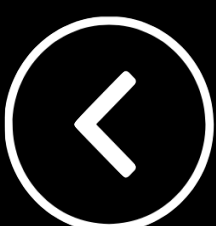

CAVITY CONDITIONER

Cavity Conditioner is a 20% polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentine and enamel before the application of your glass ionomer restorative.

It will increase the bond between your glass ionomer cement and tooth structure for added longevity.

Applied on the tooth for 10 seconds

Clinical Indications

Cleans the surface effectively | Achieve optimal chemical adhesion

Creates mechanical retention

Less risk of post-operative sensitivity

Enhances the marginal seal

Blue tint




[View instructions for use](#)



Step-by-step


1. After tooth preparation, apply CAVITY CONDITIONER to the bonding surfaces for 10 seconds using a cotton pellet or sponge
2. Rinse thoroughly with water and dry. DO NOT DESICCATE
3. Apply GC Fuji BOND LC, GC Fuji II LC (improved) or other glass ionomer/glass hybrid as a bonding agent, base/liner or restorative material
4. Close the bottle immediately after use

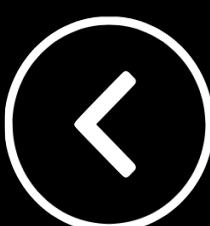



EQUIA Forte Coat®

EQUIA Forte® Coat incorporates a new multi-functional monomer that produces a tougher resin matrix for better protection. Used together with EQUIA Forte® HT Fil, the result is a strong bulk fill glass hybrid restorative system with exceptional physical properties and aesthetics.

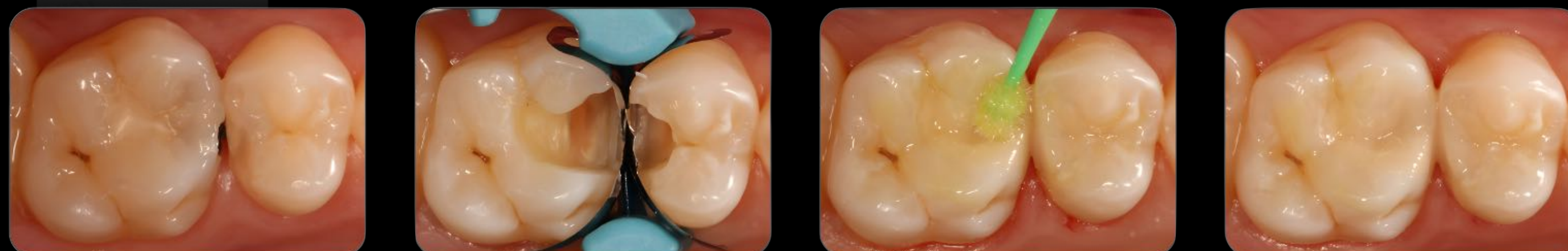

High caries risk patient - restored with Equia Forte HT and Equia Forte Coat®

Courtesy of Dr. Zeynep Bilge Kütük, Turkey

Features and Benefits


Nano-filled, self-adhesive resin coat

Single dispersion nano-filler and reinforced resin technologies

Improves strength and hardness of glass hybrid restoratives

Provides smooth glossy surface


Helps to maintain high wear resistance over time

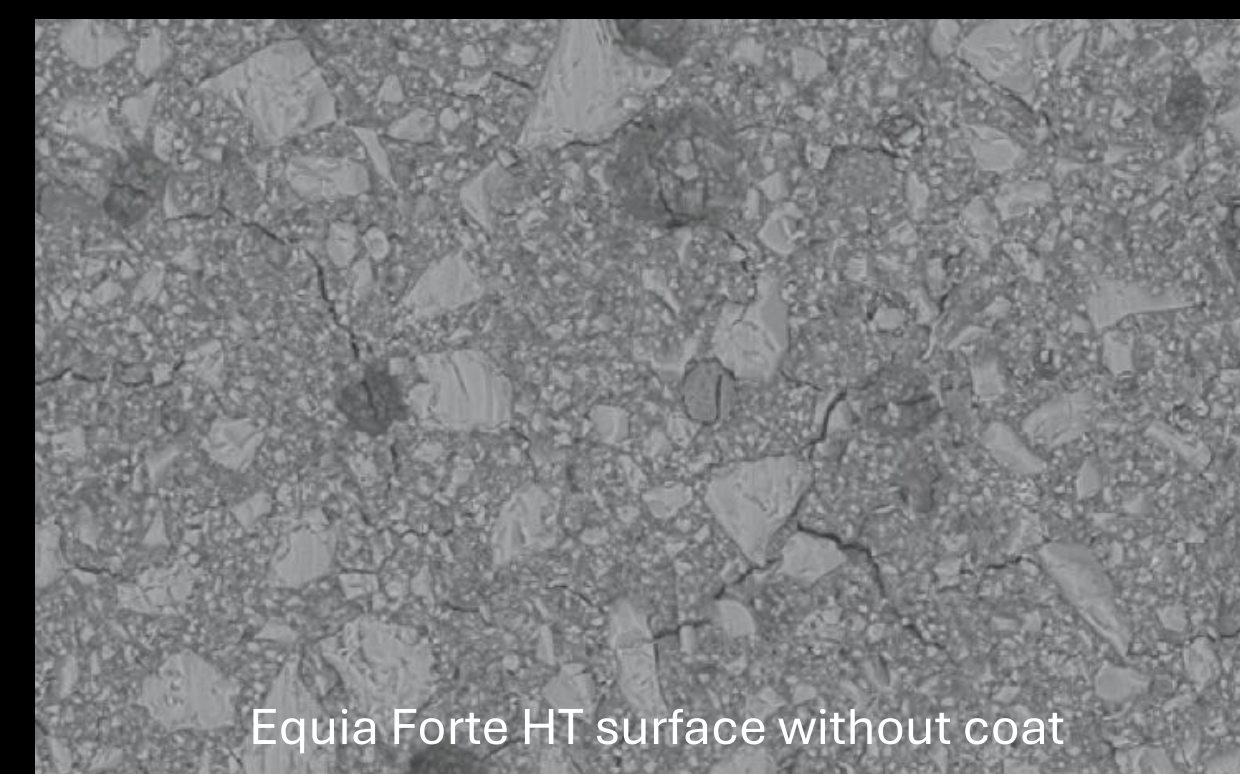

Equia Forte HT surface without coat




With Equia Forte Coat

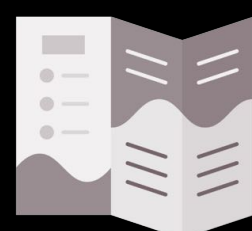


[View product brochure](#)



[View instructions for use](#)

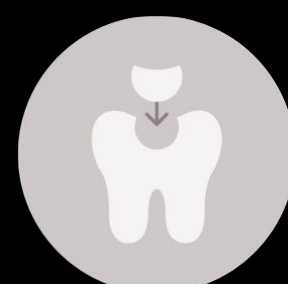

[View Technique Guide](#)

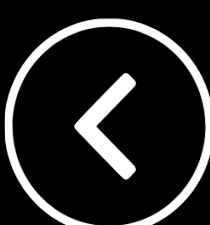

GC Fuji II LC



GC Fuji® II LC Capsule is the world's first resin-reinforced glass ionomer and has remained the benchmark for light cured glass ionomer cements. GC Fuji® II LC Capsule is a self-adhesive restorative material that is moisture tolerant, simple to use, quick to apply and is ready for final finishing immediately following 20 seconds light curing

Clinical Indications


- Class III and V restorations
- Root surface caries and cervical lesions
- Cervical lesions - Carious, Erosion, Abrasion & Abfraction
- Proximal lesions - primary and secondary teeth
- Reinforced base in sandwich technique
- Lesions where cavity margins are in dentine or
- When enamel cannot be adequately etched
- Base/liner under composite in the sandwich technique
- Core build up material
- Fissure sealant
- Adhesive for bonding orthodontic brackets

Clinical Features & Benefits

- Excellent aesthetics and polish-ability
- Abrasion resistance, ideal for cervical areas
- Good radiopacity
- Triple cure mechanism gives guaranteed cure for immediate finishing after light curing
- Available in 11 translucent Vita® shades
- Fluoride releasing and rechargeable uptake from external sources



(Dr. G. Knight)

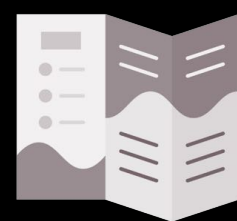

[View product brochure](#)



[View instructions for use](#)



[View capsule activation guide](#)




GC Fuji VII



Clinical Indications

- Surface and fissure protection
- Root surface protection
- Protection of erupting molars
- Hypersensitivity prevention and control
- Intermediate endodontic sealing
- Intermediate restorations
- Caries stabilisation and indirect pulp capping
- Cementation of stainless steel crowns
- Fissure protection

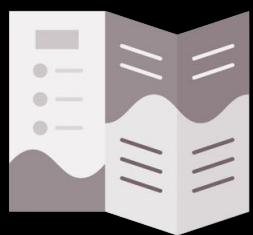

[View product brochure](#)



[View instructions for use](#)




[View capsule activation guide](#)


GC Fuji VII is the revolutionary glass ionomer sealant and surface protectant. It is fantastic at keeping young patients' teeth cavity-free. There is no isolation or bonding agent required. It works in a moist field. With GC Fuji VII, you will never have to worry about sealing over immature enamel or non-cavitated lesions. The self-bonding, high fluoride releasing GC Fuji VII creates a strong, acid-resistant fused layer.

Clinical Features & Benefits

- Flowable consistency to ensure effective wetting & intimate adhesion to tooth surfaces
- Moisture tolerant- can be applied in situations where saliva control is difficult
- Available in white and special pink shades for clear identification

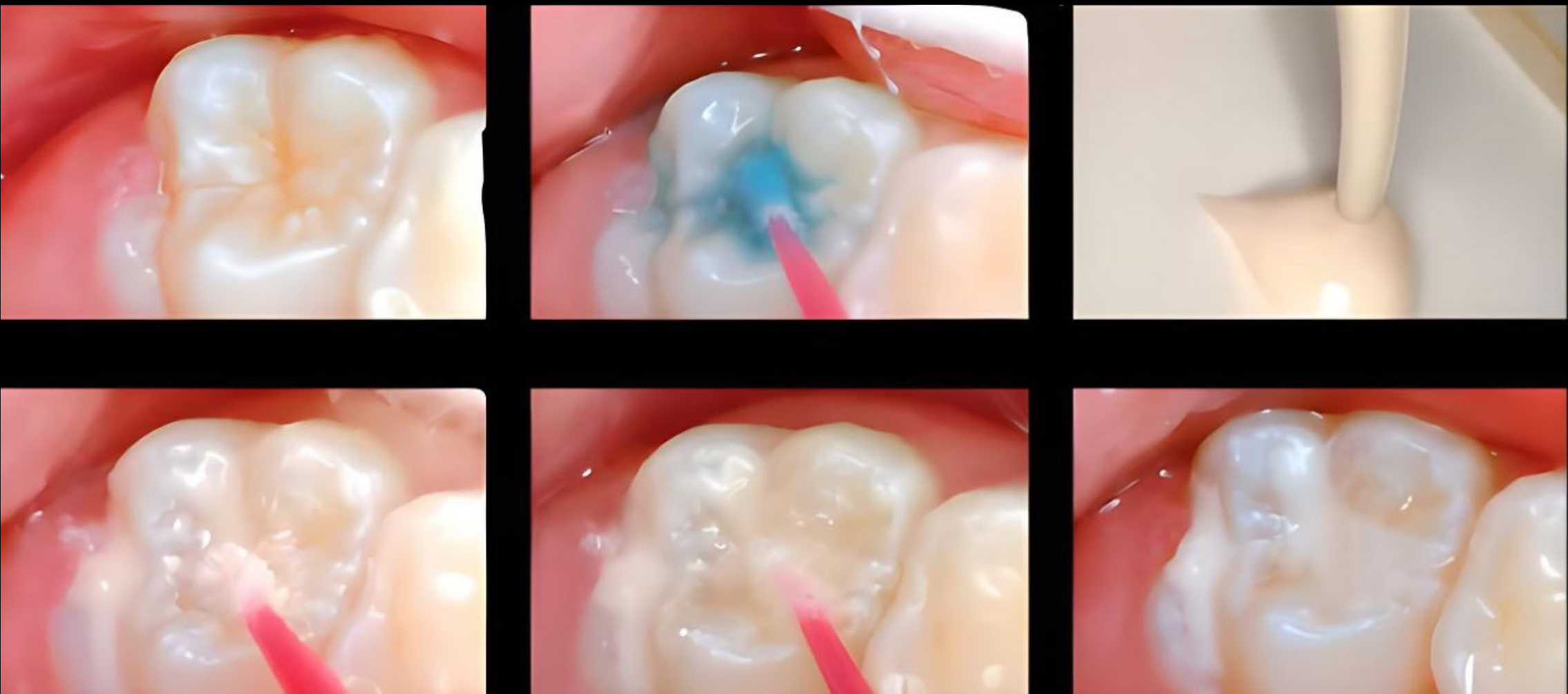

(Dr J. Lucas)

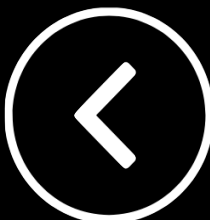


GC Fuji VII EP



Clinical Indications

- Higher risk patients
- Root surface protection
- Fissure protection
- Hypersensitivity prevention and control
- Protection for immature enamel (erupting molars)
- Temporary filling of endodontic access
- Intermediate restorations
- Protection of teeth with Molar Hypomineralisation

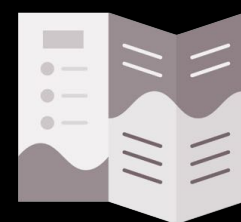

[View product brochure](#)



[View instructions for use](#)





[View capsule activation guide](#)

GC Fuji VII EP is a clinically proven glass ionomer cement formulation enhanced with RECALDENT® (CPP-ACP) for superior protection and remineralisation.

Once applied, GC Fuji VII EP forms a mechanical barrier to protect the covered and surrounding tooth surfaces and to minimise plaque accumulation. GC Fuji VII EP is essential when treating higher risk patients, including those with high acid exposure, reduced saliva flow, compromised cleaning and plaque removal capability.

Clinical Features & Benefits

- 3% RECALDENT® (CPP-ACP) enhances protection to adjacent dentine and enamel
- Remineralisation of tooth structure - prevents caries lesions from progressing
- Enhanced biofilm resistant surface
- Enhanced mineral release - calcium, phosphate, strontium and fluoride
- Superior access and delivery through GC Elongation Tips
- Available in pink and white shades

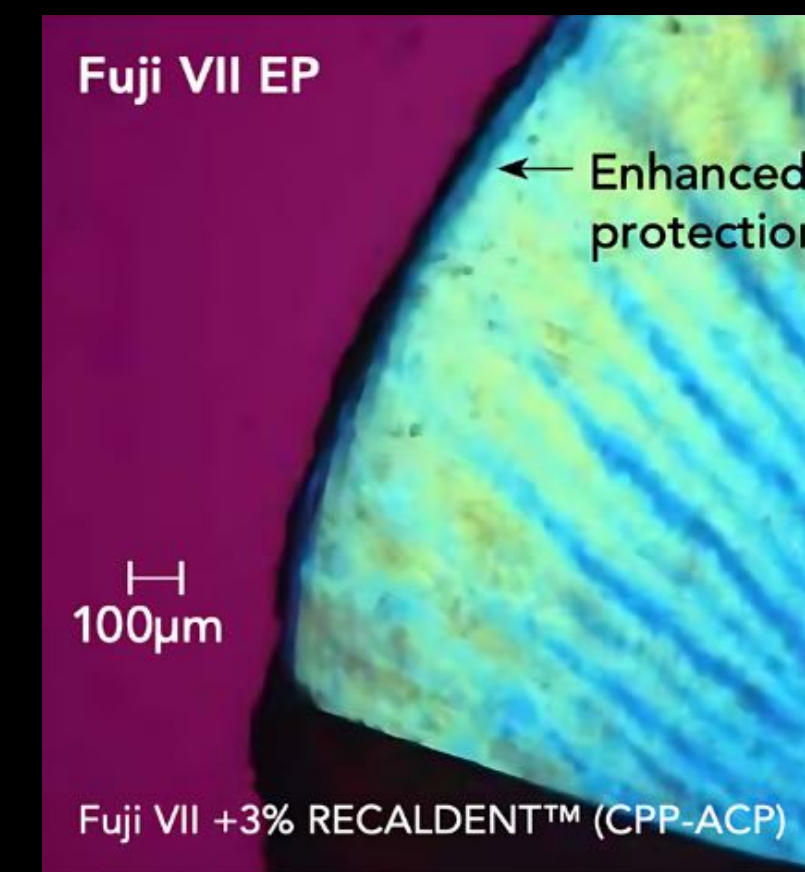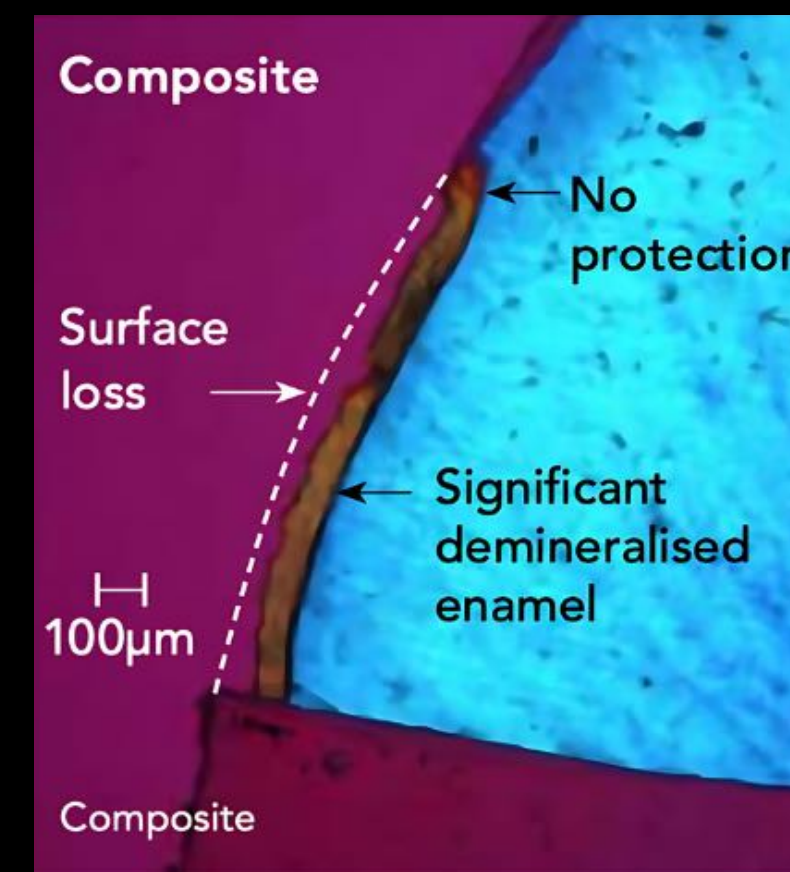

Polarised light micrographs of lactic acid (pH 4.8) demineralised lesions in enamel adjacent to Fuji VII EP and resin composite



GC Fuji BULK



Clinical Indications

- Class I and II restorations in deciduous teeth
- Non-load bearing Class I and Class II restorations in permanent teeth
- Base material for load bearing situations in Class I and Class II cavities using a sandwich laminate technique
- Transitional and intermediate restorations
- Class V and root surface restorations
- Core build-up


Perfect choice for

- Paediatrics & Geriatrics
- Dry mouth patients and patients with erosion
- Transitional restorations
- Caries-active patients and exposed root surfaces


Fuji® BULK is a robust, rapid-setting, multipurpose auto-cure glass ionomer cement (GIC) designed to meet the challenge of bulk placed restorations in more acidic oral environments. Truly remarkable in its formulation, Fuji BULK provides a unique balance between restoring function and protecting surrounding tooth surfaces from acid challenges.

Clinical Features & Benefits


- Designed for bulk placed restorations in more acidic oral environments
- Balance between restoring function and providing maximum protection to surrounding tooth surfaces from acidic challenges
- Fast setting auto-cure
- Seals and restores and is ready for final finishing just 2 minutes after start of mix

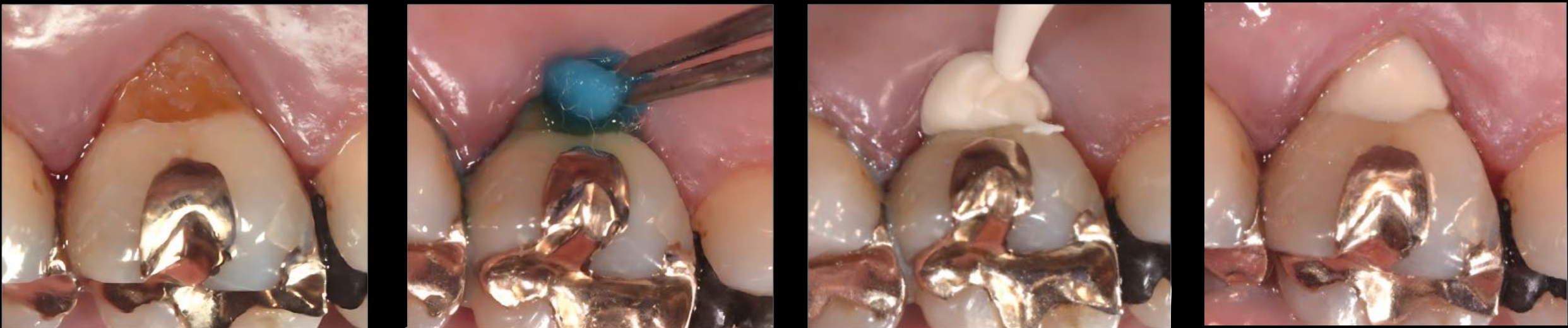

Dr E. Stephen Vouliotis

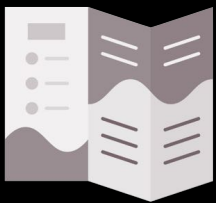

[View product brochure](#)

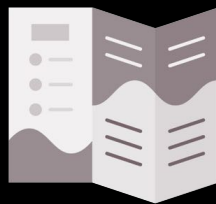


[View GIC usage guide](#)



[View instructions for use](#)



[View capsule activation guide](#)

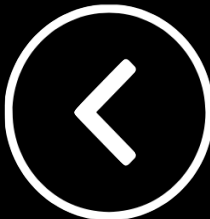

GC Fuji LINING LC




Clinical Indications

Light-cured glass ionomer cement
Liner or a base under composite or amalgam restorations

Clinical Features & Benefits

Easy adaptation to cavity walls without slumping
Excellent bonding to tooth structure offering a durable marginal seal
Controlled setting with light curing
Low water solubility
Insensitive to humidity
Radiopaque for easy post-operative diagnosis
Simple, accurate dispensing
Very easy bubble free mixing

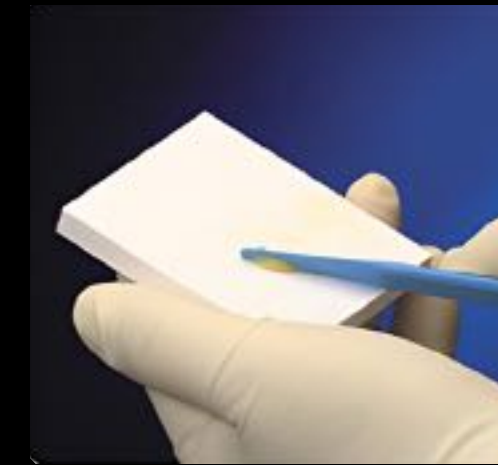

Easier Mixing

The paste-paste formulation always mixes to the ideal consistency because the exact proportions are dispensed each time. The result is a perfect mix in just 10 seconds, without any air bubble incorporation.



No dripping from the placement instrument

Once mixed, the lining cement is easily transferred from mixing pad to instrument where it holds without dripping, ready for placement exactly where you want it.



No slumping in the mouth

GC Fuji LINING LC has excellent flow, without running or slumping. You can easily control the even spread of cement over cavity walls and floor to form a strong, protective lining in even the shallowest of cavities.



Easy to identify after placement


The Universal shade balances the need for aesthetics, especially when lining under a shallow composite restoration, with the need for a visual contrast to help guide placement.



[View product brochure](#)



[View instructions for use](#)

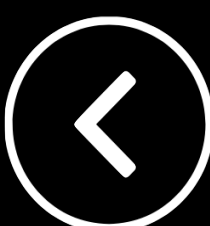

GC COMPOSITES & BONDS

Click a product image to view product information



G2-BOND Universal



G-Premio BOND



GC Modeling Liquid



Essentia Masking Liner



G-aenial Universal Injectable



G-aenial Anterior / Posterior



G-aenial Universal Flo



G-aenial Flo X



everX Flow



EverStick C&B, Perio, Ortho

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE



G2-BOND Universal



The Dual H-Technology and two bottle system of G2-BOND Universal enables the advanced optimisation of bonding both to tooth and composite by the smooth transition from hydrophilic to hydrophobic characteristics.

The remarkably hydrophobic bond layer thanks to the **HEMA-free composition** lowers the occurrence of water sorption which decreases the risk of degradation and leads to superior durability.

G2-BOND Universal offers marginal quality with its strong bond layer that avoids gap formation and microleakage which leads to sealing effectiveness with virtually no post-op sensitivity.

G2-BOND composition of three functional monomers (4-MET, MDP, MDTP) creates a durable adhesion to different substrates.


Clinical Indications

- Direct restorations
- Intra-oral repair
- Immediate Dentin Sealing (IDS)
- Luting of indirect restorations (with G-CEM LinkForce™)
- Post luting & core build-up
- Adhesive for bonding orthodontic brackets

Features & Benefits


- 4-MET for a reliable adhesion to enamel & dentin
- MDP for an excellent adhesion to enamel & dentin, zirconia, alumina and non-precious metals
- MDTP to bond to precious metals
- HEMA free bonding system for a clinically proven outstanding durability

Immediate Dentine Seal

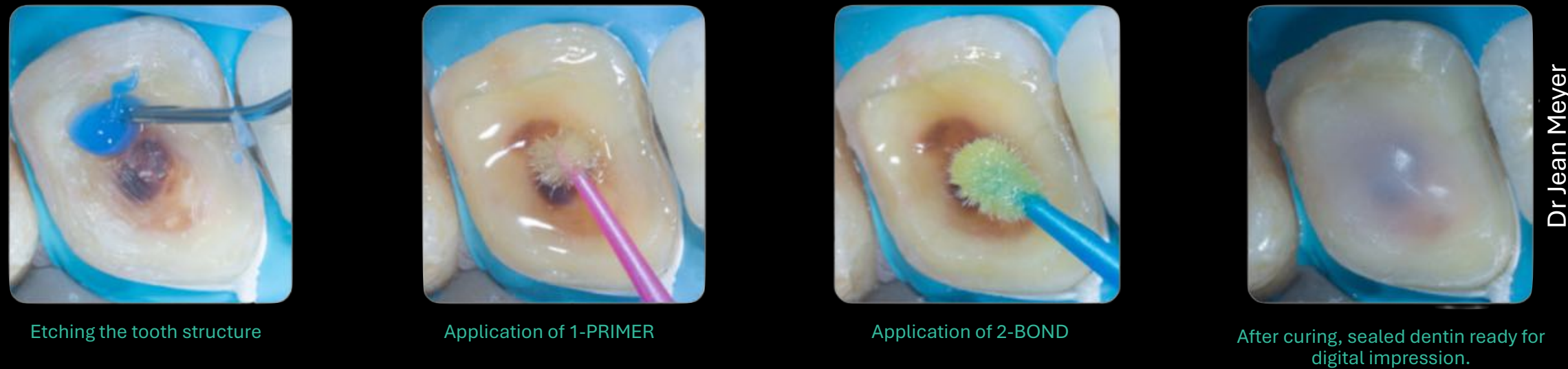

Dr Jean Meyer

Direct Restoration

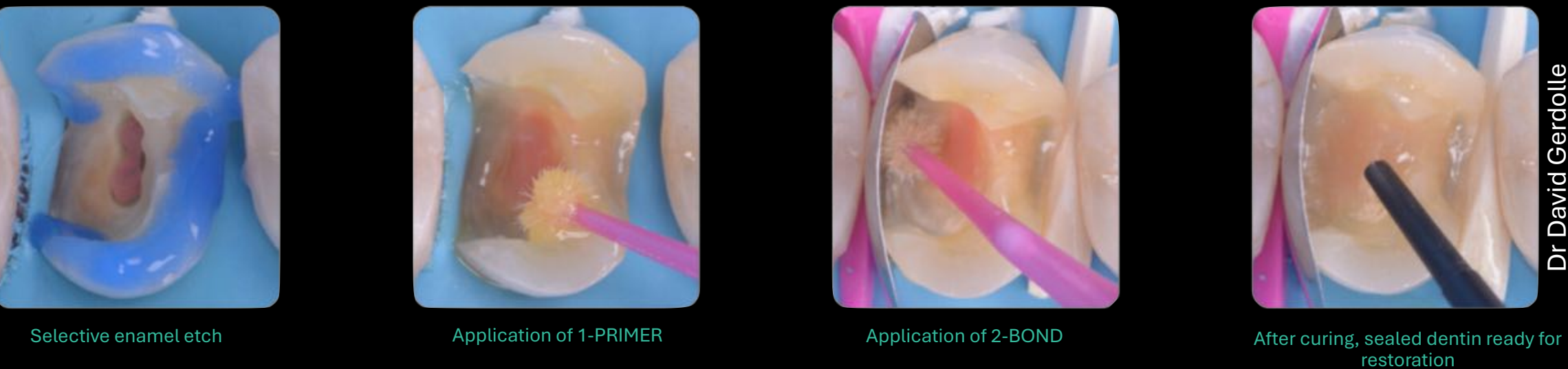

Dr David Gerdolle



View product brochure



View instructions for use




View Technique Guide



View IDS Technique Guide



View clinical article

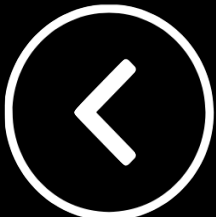

G-Premio BOND

G-Premio BOND™ is a one-bottle universal bonding agent compatible with all etching modes. It can be used for direct bonding and repair.

G-Premio BOND™ offers the advantages of a **universal adhesive**, with great performance in all situations, thanks to a unique combination of three functional monomers (4-MET, MDP, MDTP). **G-Premio bond is HEMA-free**

With an easy application of G-Premio BOND™ achieves immediate chemical adhesion and high bond strengths that are ideal clinically.




Features & Benefits


- Fast and simple, no rubbing or complex application procedures
- Immediate chemical bonding, high bond strengths without needing to rub the surface.
- Chemically bonds within the first few seconds of application
- The drying step is highly reproducible: 5 seconds, MAXIMUM air pressure
- Fast 25 sec self-etch application and chemical adhesion
- Perfect for paediatric patients or where moisture control is challenging
- Yellow colour prior to light-curing gives visual control of application
- Visible during application, invisible after light curing
- Wets the tooth just like water
- Perfect margins, 3um film thickness - avoids bond pooling
- Eliminates patient discomfort

Clinical Indications

- Direct bonding of light-cured composites and compomers to tooth structure
- Bonding of dual-cured core build up composites to tooth structure, as long as these materials are light-cured
- Intraoral repair of composite, metal-based and zirconia/alumina-based restorations
- Intraoral repair of ceramic restorations in combination with a silane coupling agent (like G-Multi PRIMER)
- Sealing of tooth preparation (cavity or abutment) for indirect restorations
- Cavities where onlays and inlays would also be indicated

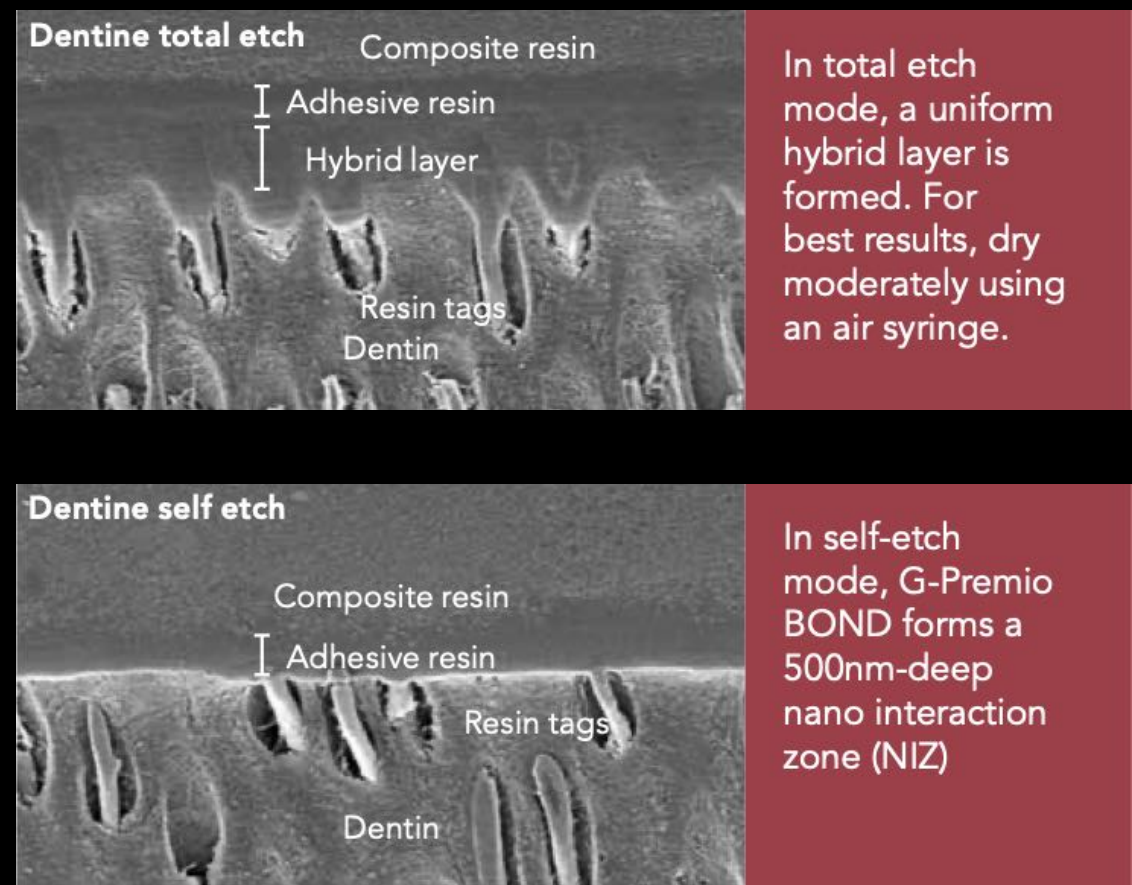

- View product brochure
- View instructions for use
- View Technique Guide
- View Repair Technique Guide
- View clinical article

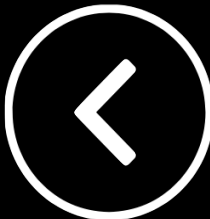

GC Modeling Liquid

Modeling Liquid is GC's solution for an easy, fast and efficient application of all direct composite materials. Applied on a brush and/or instrument, it eases the placement and shaping of composites and helps to achieve good morphology with a smooth finish.



Features & Benefits

- Can be used with an instrument or brush for quick and easy application and shaping of composites
- A smooth finish for a shorter finishing procedure
- Adapting the final layer of composite using a wetted brush aides in achieving a smoother surface
- Invisible for great aesthetic results
- GC Modeling Liquid becomes fully transparent after light-curing

Clinical Indications

- Modeling of composite materials for direct restorations
- Use with an instrument or the brush technique
- Pre-treatment of fibres in direct and indirect procedures



Dispense one drop of GC Modeling Liquid



Protect it from the light with a cover



Touch the drop with the brush or instrument in order to moisten it



Remove the excess thoroughly using a gauze swab



Use your instrument or brush to apply the composite



Use the flat brush to adapt and shape the composite



Dispense one drop of GC Model the dentin mamelons using a round brush



Shape and smoothen the final layer of composite using a wetted flat brush or instrument.




Apply the composite using your wetted instrument



The brush can be used to adapt the material cusp by cusp




GC Modeling Liquid and the brush aide to recreate the fissures and refine anatomy

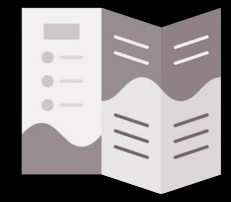


View product brochure



View instructions for use

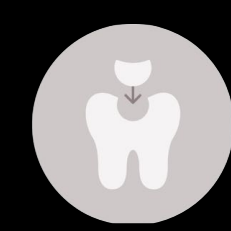

Watch How to use GC Modeling Liquid video

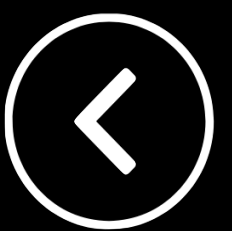

Essentia™ Masking Liner



Essentia Masking Liner is an opaque, heavily-filled injectable composite that will block discolouration without increasing value, creating the base for great aesthetics.

Clinical Indications

Suitable for anterior and posterior teeth

Covers amalgam staining

Blocks discolouration to improve aesthetic outcomes

Masking dark cores under veneers, crowns and direct restoratives

Wet and adapt well to the base of cavities and is well suited to placement in very thin layers




[View product brochure](#)



[View Essentia Clinical Guide](#)



Dr Chee Chang

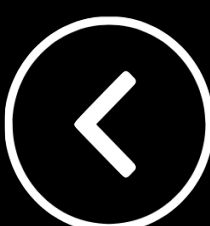


G-ænial Universal Injectable



Clinical Indications

- Direct restorative for Class I, II, III, IV and V cavities
- IMT (injection moulding technique)
- Fissure sealant
- Sealing hypersensitive areas
- Repair of direct and indirect aesthetic restorations
- Blocking out undercuts
- Liner or base
- Splinting of teeth in combination with fibres such as GC everStick fibres

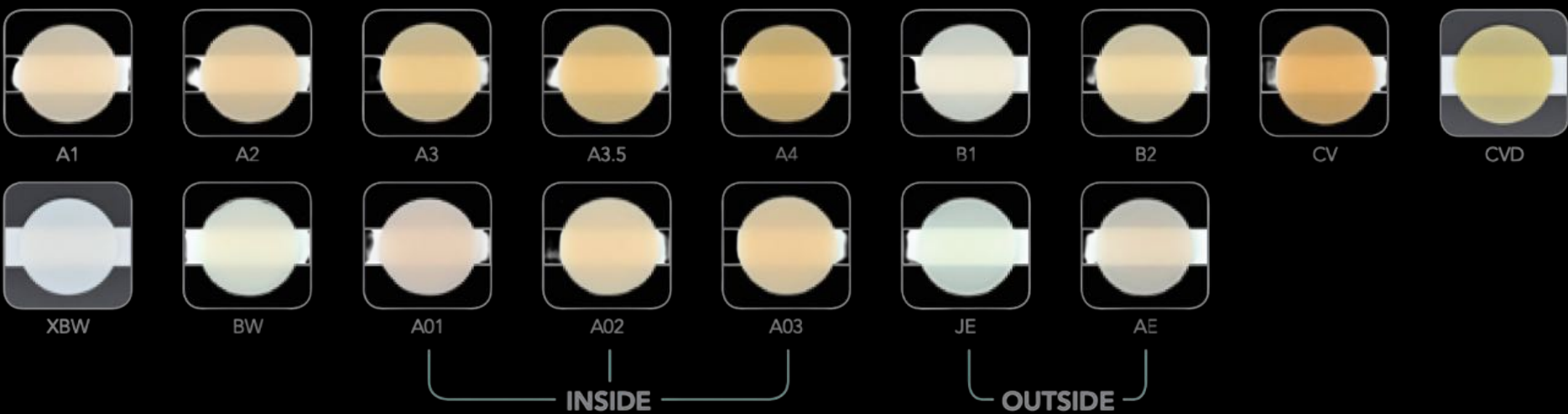


G-ænial™ Universal Injectable is a universal restorative composite displaying exceptional strength & resistance for long-lasting aesthetic restorations. It is incredibly strong - even stronger than leading paste composites on the market.

Features & Benefits

- Exceptional strength & wear resistance thanks to the combination of ultra-fine barium fillers
- GC's Full-coverage Silane Coating (FSC) technology
- Suitable for any cavity class without size limitation
- Strong enough without the need to cover with a conventional composite
- Unique thixotropic viscosity, optimal for free-hand build-up of cusps
- Shorter finishing steps as the anatomy is already created
- New syringe design featuring easy extrusion & no uncontrolled flow-out
- Optimal viscosity to easily separate the material from the tip and does not slump
- New tips with a long bendable needle for unparalleled access to difficult posterior cavities
- 16 shades in three levels of translucency, Excellent polish-ability and gloss retention
- High radiopacity (252%) - follow up your restorations and to easily detect any secondary caries



Injection Moulding Technique (Dr Anthony Tay)

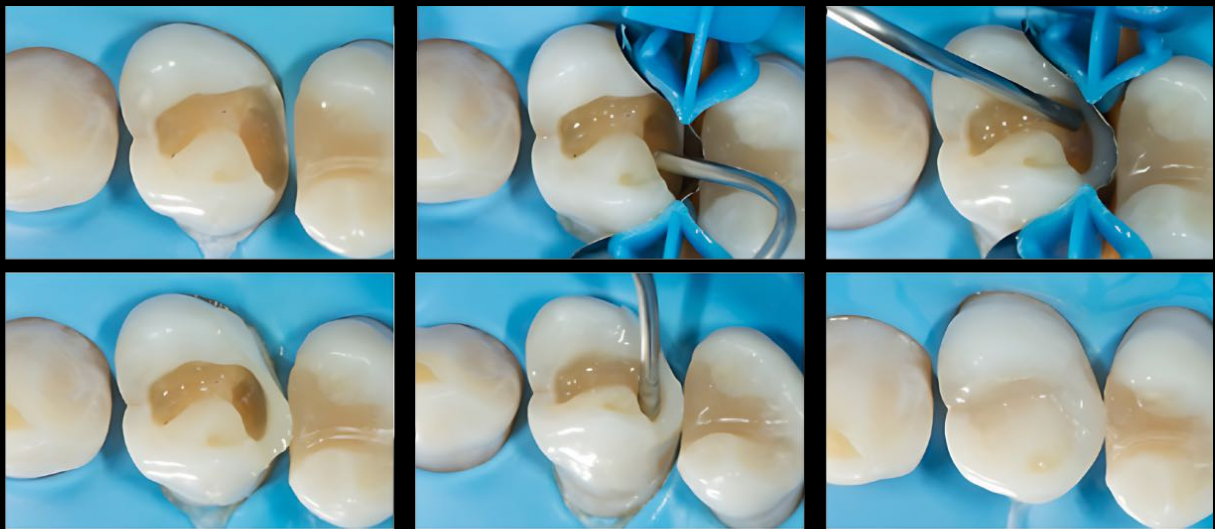

Posterior Restorations (Dr Javier Tapia Gaudix)



View Cusp
Build-up Video



View Anterior Injection
Moulding Video



View Posterior Injection
Moulding Video




View product brochure



View instructions for use



View Injection Moulding Guide

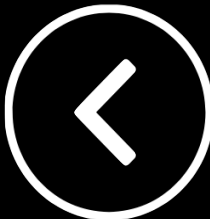

G-ænial Anterior & Posterior



Clinical Indications

G-ænial ANTERIOR

Direct restorative for Class II, IV, V cavities
For wedge-shaped defects and root surface cavities
For veneers and diastema closure

G-ænial POSTERIOR

Direct restorative for Class I and II cavities

Features & Benefits

- Natural reflectivity creates invisibility - excellent light scattering properties
- Forward-thinking shading concept
- High strength, low shrinkage, easy handling
- Anterior enables sculpting , Posterior firmer and more packable
- Single shade simplicity
- Good polishability and high gloss retention







[View product brochure](#)





[View instructions for use](#)

G-ænial Anterior and Posterior is the material of choice when you are looking for a restorative that will meet the aesthetic expectations of all your patients - in most cases with only one shade. Being a forgiving composite that offers beautiful and natural looking high gloss restorations whilst being extremely easy to use, makes it a composite suited to everyday use. G-ænial also gives you more working time so you are free to shape and sculpt it to obtain the ideal anatomical form and aesthetics with ease.


Anterior Shade Range

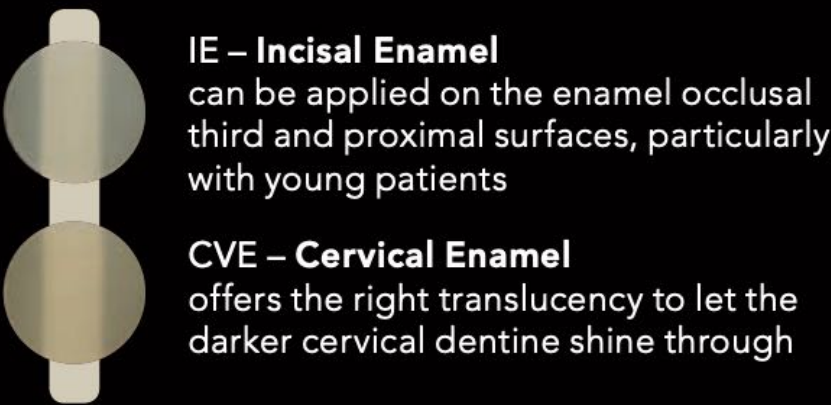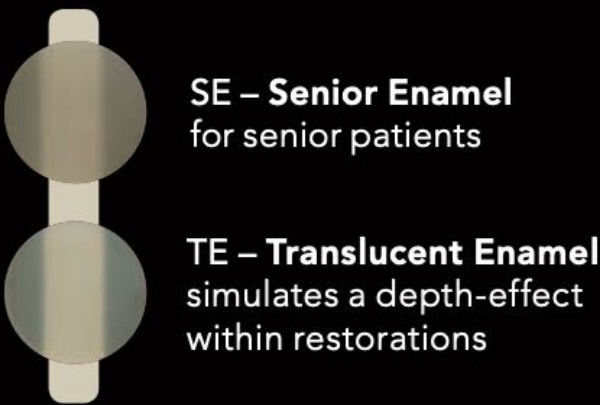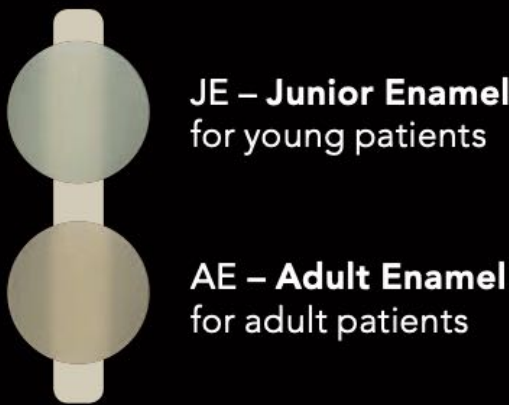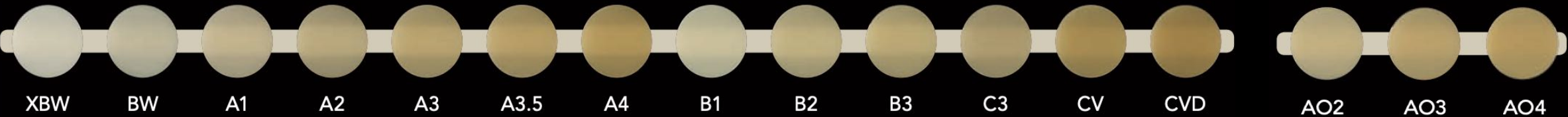

Posterior Shade Range

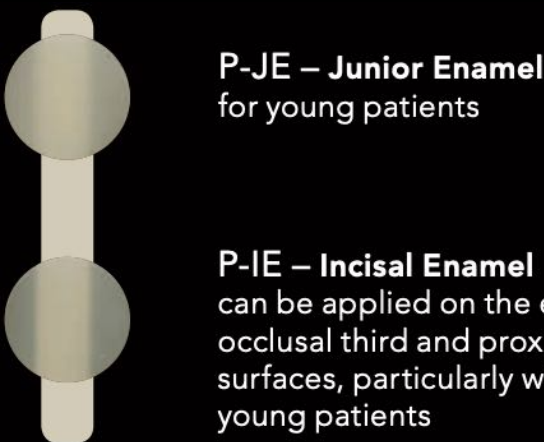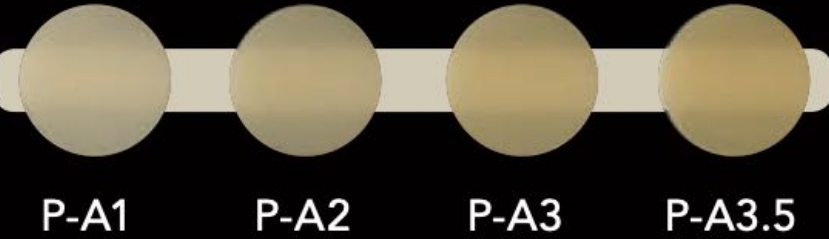

Dr P Sheridan

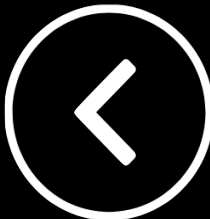

G-ænial Universal Flo



With GC G-ænial™ Universal Flo you get an exclusive blend of injectable viscosity for easy and convenient placement even in deep cavities, combined with outstanding physical properties for safe and durable restorations

G-ænial™ Universal Flo provides the answer to many difficult cases where you do not want to compromise between perfect placement, aesthetics and strength

Its unique combination of strength and flowability makes it also perfectly adapted to be used as a light-cured composite luting cement

Clinical Indications



View product brochure



View instructions for use



View Technique Guide

True innovation in filler size and composition, in filler silanation and in manufacturing techniques has given rise to a remarkable composite resin. G-ænial Universal Flo is the first truly universal Injectable Composite, a material that clinicians can place with ease for beautifully adapted, beautifully finished, fast, aesthetic and tough composite restorations

Features & Benefits

Injectable composite, feels like a flowable, behaves like a conventional restorative

Superior polish retention like a nano-fill

15 shades based on Vita guide - 3 levels of translucency

Single shade or multi-shade layering application options




High strength, strong, polishable and high wear resistance

Fine 200nm filler size and composition

Full coverage silane technology



Dr S Koide

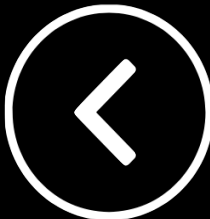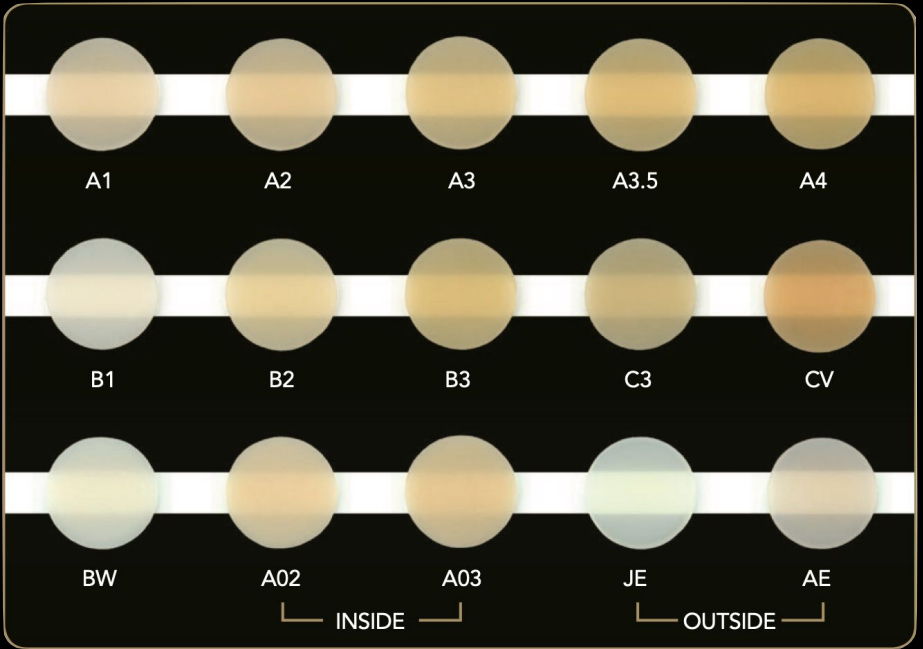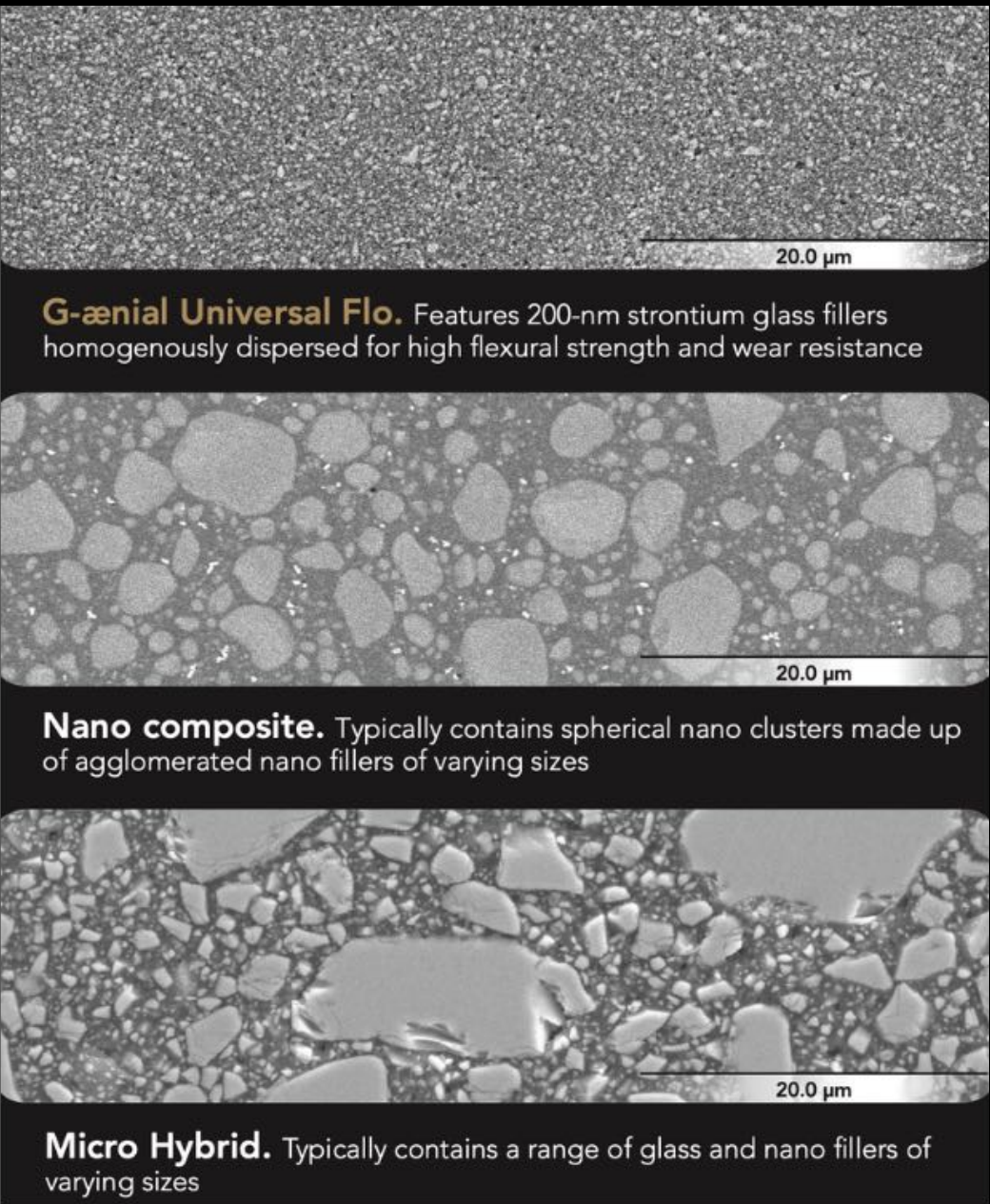

G-ænial Flo X



G-ænial™ Flo X is a high radiopacity and high strength flowable composite, designed specifically for optimum handling in lining, micro cavity and fissure sealing applications.

The best handling "conventional" flowable - yet with greater strength than most conventional composites.

Clinical Indications

Ideally suited for liner applications, sealant or where more flow is required

High radiopacity

Excellent wetting and adaptation following bond application

A full strength flowable composite for traditional flowable applications



Features & Benefits

A flowable with impressive strength



Innovation in filler technology gives you a high strength composite in a flowable consistency. Designed to be strong enough for application in any classification of cavity. G-ænial Flo X and G-ænial Universal Flo deliver similar high levels of strength while offering different viscosities tuned to the variation in composite handling preferences of individual clinicians.

High radiopacity

Perfect for defining the cavity outline in radiographic images for direct or indirect procedures. G-ænial Flo X shows one of the highest levels of radiopacity among flowable composites

Enhanced fluidity


Ideal handling for lining application G-ænial Flo X gives you total control over the flow. It's designed to wet and adapt yet maintain control for complete ease of application.

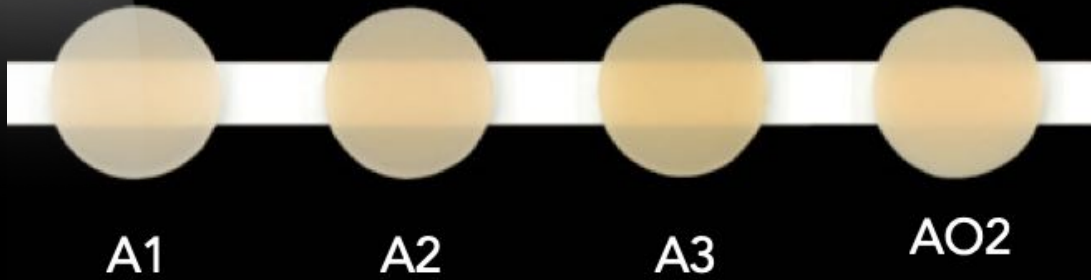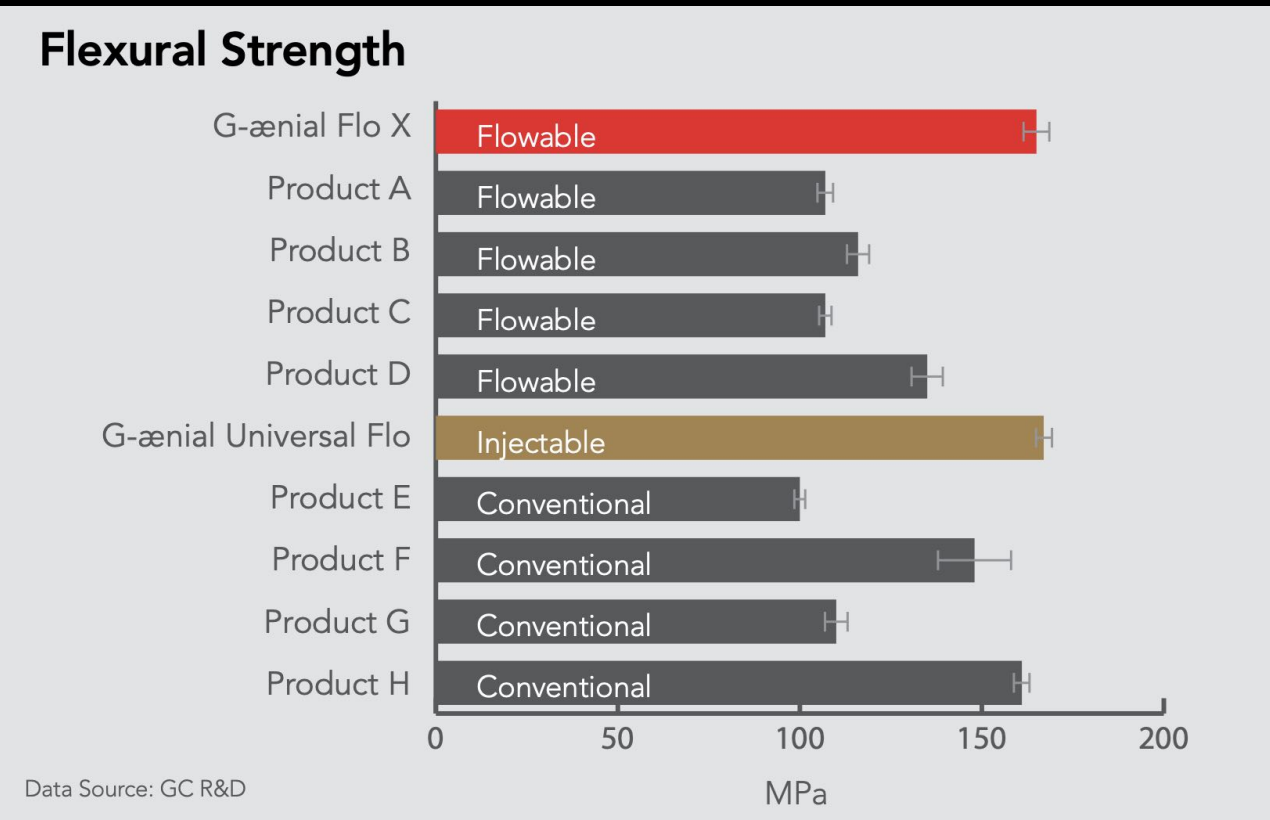

[View product brochure](#)



[View instructions for use](#)

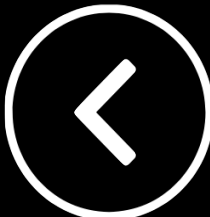

everX Flow




Clinical Indications

- Dentin replacement material for all direct composite restorations
- Large posterior cavities, deep cavities and endo-treated teeth, cavities with missing cusps or after amalgam removal
- Alternative to cavities where inlays and onlays would also be recommended
- Core build-up - crowns, inlays and onlays

Features & Benefits

- Optimal to strengthen large restorations thanks to the reinforcing effect of fibres
- Excellent fracture toughness reducing the risk of catastrophic failures
- Perfect thixotropy for easy placement and adaptation
- No slumping even in upper molars
- Excellent aesthetic results with the Dentin shade (in layers of 2mm)
- Indicated both for dentin replacement and for core build-up preparations

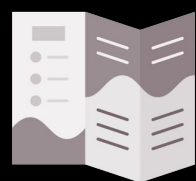

[View Product Brochure](#)



[View instructions for use](#)



[View Technique Guide](#)



[View Garoushi Paper](#)



[View Magne Paper](#)




[View Ozsevik Paper](#)

everX Flow™ is a short-fibre reinforced flowable composite indicated for dentine replacement in direct restorations (together with a conventional composite as enamel layer), and for core build-up.


Thanks to the short fibres it contains, ever Flow efficiently reinforces restorations and displays an exceptionally high fracture toughness. Fibres also help to redirect cracks and avoid catastrophic failures, which makes ever Flow an optimal material to use in weakened or cracked teeth, for instance after amalgam removal.

everX Flow has a very thixotropic viscosity which allows it to flow and adapt perfectly to the cavity floor, but without any slumping - even when used in upper molars. This optimal consistency makes your restorative treatment both faster and easier. everX Flow is available in two unique shades to answer all your clinical needs.


The Bulk shade displays a depth of cure of 5.5mm and is perfect for deep cavities, or whenever you want to speed up the treatment. The Dentine shade has a higher opacity and requires to be layered, and will be the best choice when you are looking for the best possible aesthetic results.

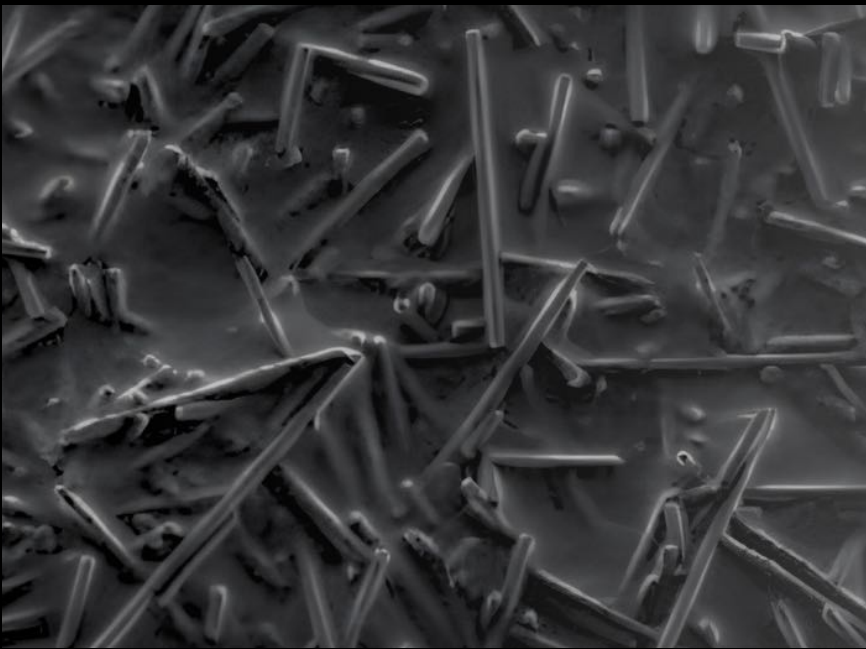

E-glass fibres in everX Flo

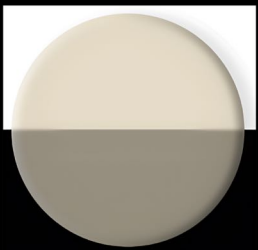

Bulk Shade for Deep Cavities and Cores - 5.5mm depth of cure

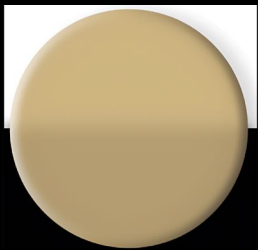

Dentine Shade for cores and aesthetics - 2.0mm depth of cure




Posterior restoration with everX fibre

Courtesy: Dr. Yo Han Choi



[Watch On-Demand Webinar](#)

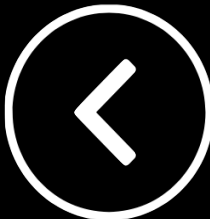

everStick



Clinical Indications

everStick®C&B

- Surface retained bridges
- Inlay and onlay bridges
- Hybrid bridges
- Full cover crown bridges
- Implant-supported bridges
- Temporary bridges
- Immediate bridges

everStick®NET

- Labial splinting of traumatised teeth
- Labial periodontal splinting
- Repair and reinforcing of veneers

everStick®ORTHO

- Aesthetic retention phase after active orthodontic treatment

everStick®PERIO

- Surface-retained or intra-coronal splinting
- Combined periodontal splints and surface-retained bridges

everStick® fibre reinforcements are made of silanated glass fibres in thermoplastic polymer and light curing resin matrix. everStick products address the advantages of minimally invasive dentistry where the patient's own healthy tooth tissue is saved for as long as clinically possible. This also means that other treatment options remain available should the patient ever need them in the future.

Features & Benefits

- Minimally invasive and reversible; leave other treatment options available
- Superior mechanical properties
- Unique patented bonding
- Elasticity close to that of dentin
- Extensive research data



[View product brochure](#)



[View Ortho brochure](#)



Instructions for use & Technique Guides



[everStick C&B](#)



[everStick Perio](#)




[everStick Post](#)




[everStick Ortho](#)



[everStick NET](#)

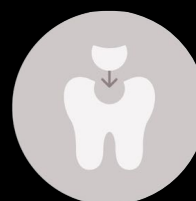


[everStick C&B
Technique Guide](#)

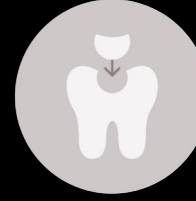

[everStick Perio
Technique Guide](#)



[everStick Post
Technique Guide](#)

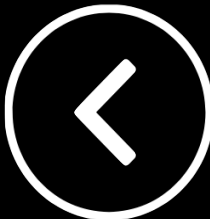

GC LUTING CEMENTS

Click a product image to view product information



G-CEM ONE



G-CEM Capsules



G-CEM Linkforce



G-Multi-PRIMER




G-CEM Veneer



GC Fuji PLUS CAPSULE



Click to download GC Luting Guide App

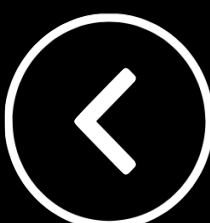

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE



G-CEM ONE™

Clinical Indications

- Cementation of all types of all ceramic inlays, onlays, crowns & bridges
- Cementation of resin and metal-based inlays, onlays, crowns & bridges
- Cementation of metal, ceramic, fibre posts, and cast post and cores
- Cementation of all ceramic and composite veneers
- Final cementation of crowns and bridges on implant abutments

Features & Benefits



- Excellent bond strength to enamel, dentin and to all indirect materials
- Optimal bond strength for retentive and non-retentive preparations
- Optional Adhesion Enhancing Primer
- Exceptional self-curing for your peace of mind when thick or opaque restorations are used
- Tack cure feature for easy excess removal
- Increased moisture and saliva tolerance when Adhesive Enhancing Primer is used
- Optimised flow of the paste, virtually no post-operative sensitivity
- Invisible, wear resistant margin for aesthetic outcomes
- 4 stable shades: A2, Translucent, AO3, BO1 (White Opaque)



G-CEM ONE™ is a truly universal, non-technique sensitive, versatile and reliable product that gives the flexibility of being effective in all cementation procedures for any type of restorations; from metal-based to resin and all-ceramic inlays, onlays, crowns, bridges, and posts. It demonstrates excellent bond strength to enamel, dentine and all indirect restorations.

In addition, the optional Adhesive Enhancing Primer (AEP) and its innovative GC Touch Cure feature provides optimal bond strength in challenging clinical situations, such as low retentive surfaces. The chemical initiator in the AEP accelerates the chemical cure of the cement to ensure optimal bonding. The AEP is easy to apply; no additional light-curing is needed - apply and air-dry.


G-CEM ONE can be combined with the dedicated Adhesive Enhancing Primer (AEP). This innovative Primer induces what we call "touch cure mechanism" for an accelerated and very effective dark-cure (self-cure) reaction of G-CEM ONE.

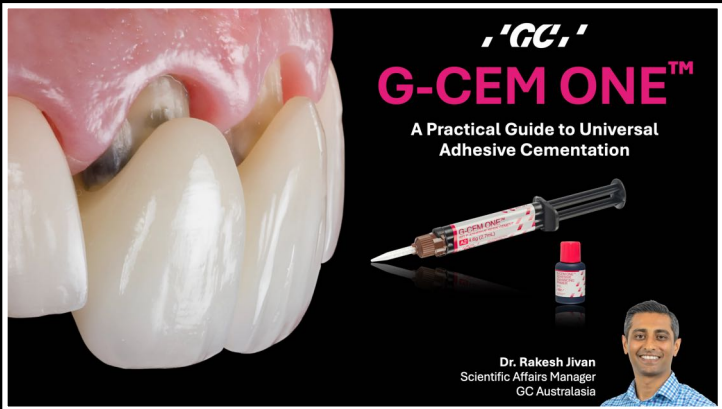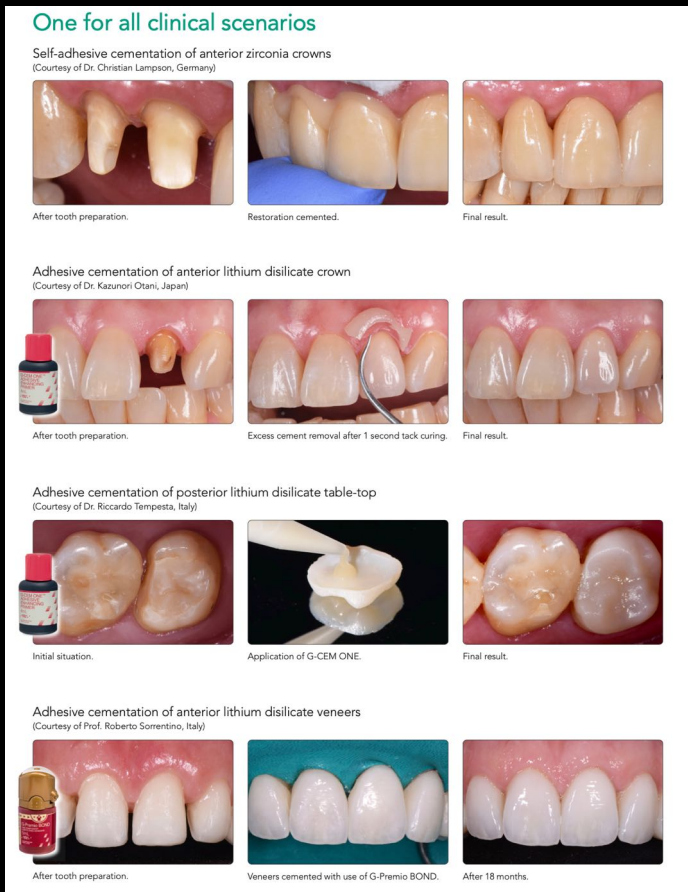

View On-Demand G-CEM One Clinical Presentation (1 CPD)



View On-Demand Zirconia Bonding Webinar (1 CPD)

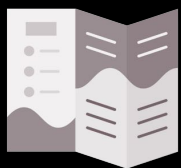

View product brochure



View Luting Guide brochure



View instructions for use



View Technique Guide



View Dr Miles Cone Video Part 1



View Dr Miles Cone Video Part 2



View Dr Miles Cone Video Part 3



View Dr Miles Cone Video Part 4




View Dental Advisor Video



View Simply The One Video



View Luting Non-Retentive Lithium Disilicate Cementation

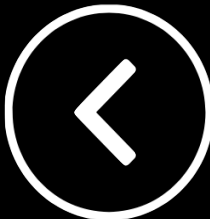

G-CEM Capsule




Clinical Indications


Cementation of all types of all ceramic, resin and metal-based inlays, onlays, crowns and bridges (max. 2 abutments)

Features & Benefits

- Excellent bonding property to restorative materials, including Zirconium
- High bond-strength to enamel in both self-cure and light-cure modes
- No need for pre-conditioning on the tooth structure
- Direct delivery into the root canal with special elongation tip
- Long-term stability and colour stability since there is no dissociation after curing
- Lowest linear expansion

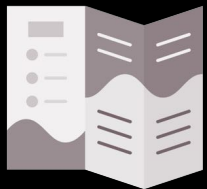

View product brochure

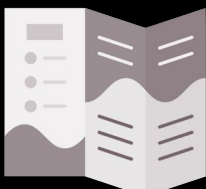

View Luting Guide brochure



View instructions for use



View capsule activation guide


G-CEM is a dual-cure self-adhesive universal resin cement delivered in capsules, designed for the adhesive luting of all-ceramic, zirconia, alumina, metal or composite indirect restorations.

G-CEM combines the improved handling and self-adhesion of conventional cements with the superior mechanical properties, adhesion and aesthetics of resin cements.



Light cure for 2-4 seconds until the cement has reached a gel phase. The excess cement is then easily removed using a scaler tip.

Cementation indications	Surface pre-treatment†		G-CEM
- Inlays, Onlays	Sandblast or etch	Ceramic Primer	Self-adhesive luting cement
Metal, PFM			
- Inlays, Onlays	✓		Y
- Crowns	✓		Y
- Bridges	✓		Y
- Endodontic Posts	✓		Y
Ceramics (less than 600 MPa flexural strength)			
- Inlays, Onlays	✓	✓	Y
- Crowns	✓	✓	Y
- Bridges	✓	✓	Y
High Strength Ceramics (greater than 600 MPa flexural strength)			
- Crowns	✓		Y
- Bridges	✓		Y
- Endodontic Posts	✓		Y

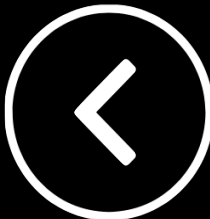

G-CEM LinkForce




Clinical Indications


Inlays, Onlays, Veneers, CAD/CAM, Crowns, Bridges

From inlays/onlays, overlays, veneers and tabletops to CAD/CAM prosthetics, the choice between different types of restorations has become wider. Add the introduction of new substrates such as zirconia, lithium disilicate and hybrid ceramics and it is no surprise that mastering all the cementation procedures for all indications is now a difficult challenge. Is there one solution that can adapt to all situations?


GC has developed a universal adhesive resin cement for you that offers predictable results and allows you to work in a standardised way. G-CEM LinkForce® from GC - the universal and powerful solution to all your adhesive cementation challenges that ensures a strong bond to all substrates in all indications, without compromises.

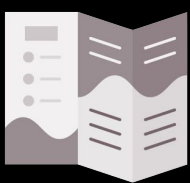

View product
brochure

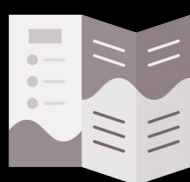

View Adhesive Resin
Cement brochure

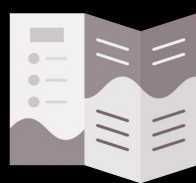

View Luting Guide
brochure



View G-Cem Linkforce
technique guide



View G-Cem Try-in Paste
instructions for use



View G-Cem LinkForce
instructions for use

Features & Benefits

Secured adhesion in all situations with only one system with three base elements

G-Premio BOND bonds to ALL preparations with no compromises: bonds to teeth, metal abutments and composite core build-ups

G-Multi Primer ensures a stable adhesion to ALL restorations: stable chemical adhesion on all surfaces, even on precious metals

G-CEM LinkForce provides a strong link in ALL indications: universal without limitations in indications or substrate

Light-cure, dual-cure: the choice is yours

High bond strength thanks to an efficient light-curing of the bond: very thin 3um film thickness does not interfere with crown placement


Efficient self-cure mode: particularly useful when luting opaque or thick restorations

Optimal light-curing of the resin cement: ideal for luting veneers

Aesthetic & stable over time

Shades to match all needs, accompanied by their corresponding try-in pastes

Tooth-like fluorescence, colour stability over time

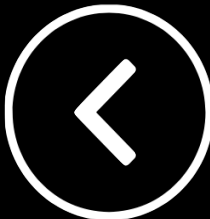

G-Multi PRIMER



Clinical Indications

Priming of the adherent surface of indirect restorations made of glass ceramics, hybrid ceramics (e.g. CERASMART), zirconia, alumina, composites and metal before luting procedures


Intra-oral and extra-oral repair of fractured indirect restorations made of ceramics, hybrid ceramics, zirconia, alumina, and composites

Features & Benefits

- Silane will provide adhesion to glass ceramics, Lithium Disilicate, Hybrid, ceramics and composites
- MDP ensures adhesion to Zirconia and non-precious metal alloys
- MDTP will ensure adhesion to precious metal alloys
- No refrigeration: 2-year shelf life

G-Multi PRIMER™ is a universal primer designed to pre-treat virtually any substrate for indirect restorations. G-Multi PRIMER™ uses three different chemical agents to ensure perfect adhesion in all situations. By adding silane to a universal primer rather than a bonding agent, the stability of the adhesion is assured.

Substrate	Pre-treatment at lab or chairside	Step 1	Step 2	Step 3	Step 4	Step 5 In case contamination with saliva/ blood before primer application	Step 6 Primer Application	Remark
Fedspathic ceramics, Leucite-reinforced ceramics	Etching at chairside	Try-in	Rinse thoroughly & dry	Etch inner surface for 60 secs with 5% HF acid	Rinse & dry	Clean with alcohol & dry	G-Multi PRIMER Apply & dry	Adhesion of G-Multi PRIMER through silane
Lithium disilicate	Etching at chairside	Try-in	Rinse thoroughly & dry	Etch inner surface for 20 secs with 5% HF acid	Rinse & dry	Clean with alcohol & dry	G-Multi PRIMER Apply & dry	
Zirconia Alumina	Sandblasting at chairside	Try-in	Rinse thoroughly & dry	Sandblast	Rinse & dry	New sandblasting or clean with Ivoclean*	G-Multi PRIMER Apply & dry	Do not clean the Zr oxide surfaces with phosphoric acid.
	Already sandblasted by lab	Try-in	Rinse thoroughly & dry			New sandblasting or clean with Ivoclean*	G-Multi PRIMER Apply & dry	Adhesion of G-Multi PRIMER through MDP
Metal Composite Hybrid Ceramics	Sandblasting at chairside	Try-in	Rinse thoroughly & dry	Sandblast**	Rinse & dry	Clean with alcohol & dry	G-Multi PRIMER Apply & dry	Adhesion of G-Multi PRIMER through silane (to glass fillers), MDP (for non-precious metal, resins) and MDTP (for precious metal)
	Already sandblasted by lab	Try-in	Rinse thoroughly & dry			Clean with alcohol & dry	G-Multi PRIMER Apply & dry	
Fibre Post	At chairside	Try-in	Rinse thoroughly & dry			Clean with alcohol & dry	G-Multi PRIMER Apply & dry	

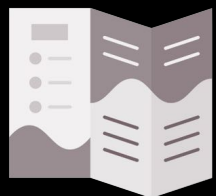

View product brochure



View instructions for use



View Technique Guide



Silane


Adhesion to glass ceramics, hybrid ceramics and composites

MDP

Adhesion to zirconia, alumina and non-precious metals

MDTP

Adhesion to precious metals

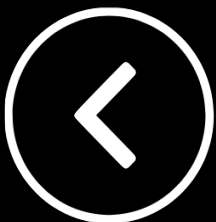


G-CEM Veneer



Introducing G-CEM Veneer: predictable aesthetics, high mechanical properties and unique flow.* G-CEM Veneer has all the ingredients to deliver a stunning result for a confident smile. The secret is in its optimally balanced formula with Full-coverage Silane Coating (FSC) technology. This innovative filler treatment allows a high filler rate to be achieved with homogeneous repartition of fillers in the matrix. The result: unique handling and excellent physical properties.

Clinical Indications

Adhesive luting of ceramic and composite veneers, inlays and onlays with a thickness (< 2.0mm) and translucency that enables the complete light-curing of the cement

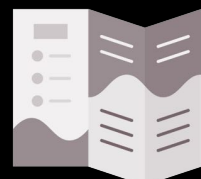

[View product brochure](#)



[View Luting Guide brochure](#)



[View Adhesive Resin Cement brochure](#)




[View instructions for use](#)




[View Technique Guide](#)

Features & Benefits of G-CEM Veneer

- Stays in place when untouched and it is fluid while put under pressure
- Covers the entire surface uniformly without voids, while the film thickness is kept to a minimum
- Not sticky; it is easy to apply with high precision and it allows easy excess removal
- Unique consistency allows for an optimally balanced flow, without the need for preheating
- Well suited for the cementation of aesthetic restorations and offers high colour stability
- G-CEM Try-in Pastes, for trial-fit of your indirect restorations
- The shades of G-CEM Try-in Paste perfectly match the four shades of G- CEM Veneer and G-Cem LinkForce
- Use of G-CEM Try-in pastes will aid in the verification of the selected shade for the desired aesthetic result
- Water-based for easy removal after application

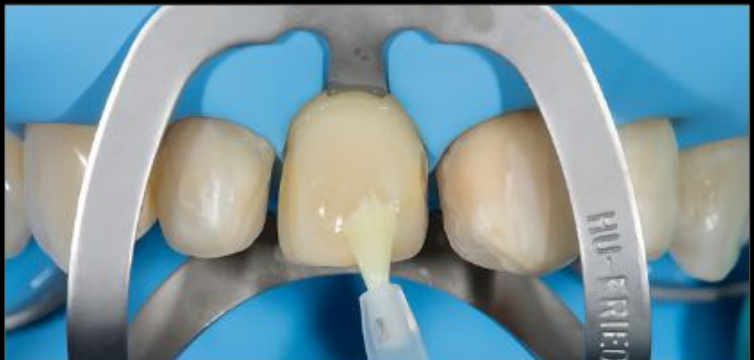

Dr. J Tapia Guadix,

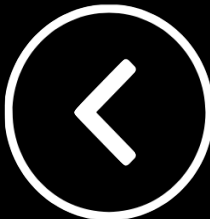

GC Fuji PLUS CAPSULE



Fuji PLUS has been designed to offer you simple and error-free cementation procedures and a strong and durable bond you can rely on. The outstanding performance and the availability in various shades make GC Fuji PLUS a multi- purpose cement.

Fuji PLUS is a resin modified glass ionomer luting cement that has consistently delivered stress-free cementation for routine crown and bridge restorations. It provides the highest level of confidence through exceptional long-term clinical results. Fuji PLUS has an extended working time for luting long-span bridge work, combination work and luting of several restorations in one step.


Clinical Indications - Cementation of

All types of metal, porcelain fused to metal, and resin crowns, inlays, onlays and bridges

Porcelain inlays

All types of core material including prefabricated pins, posts and cast posts


Cementable reinforced all ceramic crowns (e.g. Procera)

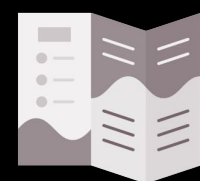

[View product brochure](#)



[View instructions for use](#)

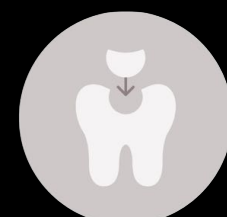

[View capsule activation guide](#)



[View Luting Guide brochure](#)

Features & Benefits

Easy mixing and handling like a conventional cement

Similar mechanical properties to resin cements

Intrinsic adhesion to tooth structure

Minimal film thickness

No post-operative sensitivity

Radiopaque

In combination with the GC Fuji PLUS Conditioner

Optimal marginal seal

Continued high fluoride release

Step-by-step technique

1. Clean the prepared teeth with pumice and water. As an option for increased retention - apply Fuji PLUS conditioner for 20 seconds. Rinse and gently dry.




2. Mix GC Fuji PLUS. Coat the internal surface of the restoration with 1mm of cement and seat within 30 seconds.



3. Remove excess cement at the first formation of gel stage (approx. 1 minute after seating).



4. Maintain isolation until the final set is verified (approx. 4 minutes).

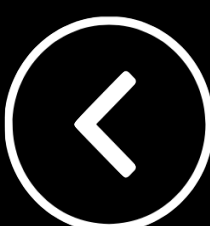

GC IMPRESSION & RELINE

Click a product image to view product information



COE Spacer Trays



GC AROMA FINE PLUS



GC EXACLEAR



Fit Checker Advanced &
Advanced Blue



Soft-Liner

ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE




COE Spacer Tray

Features & Benefits


- Disposable, strong, rigid plastic, excellent stability
- Handle will remain intact when removing the impression
- Unique, zigzag spacer, Anatomically designed
- Avoids pressure on lingual frenum and soft tissue
- Provides increased retention and maximum compression
- Perforate style/solid style
- Lessens the risk of cross contamination; no clean up or sterilisation
- Minimises contact of cusps to tray,
- Ensuring sufficient bulk of impression around the teeth

The right impression material and a good impression-taking technique go a long way to ensuring the best results; and the tray is the foundation of every accurate impression. As one of the world's leading manufacturers of impression trays and materials, GC offers a wide range of stock trays, as well as accessories and materials to make custom trays.


Full Arch Solid

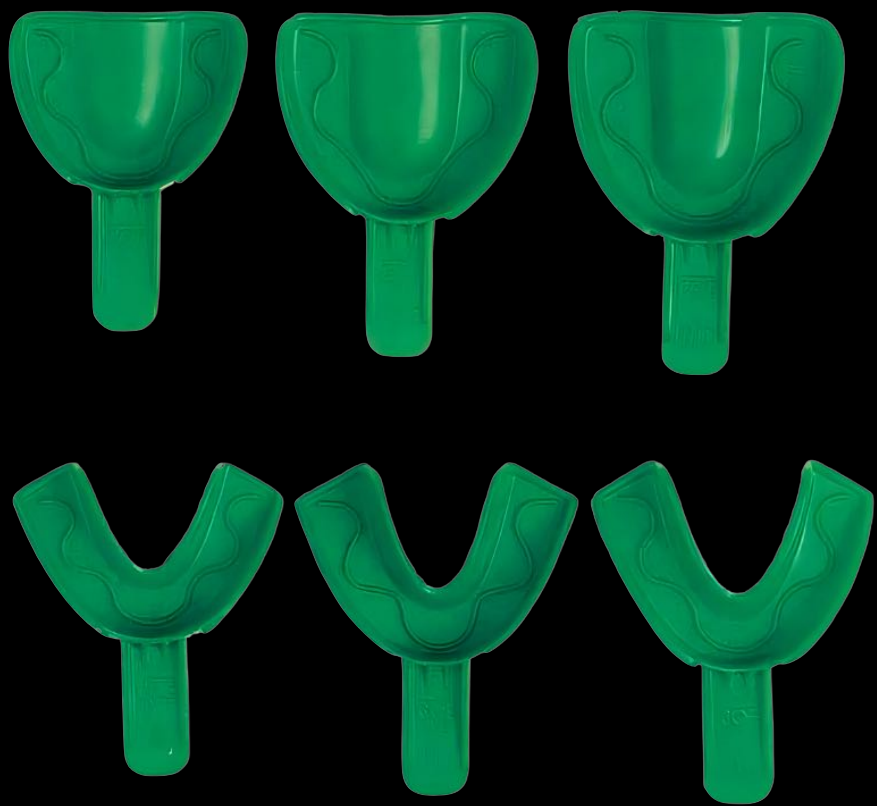

Full Arch Perforated

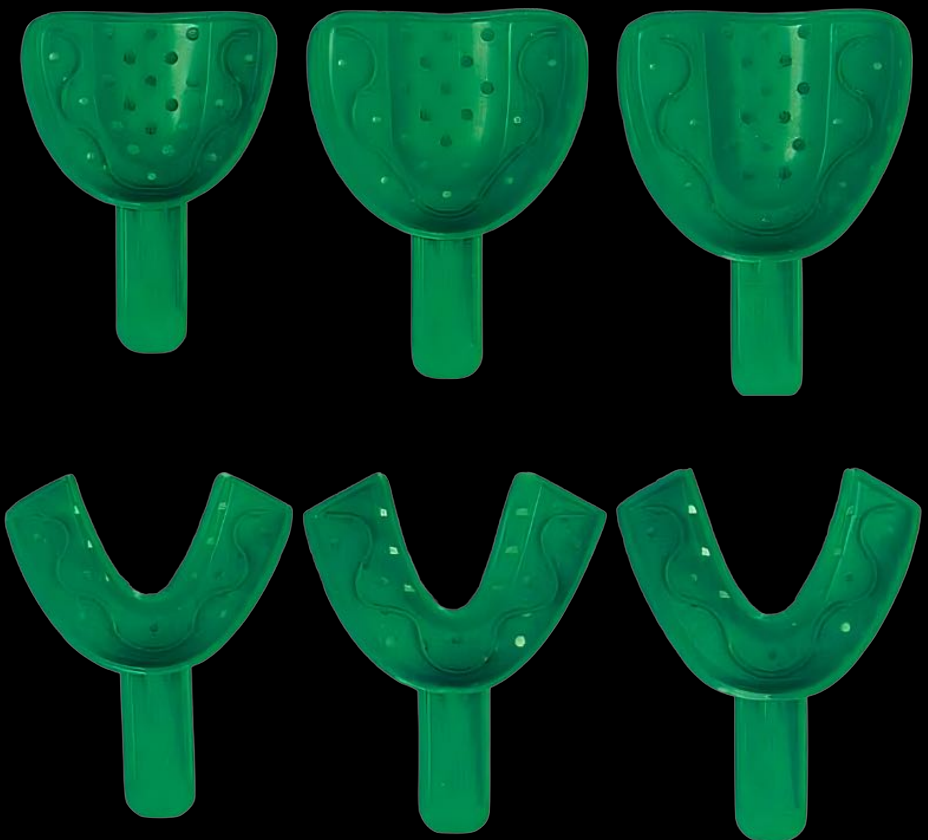


Quadrant

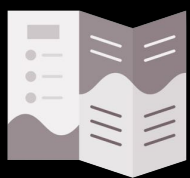

[View product brochure](#)



[View instructions for use](#)

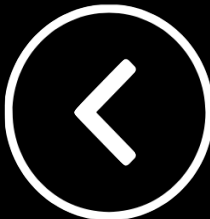

AROMA FINE PLUS

Based on GC's extensive knowledge of alginate materials, AROMA FINE PLUS has been designed to offer users final impressions of the highest quality combined with an easier and faster mixing.

Advanced formulation

Easy and rapid powder incorporation into water within just 3 seconds

Dust free formulation allowing fast and vigorous mixing for a smooth mixture without bubbles

Available in 2 setting types: Fast set (green 90 sec) and Normal set (pink 120 sec)

Ideal thixotropic property



Does not leak down the patient's throat during impression taking

Under pressure the alginate flows into the narrowest spaces for highest detail reproduction

Clinic and Laboratory optimised

Compatible with a wide range of Type IV stones for a better detail reproduction

Smooth model surface

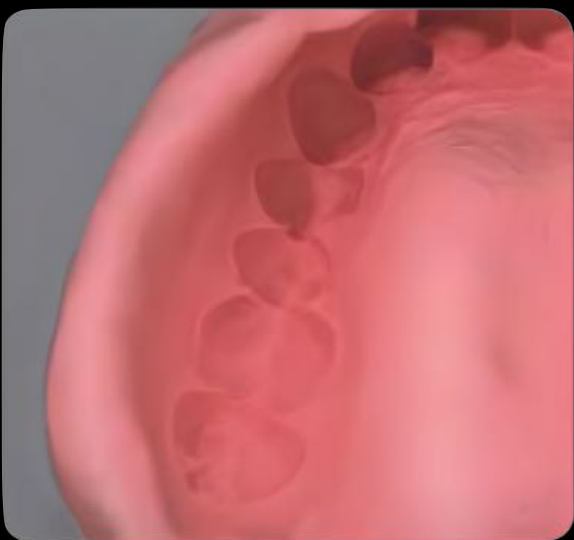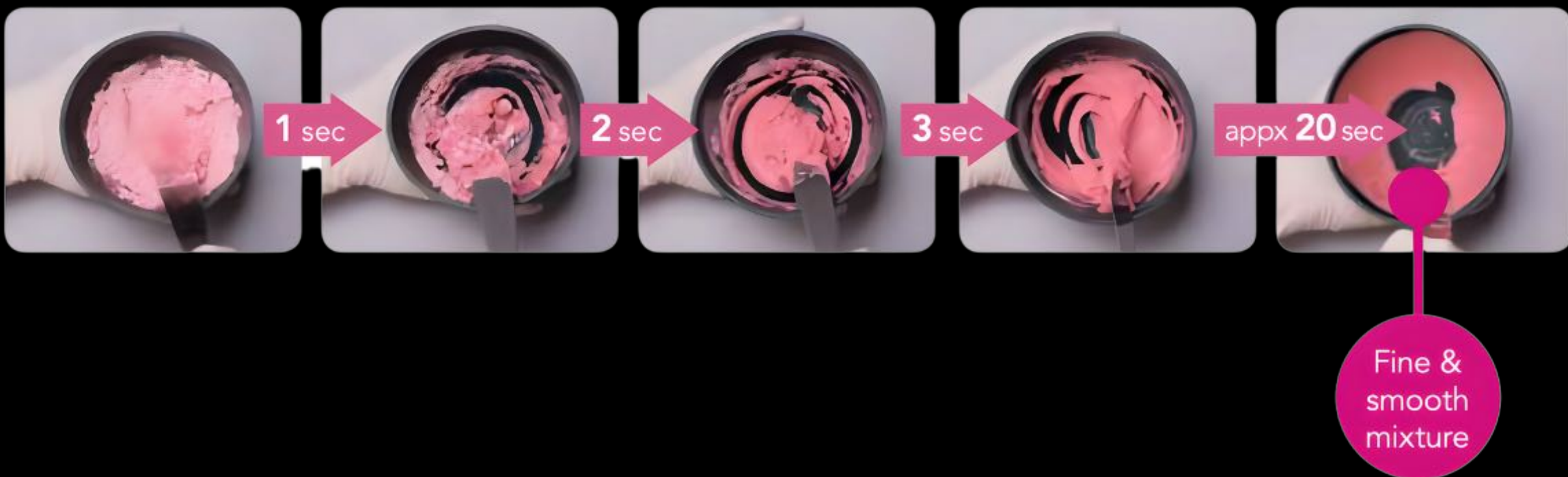


Easy-mixing of AROMA FINE PLUS results in a creamy, bubble-free consistency that captures details accurately.




AROMA FINE PLUS provides a balance of good drip resistance and fluidity to effectively provide coverage enabling detailed impression taking with minimal discomfort to patient.



AROMA FINE PLUS has good compatibility with hard stones producing smooth and precise detail reproduction on the gypsum model.

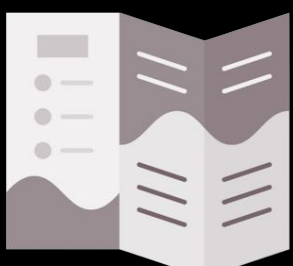

[View product brochure & Instructions for Use](#)

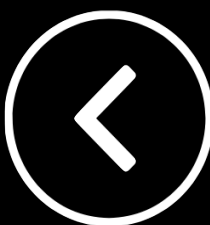

EXACLEAR




Clinical Indications

Injection Moulding Technique (IMT)
Clear stent fabrication

Features & Benefits

- High transparency - crystal clear for absolute control
- Very effective light-curing through the silicone
- Optimal consistency - balanced handling and flow under pressure
- Intra-oral setting time - 2 mins
- Extra-oral setting time- 7 minutes

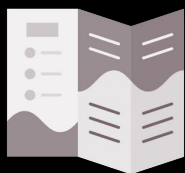

View product brochure



View instructions for use



View Injection
Moulding Guide



View Anterior Injection
Moulding Video




View Posterior Injection
Moulding Video

EXACLEAR™ is an innovative, clear silicone material that fulfils the needs of demanding aesthetic cases and facilitates consistent results. It's absolute transparency offers total visual control making it ideal for creating outstanding aesthetic results in complex clinical situations.

EXACLEAR™ offers the perfect balance of properties for ease of handling, staying in place when extruded but flowing nicely under pressure, resulting in perfect adaptation and reproduction of details. EXACLEAR™ is ideally suited for both chair side and laboratory use, the injection moulding technique and clear stent fabrication for creation of provisional and permanent restorations.



Dr. Anthony Tay

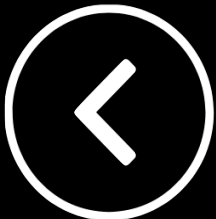

FIT CHECKER™ ADVANCED & FIT CHECKER™ ADVANCED BLUE

Vinyl Polyether materials for checking pressure spots and the fit accuracy of prosthetic works.

Thanks to their very thin consistency, sharp setting time and hydrophilicity, both materials are very convenient to use and offer an accurate identification of misfits of prosthetic works of all kinds.

The choice between 2 transparent, well contrasting colours allows an easy evaluation of any type of prosthesis: blue, for aesthetic restorations such as ceramic based crowns & bridges and for occlusal checking; white, for metal based restorations and dentures.

Clinical Features & Benefits

- ✓ Ideal consistency for easy flow onto dentures and oral tissues
- ✓ Excellent wettability against saliva coated surfaces
- ✓ Sufficient working time - up to 1min
- ✓ Rapid setting time - 1 min in the mouth
- ✓ A perfect colour gradation for clear and precise visualisation of poorly fitting areas
- ✓ Easy adjustments - the set material does not get caught by the bur while relieving pressure spots.
- ✓ Easy to peel off ensuring a fast, clean procedure.



Clinical Indications

- ✓ Checking fit for indirect restorations - ceramics, metals, zirconia, acrylic
- ✓ Evaluation of occlusal contacts
- ✓ Fit of occlusal splints
- ✓ Can be used for bite registration



Apply & Seat



Check & Adjust



Ideal for metal restorations




Seat



Check & Adjust



Bite Registration

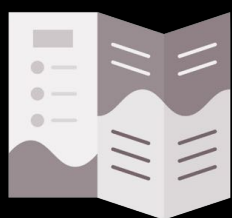

[View product brochure](#)



[View instructions for use](#)



SOFT-LINER - TISSUE CONDITIONER

Soft relining material for short-term use, including functional impression taking using existing removable dentures.

- Tissue conditioning
- Temporary relining
- Functional impression taking

Contradictions


Avoid use of this product in patients with known allergies to methacrylate polymer, ethanol or phthalic ester-based plasticiser.


Clinical Features & Benefits

- Fast application
- No heat irritation
- Easy to trim
- Good retention
- Smooth surface minimises unwanted food debris retention
- Simple to remove from the denture at the end of the treatment



[View instructions for use](#)

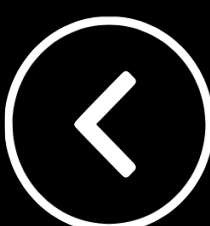

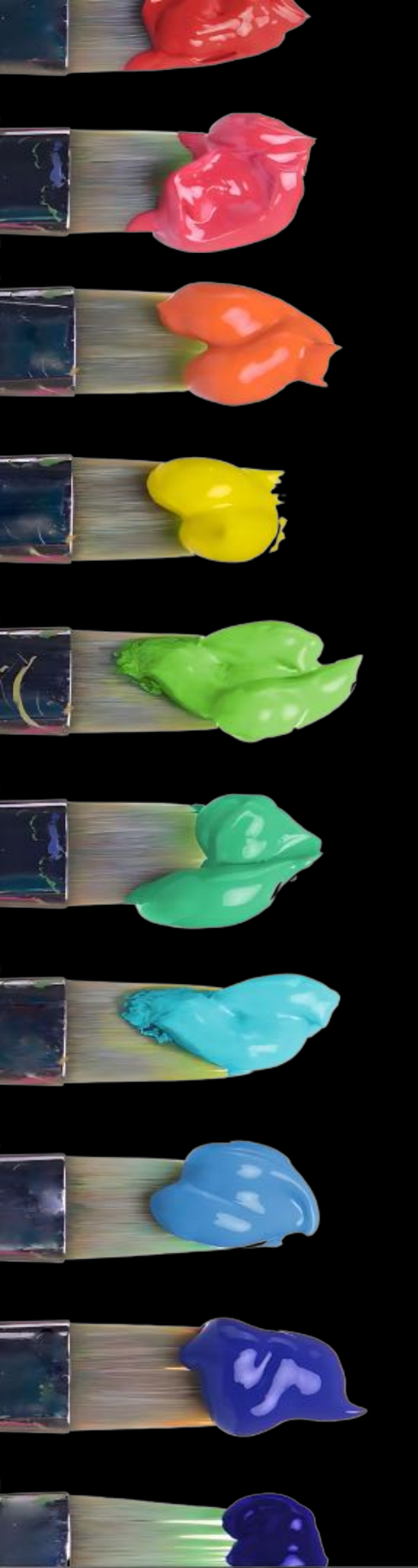


Subscribe to the GC CPD Monthly Newsletter

Webinars | Courses | Events



GC SMART GUIDE INSTRUCTIONS


1. Click to select a master product category

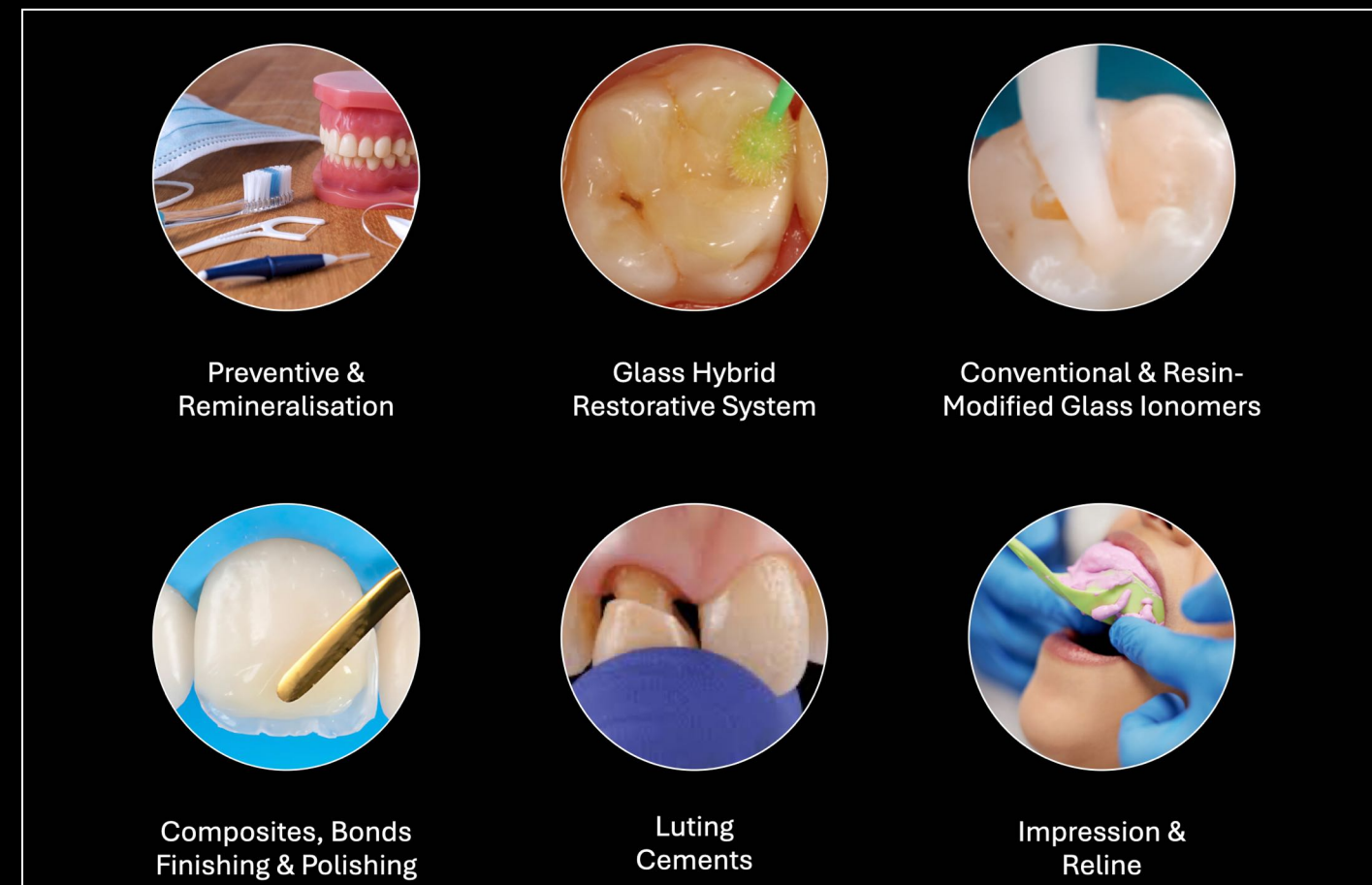


2. Click a product image to view product information



7. Click the product name to open the Henry Schein Online product order page

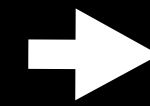

3. The selected product information will be displayed

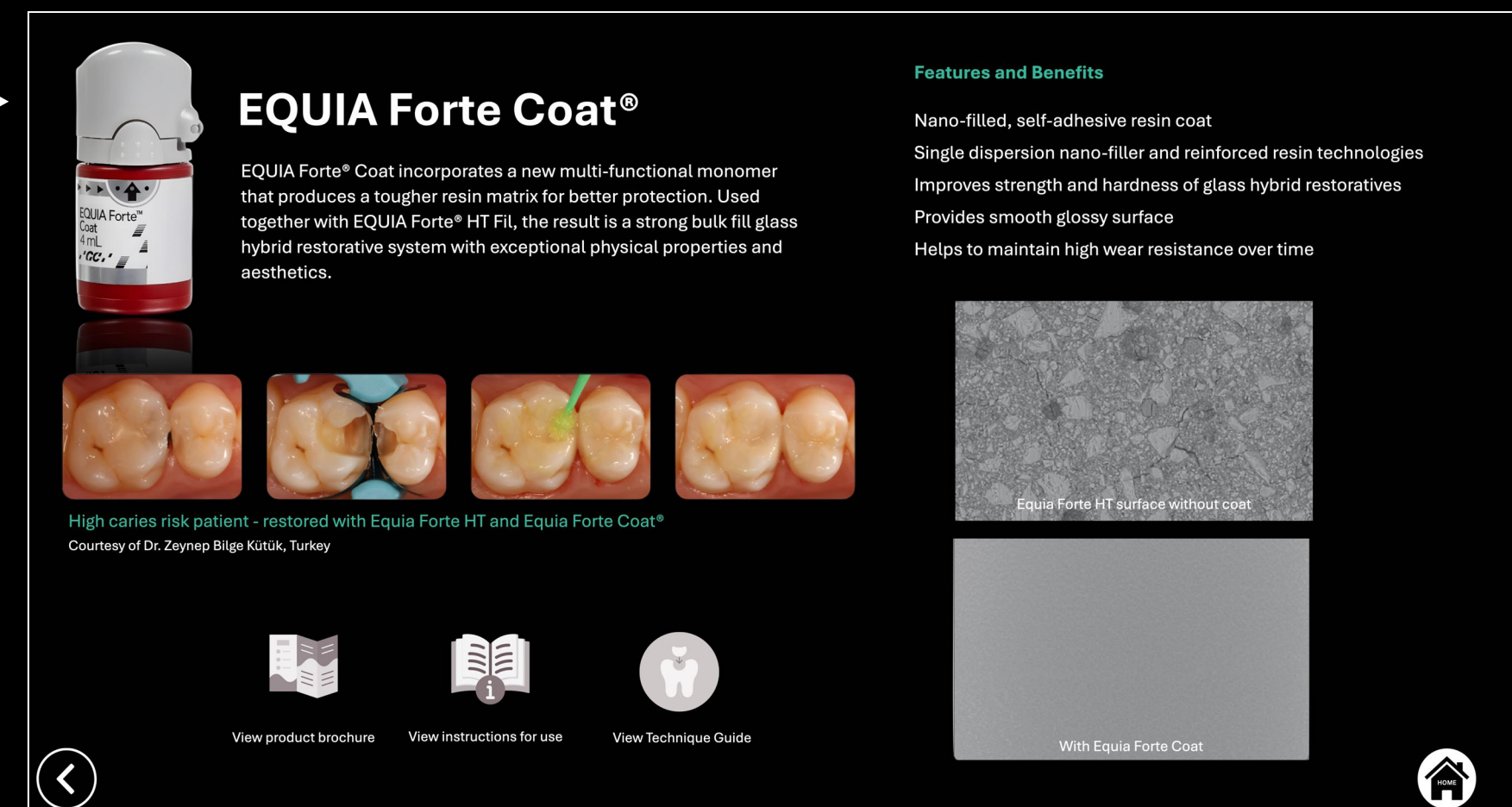

4. Click the back button on any product page to go back to the product selection page for that category



5. Click the home button on any page to go back to the master product category selection page



6. Click the icons on the product information page to view further information, brochures and instructions for use



GC PRODUCT SPECIALIST CONTACT

Click an email address to automatically open your e-mail client and send an email or click a phone number to call a product specialist



GC Australasia Dental Pty Ltd (Head Office)

1753 Botany Road Banksmeadow

NSW 2019 Australia

info.australasia@gc.dental

www.gc.dental/australasia

+61 2 9301 8200

NSW | ACT

Leonie De-Bellis

leonie.de-bellis@gc.dental

+61 436 466 411

Sue Troxler

sue.troxler@gc.dental

+61 455 479 843

Gina Ghabrous

gina.ghabrous@gc.dental

+61 413 559 113

Rushan Dissanayake

rushan.dissanake@gc.dental

+61 417 416 737

QLD

Haylea Burman

haylea.burman@gc.dental

+61 400 395 774

Zenda Foon

zenda.foon@gc.dental

+61 421 336 171

SA / VIC

Helen Dunn

helen.dunn@gc.dental

+61 408 548 490

VIC

Amanda Smart

amanda.smart@gc.dental

+61 407 529 638

Bridget Smith

bridget.smith@gc.dental

+61 418 313 533

WA

Michele Gates

michele.gates@gc.dental

+61 401 887 331

VIC | TAS | NT

Tricia Hogan

tricia.hogan@gc.dental

+61 400 836 922

