Prior to use, carefully read the instructions for use.

EN

## BITE COMPOUND

IMPRESSION COMPOUND FOR BITE REGISTRATION

For use only by a dental professional in the indications for use.

**INDICATIONS FOR USE**  
Checking occlusal registrations.

### DIRECTIONS FOR USE

Soften over a flame or in a warm water bath and adapt in this plastic state. The right moment to take the impression is when BITE COMPOUND does not show any more threads when pulled apart, but can still easily be formed. Note: Do not use on patients whilst too hot (over 45°C).

### STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

### PACKAGE

Standard package: 120 g (15 sticks)

### CAUTION

- In case of contact with eyes, remove and flush with water.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gc.dental/europe> They can also be obtained from your supplier.

**Undesired effects- Reporting:**  
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental) In this way you will contribute to improve the safety of this product.

Last revised: 04/2025

CE

'GC'

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

MANUFACTURED by  
NIMI CHEMICAL INDUSTRIAL CO., LTD.  
4-13 Yokoyama-cho, Kiryu, Gunma 376-0055, Japan

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