BROWN SILICONE POINTS

RUBBER POLISHING POINTS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Polishing of prosthetic metals such as gold, silver-palladium, nickel chrome, etc.

CONTRAINDICATIONS

In vivo use.

DIRECTIONS FOR USE

When polishing with the BROWN SILICONE POINT, use the following recommended handpiece speeds (RPM).

#113, 128, 103, 104, 110 : less than 30,000 RPM.
#114, 115 : less than 10,000 RPM.

NOTES

1. Prior to use, make sure that the shaft of the BROWN SILICONE POINT is securely fixed into the handpiece according to the handpiece manufacturer's instructions.
2. Do not use a damaged shank (for example showing signs of rusting, superficial bruising, bending or contamination).
3. Before using, check that the shank of the point rotates correctly without distortion.
4. For safety, be sure to use the recommended rotation speed.
5. Do not use excessive pressure when working with the point.
6. When using the point, wear a dust mask to avoid inhaling cutting dust.
7. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
8. When disinfecting the point, do not use highly metal corrosive liquids, for example containing chloride. When soaking in other disinfectants for an extended period of time, please use an anti-rusting agent.

SHAPES

7 shapes

STORAGE

Keep the BROWN SILICONE POINTS dry and store in a dry place to avoid rusting.

PACKAGE

Box of 12 pieces in HP

CAUTION

1. In case of contact of cutting dust with eyes, flush immediately with water and seek medical attention.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

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GC DENTAL (SUZHOU) CO.,LTD.