CAPSULE MIXER CM-II
DENTAL CAPSULE MIXER

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE
Mixing of glass ionomer cements and other preformed cements in capsules

IDENTIFICATION OF PARTS

- Safety cover
6. Timer selection button
- Capsule holder
7. Time display
- Power indicator
8. Start button
- Mode button
9. Adapter
10. Power cord

SETUP

- Remove the two black transportation screws from the bottom of the device (Fig. 1). Keep the device as horizontal as possible when unscrewing. Loosen screws 1 and 2.
- Store the screws in the storage holes (Fig. 2). The screws will be necessary again in case of breakdown or transport in the future.
- Firmly connect the power cord to the device and subsequently to the main power socket (100-240 V AC) (Fig. 3).

SAFETY INSTRUCTIONS

- The unit should only be used by a qualified dental professional according to the instructions.
- The unit should not be exposed to direct sunlight, dust, a wet environment or placed near a heater.
- Keep out of the reach of infants or children.
- For use, make sure the unit is properly connected to the appropriate power source.
- The unit should be checked on a regular basis.
- Turn on the power switch and the mode indicator and time display will light up.
- Install the unit with a flat and stable place.
- In case of breakdown, disconnect the unit and appropriately label the unit with a qualified person. The repair of the unit may be carried out by a dental professional.

DIRECTIONS FOR USE
Make sure the safety cover is tightly closed. If the cover is open, OF is shown on the time display.

- Turn on the power switch. The mode indicator and time display will light up.
- Set the mixing time according to the manufacturer's instructions of the encapsulated product. Two preferred time settings can be saved as mode 1 or mode 2. Both modes are preset at the factory at 10 seconds. To change the time setting, follow the below steps.
- Press either "+" or "-" once to activate display starting flashing.
- Press either "+" or "-" again to set the desired mixing time by 1 sec. Mixing time in the range of 1-30 seconds may be selected.
- Press the mode button to set/save the selected mixing time (display stops flashing).
- Open the cover. Hold the capsule with the nozzle facing upward, install it in the capsule holder and press down until it fits in the fixed arm as shown in figure 4.
- Close the cover and press the start button. Once the mixer starts, the time display starts counting down. The cover will not start if the cover is open.
- Once the selected mixing time has finished, the cover can be opened. Remove the mixed capsule from the capsule holder. Make sure the capsule nozzle is facing upward when removed from the holder (Fig. 5).
- Do not attempt to start mixing without a capsule being installed in the device. This may cause breakage of the device. Breakage caused by improper use will be covered by the warranty.
- In case of emergency, the mixing process can be stopped by opening the safety cover.
- Oil or particles and material comes into contact with the device, wipe off immediately.

MAINTENANCE AND CARE
Put the device under direct sunlight, strong heat sources and dusty areas. Prior to routine cleaning and disinfection, disconnect the power cord from the main socket. The device can be cleaned with a soft cloth soaked in warm water or mild detergent. Avoid using volatile solvents.

TROUBLESHOOTING
If any difficulty with the device is experienced, please check below for possible causes of the problem and suggested corrective actions before seeking further advice or repair from an authorized dealer.

Problem	Check	Corrective action
The power switch is on but time display or mode indicator is not lit	Is the power cord correctly connected to the main socket? Is the adapter correctly connected to the device?	Correctly connect the power cord to the main socket. Correctly connect the adapter to the device.
The device does not start mixing even though the start button is pressed	Is the cover closed?	Close the cover tightly.
The device moves during mixing	Is the device placed on a flat surface?	Place the device on a flat surface or use a rubber mat under the device.

WARRANTY / REPAIR
Warranty: 2 years from the date of purchase.
In case of a breakdown during the warranty period, repair will be carried out free of charge provided that the unit has been used under normal conditions and conforms to the instructions for use.

DISPOSAL
Dispose of the unit according to the local regulations.
Your product is marked with the following symbol:
For an environmental responsible recycling / disposal of your product. Do not mix plastic with other normal waste. When using the crossed-out waste bin symbol, means that this product has been put in circulation after the 08.08.2005.
Please be aware that this product is submitted to the European Directive 2002/96/EC (WEEE) and the laws applicable in your country for recycling / disposal of electrical equipment.
Please ask your dealer or GC Europe when your product will be disposed off.

PACKAGING
Mixer 1
AC adapter 1
Power cord (EU type, UK type) 2

SPECIFICATIONS

Equipment class	Class I (For medical device)
Protection from ingress of liquid (IPX) (optional equipment)	Class II Electrical appliance (For equipment class I)
Observe instructions for use	Δ
Power supply voltage	DC 24 V
Power input	50 VA
AC Adapter voltage	INPUT: 100-240 V AC 50/60 Hz OUTPUT: DC 24 V 1.5 A
Mixing speed	Approx. 4,000 rpm
Mixing time	Selectable from 1-30 seconds
Noise level	< 70 dB(A)
Ambient temperature during use	10°C - 40°C
Storage condition	Temperature: 10°C-60°C Humidity: 10%-85% Pressure: 500-1060 hPa
Dimensions	(Device) W x L x H 190 mm x 180 mm x 150 mm (Adapter) W x L x H 190 mm x 120 mm x 40 mm
Weight	2.9 kg (including adapter)
COMPLIANCE	EN60601-1 EN60601-1-2

The specifications of the unit may be subject to change without notice.

CAUTION
Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Undesired effects of any kind:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country.
https://ec.europa.eu/health/information_systems/index_en.htm
https://ec.europa.eu/health/information_systems/index_en.htm
As well as to our internal vigilance system: vigilance@gcc.dental
In this way you will contribute to improve the safety of this product.

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