Fig. 1

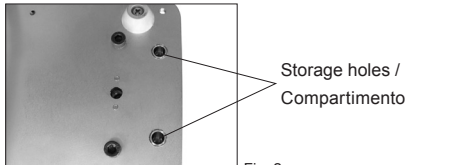
Fig. 2

Fig. 3

Fig. 4

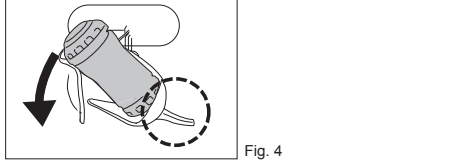
Fig. 5

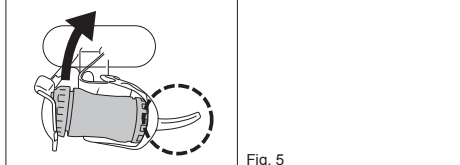
Fig. 6

CAPSULE MIXER CM-II

DENTAL CAPSULE MIXER

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE
Mixing of glass ionomer cements and other predestined cements in capsules

IDENTIFICATION OF PARTS
1. Safety cover 6. Timer selection button
2. Capsule holder 7. Time display
3. Power switch 8. Start button
4. Mode button 9. Adapter
5. Mode indicator 10. Power cord

SETUP
1. Remove the two black transportation screws from the bottom of the device (Fig. 1). Keep the device as horizontal as possible when unscrewing. Loosen both screws evenly.
2. Store the screws in the storage holes (Fig. 2). The screws will be necessary again if the device has to be transported in the future.
3. Firmly connect the power cord to the device and subsequently to the main power socket (100-240V AC) (Fig. 3).

SAFETY INSTRUCTIONS

- The unit should only be used by a qualified dental professional according to the instructions.
- The unit should not be exposed to direct sunlight, dust, a wet environment or placed near a heater.
- Keep out of the reach of infants or children.
- Prior to use, make sure the unit is properly connected to the appropriate power source.
- The unit should be checked on a regular basis.
- Do not handle the unit with wet hands.
- Details of the unit on a flat and stable place.
- In case of breakdown, disconnect the unit and appropriately label the unit with details of possible fault. The repair of the unit may only be carried out by a qualified person.

DIRECTIONS FOR USE

Make sure the safety cover is tightly closed. If the cover is open, OF is shown on the time display.
1. Turn on the power switch. The mode indicator and time display will light up.
2. Set the mixing time according to the manufacturer's instructions of the encapsulated product. Two preferred time settings can be selected as mode 1 or 2. Both modes are preset at the factory at 10 seconds. To change the time setting, follow the below steps.
a) Press either "+" or "-" once to activate (display starts flashing).
b) Then press "+" or "-" button to increase or decrease the mixing time by 1 sec.
c) Press the mode button 1 or 2 by pressing the mode button.
d) Press the mode button to set/save the selected mixing time (display stops flashing).
3. Open the cover. Hold the capsule with the nozzle facing upward, install it firm in the swing arm of the capsule holder and press down until it fits in the fixed arm as shown in figure 4.
4. Close the cover and press the start button. Once the mixer starts, the time display starts counting down.
Note:
For safety reasons, the mixer will not start if the cover is open.
Once the selected mixing time has finished, the cover can be opened. Remove the mixed capsule from the capsule holder. Make sure the capsule nozzle is facing upward when removing from the holder (Fig. 5).
Note:

- Do not attempt to start mixing without a capsule being installed in the device. This may cause breakage of the device. Breakage caused by improper use is not covered by the warranty.
- In case of emergency, the mixing process can be stopped by opening the safety cover.
- If a capsule leaks and material comes into contact with the device, wipe off immediately.

MAINTENANCE AND CARE

Prior to device away from direct sunlight, strong heat sources and dusty areas. Put to routine cleaning and maintenance of the unit, disconnect the power cord from the main socket. The device can be cleaned with a soft cloth soaked in warm water or mild detergent. Avoid using volatile solvents.

TROUBLESHOOTING

If any difficulty with the device is experienced, please check below for possible causes of the problem and suggested corrective actions before seeking further advice or repair from an authorized GC Dealer or Service center.

Problem	Check	Corrective action
The power switch is on but time display or mode indicator is off.	Is the power cord correctly connected to the main socket?	Correctly connect the power cord to the main socket.
The device does not start mixing even though the start button is pressed.	Is the adapter correctly connected to the device?	Correctly connect the adapter to the device.
The device moves during mixing.	Is the device closed?	Close the cover tightly.
	Is the device placed on a flat surface?	Place the device on a flat surface or use a rubber mat under the device.

WARRANTY / REPAIR
Guarantee: 2 years from the date of purchase.
Your product is marked with the following symbol:  for an environmental responsible recycling / disposal of your product. Do not discard it with normal waste. The black line, under the crossed waste bin symbol, means that this product has been accepted for circulation after the 13.08.2005 (see 2002/96/EG and DIN EN 50419:2005). Please be aware that this product is submitted to the European Directive 2002/96/EG (WEEE) and the laws applicable in your country for the environmentally responsible recycling / disposal. Please ask your dealer or GC Europe when your product will have to be disposed off.

DISPOSAL
Dispose of the unit according to the local regulations.
Your product is marked with the following symbol:  for an environmental responsible recycling / disposal of your product. Do not discard it with normal waste. The black line, under the crossed waste bin symbol, means that this product has been accepted for circulation after the 13.08.2005 (see 2002/96/EG and DIN EN 50419:2005). Please be aware that this product is submitted to the European Directive 2002/96/EG (WEEE) and the laws applicable in your country for the environmentally responsible recycling / disposal. Please ask your dealer or GC Europe when your product will have to be disposed off.

PACKAGING
Adapter 1
Power cord (EU type, UK type) 2

Specifications	
Equipment class	Class I (For medical device) ☒ Class II Electrical appliance (For equipment class)
Protection from ingress of liquid	IPX0 (Ordinary equipment)
Observe instructions for use	
Power supply voltage	DC 24 V
Power input	50 VA
AC Adapter voltage	INPUT 100-240 V AC 50/60 Hz OUTPUT DC 24 V 1.5 A
Mixing speed	Approx. 4,000 rpm
Mixing time	Selectable from 1-30 seconds
Noise level	<70 dB(A)
Ambient temperature during use	10°C-40°C
Storage condition	Temperature -10°C-60°C Humidity 10%-85% Pressure 500-1060 hPa
Dimensions	(Device) W x L x H 190 mm x 180 mm x 150 mm (Adapter) W x L x H 60 mm x 120 mm x 40 mm
Weight 2.9 kg (including adapter)	
COMPLIANCE	CE EN60601-1 EN60601-1-2

The specifications of the unit may be subject to change without notice.

CAUTION
Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom
Last revised 05/2023

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