Prior to use, carefully read the instructions for use.
**EN**

## CERAMIC PRIMER

PRIMER FOR CERAMIC BONDING

CERAMIC PRIMER is a priming agent designed to firmly bond ceramic prostheses to tooth structure using resin adhesive cement as well as to bond laboratory resins, composites and acrylics to ceramic surfaces.

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**

Priming the bonding surfaces of ceramic (porcelain) prostheses.

**CONTRAINDICATIONS**

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**DIRECTIONS FOR USE**

**I. For Operatory Use**

- Treatment of ceramic prosthesis with phosphoric acid etchant Using an ultrasonic cleaner or other device, clean the ceramic prosthesis and dry with an air syringe. Apply phosphoric acid etchant to the bonding surface, rinse with water and dry according to manufacturer's instructions for use.
- Mixing and Application of CERAMIC PRIMER Dispense one drop each of CERAMIC PRIMER Liquid A and Liquid B into a dispensing dish. Mix for 5 seconds using a Micro-tip applicator, etc. Apply a thin layer to the bonding surface of the ceramic prosthesis and dry with an air syringe.
- Cementation Apply a suitable resin adhesive cement to the primed surface of the prosthesis. For details, refer to manufacturer's instructions for use.

**II.For Laboratory Use**

- Roughen the bonding surface of porcelain fused to metal using a carbundrum point, etc. or sandblast the bonding surface with 110µ aluminium oxide. Clean with dry, oil-free air. Note : When using a hydrofluoric acid etching gel, it is not necessary to apply a phosphoric acid solution in the following step No.2.
- Apply a phosphoric acid solution to the roughened bonding surface of the porcelain. Rinse with water and dry according to manufacturer's instructions.
- Mix CERAMIC PRIMER A and B liquids. Apply to the etched bonding surface and softly dry with air.
- Apply GC COMPOSITE PRIMER to the primed bonding surface and spread thinly using an air syringe. Light-cure for 1 minute using the GC LABOLIGHT LV-III.
- Apply light-cured composite resin such as GC GRADIA or GC GRADIA gum shades to the bonding surface.

**STORAGE**

Recommended for optimal performance, store in a cool and dark place (1 - 28°C / 33.8 - 82.4°F). (Shelf life : 2 years from date of manufacture)

**PACKAGE**

1-1 package : A 2mL, B 2mL

**CAUTION**

- Wear protective gloves and eye protection
- CERAMIC PRIMER is highly flammable liquid and vapor. Do not use near naked flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
- Take care to avoid ingestion of the material.
- In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Flush immediately after cementing.
- In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Replace caps immediately after use. If dew condensation is present on the bottle of CERAMIC PRIMER A, immediately wipe off.
- Do not mix with any other primers.
- Store in a well-ventilated place. Keep cool.
- Dispose of all waste according to local regulations.

The Safety Data Sheets for this product are available at http://www.gceurope.com/download They can also be obtained from your supplier.

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