Prion to use, carefully read the instructions for use.

CERAMIC PRIMER II

PRIMER FOR CERAMIC AND COMPOSITE BONDING

CERAMIC PRIMER II is a priming agent designed to treat the adherend surface of ceramic and composite prostheses to promote a strong bond to resin-based materials as resin-based cements in case of luting procedures or direct composites in case of repairs.

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

1. Priming of the adhering surfaces of indirect restorations made of ceramics (glass ceramics, zirconia and alumina), hybrid ceramics (e.g. CERASMART) and composite before luting indirect restorations made of ceramics, hybrid ceramics and composites.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or ethanol.

COMPOSITION

Ethanol, dimethacrylate, phosphoric acid ester monomer, silane coupling agent

DIRECTIONS FOR USE

- For luting procedure
- To treat the bonding surfaces of prostheses according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. We recommend cleaning in an ultrasonic bath or with alcohol, then rinse and dry with an oil-free air syringe.
- Bottle type: Dispense one drop of CERAMIC PRIMER II into a dispensing dish. Unit dose type: Unscrew cap of unit dose counter-clockwise. Make sure to identify the top and bottom of the unit dose capsule before opening.
- Apply a thin layer to the bonding surface of the ceramic or composite prosthesis using a micro-pip applicator and dry with an air syringe.
- Apply suitable resin adhesive to the primed surface of the prosthesis. For these steps, refer to the manufacturer's instructions for use of the corresponding luting cement.

For Intra-oral Repair

1. Roughen the bonding surface of ceramic or composite using a coarse diamond bur or a carbundrum point.
2. Treat the bonding surfaces of the prosthesis according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.
3. Bottle type: Dispense one drop of CERAMIC PRIMER II into a dispensing dish. Unit dose type: Unscrew cap of unit dose capsule before opening.
4. Apply a thin layer to the fractured surface of the ceramic or composite prosthesis using a micro pip applicator and dry with an oil-free air syringe.
5. Continue the repair by using a light-cure adhesive (e.g. G-Premio BOND) and a light-cure composite (e.g. G-anial Universal Flo), referring to their respective manufacturer's instructions for use.

If the treated adherend surface is contaminated with saliva or blood, rinse with water and dry. Re-treat the surface according to the respective manufacturer's instructions and re-apply CERAMIC PRIMER II according to the instructions above.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

PACKAGING

- CERAMIC PRIMER II, 1 x 1 bottle of 3 ml
- CERAMIC PRIMER II, 10 x unit dose of 0.1 ml

CAUTION

1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. Avoid contact with oral tissues. Immediately remove with a cotton pellet or a sponge. Rinse immediately after luting or repairing.
3. In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
4. In case of contact with eyes, flush immediately with water and seek medical attention.
5. Replace the cap of bottle immediately after use. If weak condensation is present on the bottle of CERAMIC PRIMER II, immediately wipe it off.
8. Unit dose is disposable. Do not recar or reuse to avoid cross contamination between patients.
9. If the cap of the unit dose is too tight to open, do not try to open it with force to avoid accidental contact of the liquid with skin and eyes. Use another unit dose.
10. Do not deal with any other primers.
11. Dispose of all waste according to local regulations.
12. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and consult a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gce.dental/europe>

THEIR II, nezávadná látka, produkt z može da priniesť zdravotnú ujavu. Do not immerse. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUADAMED (<https://ec.europa.eu/tools/euadamed/>) or contact us at Regulatory.gce@gce.dental

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them to the relevant vigilance system, by selecting the primary authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gce.dental. In this way you will contribute to improve the safety of this product.

Last revised: 01/2025

MANUFACTURED BY: GCE DENTAL PRODUCTS CORP. 2-285 Tomiatsu-cho, Kasugai, Aichi 486-0844, Japan

GCE CORPORATION 1-1-1 Nishikuji, Itabashi-ku, Tokyo 147-8585, Japan

EU LEU: GCE EUROPE NV. 1000 Leuven, Belgium 1000, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

PRINTED IN JAPAN

Prion to use, carefully read the instructions for use.

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