**COE-PAK™**  
**COE-PAK AUTOMIX NDS™**

PERIODONTAL DRESSING (NO EUGENOL; NO ASBESTOS)  
For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**  
For surgical dressing and periodontal pack. Use COE-PAK whenever a periodontal dressing is required.

**CONTRAINDICATIONS**  
In rare cases this material may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**DIRECTIONS FOR USE**

- a) For non-automix, use a spatula and a COE mixing pad. Do not use a rigid or narrow spatula. Through mixing is very important. This is more easily done with a wide, flexible spatula.
- b) When using COE-PAK AUTOMIX NDS, load the cartridge into the GC Cartridge Dispenser II and attach a mixing tip. Express the required length of COE-PAK onto the mixing pad (do not use more than 4 full strokes of the trigger). Leave the AUTOMIX NDS mixing tip in place until required for the next use.
- For non-automix, extrude equal lengths of paste from catalyst and base tubes. Spatulate for 30-45 seconds or until the two pastes are a uniform color. If you would like a harder dressing, add a small amount of zinc oxide powder to the pink paste (catalyst) before spatulating.
- Using a spatula, form the mixed pastes into a cylindrical shape.
- Lightly lubricate your fingers with a thin film of vaseline, lanolin, cold cream, or water and test the mixed pastes for tackiness (it should lose its tackiness after 2-3 minutes; 1 minute for Hard & Fast Set). NOTE: COE-PAK AUTOMIX NDS can be handled after about 30 seconds. Lubrication of fingers is not necessary.
- COE-PAK can be handled and molded 2 to 3 minutes after mixing. With your fingertips, roll the cylindrical form off the COE mixing pad (1 minute for Hard & Fast).
- COE-PAK remains workable for 10-15 minutes (5-8 minutes for Hard & Fast and at least 8 minutes for AUTOMIX NDS). During this time it can be formed in the usual manner into ropes or other desired shapes.
- During the working stage COE-PAK is extremely cohesive. It can be formed into ropes of virtually any diameter for easy placement against tissue and teeth.

**COMPARISON OF COE-PAK/COE-PAK HARD & FAST/COE-PAK AUTOMIX NDS**  
(equal lengths - 1/2 minute mixing time)

	COE-PAK REGULAR	COE-PAK HARD & FAST	COE-PAK AUTOMIX NDS
After mixing, ready to use	1 minute	1 minute	30 seconds
Consistency	plastic	heavy, but still plastic	plastic
Working time	10-15 minutes	5-8 minutes	at least 8 minutes
Ability to bond one piece of material to another	10-15 minutes	5-8 minutes	8-12 minutes
Final hardness	30 minutes (still resilient but not brittle)	10 minutes (much harder than COE-PAK, but not brittle)	10 minutes

	COE-PAK REGULAR	COE-PAK HARD & FAST	COE-PAK AUTOMIX NDS
Alter mixing, ready to use	1 minute	1 minute	30 seconds
Consistency	plastic	heavy, but still plastic	plastic
Working time	10-15 minutes	5-8 minutes	at least 8 minutes
Ability to bond one piece of material to another	10-15 minutes	5-8 minutes	8-12 minutes
Final hardness	30 minutes (still resilient but not brittle)	10 minutes (much harder than COE-PAK, but not brittle)	10 minutes

**CAUTION**

- Sensitivity to some of the components of this product has been reported. The reported sensitivity varies from simple dermal redness, to rash, swelling and nausea and/or vomiting. The practitioner should assure, prior to use, no such sensitivity to this product is present in the intended patient.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of contact with skin, wash with soap and water. If irritation develops, seek medical attention. NOTE: GC Carridge Dispenser II is suggested for use with the NDS, New Delivery System cartridge.
- Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

**STORAGE**  
Recommended for optimal performance, store COE-PAK in a dry location at room temperature (70° to 77°F; 21° to 25°C).

**PACKAGES**

- COE-PAK Standard (or Regular) Package 90g tube base x 1, 90g tube catalyst x 1
- COE-PAK Hard and Fast Package 90g tube base x 1, 90g tube catalyst x 1 Cartridges
- 2-Pack COE-PAK AUTOMIX NDS Cartridges

**CLEANING AND DISINFECTING RECOMMENDATION**  
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients the bottles and measuring devices require mid-level disinfection. Immediately after use inspect the bottles, measuring devices and label for deterioration. Discard if damaged. DO NOT IMMERGE. Thoroughly clean bottles and measuring devices to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at:

https://www.gc.dental/europe or for the Americas: https://www.gc.dental/america They can also be obtained from your supplier. Undesired effects - Reporting If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

UK Responsible Person GC UNITED KINGDOM Ltd. Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 03/2024

**Rx Only** **CE 2797** **UK CA 0086**

**IGCI**

MANUFACTURED & DISTRIBUTED BY GC AMERICA INC. 3737 West 12th Street, Aliso Viejo, CA 92683 U.S.A. TEL: +1 714 959 0200 www.gc.dental/america

DISTRIBUTED BY GC EUROPE N.V. Heideveldpark, Heideveld 1240, Interleuvenlaan 33, 3000 Leuven, Belgium TEL: +32 16 74 10 00 GC ASIA DENTAL PTE. LTD. 5 Tampines Central 1, #06-01 Tampines Plaza 2, Singapore 529541 TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD. 170 Berrigal Rd, Berrigal NSW 2516, Australia TEL: +61 2 9321 8200

GC SOUTH AMERICA Rua Helderborg, 366, São Paulo, SP, BRASIL CEP: 02022-081 - TEL: +55 11 2629 0865 CNPJ: 06.279.999/01-41

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