Prior to use, carefully read the instructions for use.

EN

COMPOSITE PRIMER

PRIMER FOR MICRO-FILLED COMPOSITE AND ACRYLIC RESIN

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Surface treatment of micro-filled resin (MFR) or micro-filled hybrid resin (GRADIA, GRADIA gum shades) prior to luting or repairing procedures.
- Surface treatment of acrylic resin prior to repairing procedure.

CONTRAINDICATIONS

Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

COMPOSITION

Methacrylate, dimethacrylate, initiator, stabilizer

DIRECTIONS FOR USE

- Surface treatment of the restoration prior to luting procedure
 - PREPARATIONPrepare the surface to be bonded to using a sandblaster (aluminum oxide) with a pressure of at least 3 kg/cm² (2,94 bar). Refer to the manufacturer's instructions for use. Clean with oil-free air. Note: Apply masking tape to the surfaces where no treatment is needed.
 - APPLICATIONDispense one or two drops of COMPOSITE PRIMER into a dispensing dish. Apply a thin layer to the surfaces to be bonded to using a brush. Note:1) Use immediately after dispensing. Replace the bottle cap immediately after use.2) Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.
- LIGHT CURINGLight cure using a visible light curing unit (470 nm wavelength). Refer to the light manufacturer's instructions for use. After light curing, do not remove the oxygen inhibited outer layer. Halogen/LED (> 700 mW/cm²) : 20 seconds High power LED (> 1200 mW/cm²) : 10 seconds
- CEMENTATIONApply a suitable luting cement on the treated surfaces of the restoration and continue with the luting procedure. Refer to the manufacturer's instructions for use.

- Surface treatment of the restoration prior to repairing procedure
 - PREPARATIONPrepare the surface to be bonded to using a diamond bur or sandblaster (aluminum oxide) with a pressure of at least 3kg/cm² (2,94 bar). Take the necessary safety measures for the intraoral use of the sandblaster. Refer to the manufacturer's instructions for use. Clean with oil-free air. Note: Apply masking tape to the surfaces where no treatment is needed.
 - APPLICATIONDispense one or two drops of COMPOSITE PRIMER into a dispensing dish. Apply a thin layer to the surface to be repaired using a brush. Note:1) Use immediately after dispensing. Replace the bottle cap immediately after use.2) Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.
- LIGHT CURINGRefer to A.3. above. When using Labolight DUO, LABOLIGHT LV-III, light cure for 1 minute.
- REPAIRApply composite material such as GRADIA, GRADIA DIRECT, GC KALORE or GRADIA gum shades on the treated surfaces of the restoration and start to finish. Refer to the manufacturer's instructions for use.

STORAGE

Recommended for optimal performance, store in a cool, dark place (4-25°C) (39,2-77,0°F).

PACKAGE

Bottle of 3 mL

CAUTION

- COMPOSITE PRIMER is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
- In case of contact with oral tissues, immediately remove with a sponge or cotton pellet. Flush with water after treatment is completed.
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Do not mix with any other primers. Do not reuse a brush with other products.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

or for The Americas

<http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS
United Kingdom

Last revised: 01/2024

GC

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Tomimatsu-cho, Kasugai, Aichi 486-0944, Japan

RESPONSIBLE MANUFACTURER IN CANADA:

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Methacrylat, Dimethacrylat, Initiator, Stabilisator

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