Prior to use, carefully read the instructions for use.

## DENTIN CONDITIONER

POLYACRYLIC ACID CAVITY CONDITIONER

For use only by a dental professional in the recommended indications.

**INDICATIONS FOR USE**  
DENTIN CONDITIONER is a mild polyacrylic acid solution designed to remove the dentinal smear layer and to condition dentin, thus enhancing the bond between glass ionomer / glass hybrid and the dentin.

**CONTRAINDICATIONS**  
Avoid use of this product in patients with known allergies to polyacrylic acid.

- DIRECTIONS FOR USE**
- After tooth preparation, apply DENTIN CONDITIONER to the bonding surfaces for 20 seconds using a cotton pellet or sponge.
  - Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).
  - Apply GC Fuji BOND LC, GC Fuji II LC or other glass ionomer / glass hybrid as a bonding agent, base / liner or restorative material.
  - Close the bottle immediately after use.

**STORAGE**  
Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

**PACKAGE**  
Bottle of 25 g (23.8 mL)

- CAUTION**
- In case of contact with oral tissue or skin, flush immediately with water.
  - In case of contact with eyes, flush immediately with water and seek medical attention.
  - In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
  - Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

They can also be obtained from your supplier.

**CLEANING AND DISINFECTING**  
**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERS. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

**Undesired effects- Reporting:**  
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)  
In this way you will contribute to improve the safety of this product.

UK Responsible Person  
GC UNITED KINGDOM Ltd.  
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

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