

# EQUIA Forte™ HT



1  
Clean the preparation and apply Cavity Conditioner (10s) or Dentin Conditioner (20s).



2  
Rinse and gently dry, do not desiccate.



3  
Shake or tap your GC capsule to loosen powder, then depress plunger.



4  
Mix for 10s at high speed. Check IFU for working time.



5  
Insert on Capsule Applier. Click twice to prime capsule.



6  
Proceed to capsule activation, then dispense immediately after (within 10s).




7  
Pack and contour. Avoid moisture contamination and dry-out. Working time is 1'15mn



8  
Finish under water spray using superfine diamond burs after 2 minutes 30 seconds from start of mixing.



9  
Finish the restoration by applying EQUIA Forte Coat. DO NOT AIR BLOW.

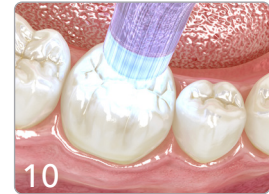

10  
Light-cure for 20s.

