

Have a clear view



EXACLEAR from GC

Clear vinyl polysiloxane material

GRADIA® PLUS' partner
for transparent work

- Excellent control of details
- Easy handling
- Fast setting

The logo for GC, featuring the letters 'G' and 'C' in a stylized, teal, italicized font with a double underline effect.

EXACLEAR and GRADIA PLUS

from GC

EXACLEAR is very useful to create temporary crowns or veneers. Using the GC GRADIA PLUS Light Body rather than acrylic is easier and less technique sensitive. Simply inject the material into the EXACLEAR mould!

You have an optimal visual control due to the high transparency and efficient polymerisation can be obtained by light-curing through the mould. The material is firm but flexible and can be removed and repositioned easily, without tearing.

Space can be kept for coverage with Heavy Body Enamel HB-E. Final painting and glazing with Lustre Paint is like the icing on the cake!



EXACLEAR injected into a vacuum formed template, placed over the wax-up.



Prepped model, coated with a silicone separator over the preps and with GC GRADIA PLUS separator over the rest.



Injecting the Light Body.



Composite is cured through the extremely clear EXACLEAR.



After removal of the silicone and clean-up of the vents.



Temporary restorations finished with Lustre Paint.

Your partners in complex clinical situations



Temporary implant abutments with opaque layer of GC GRADIA PLUS and GC Stick reinforcement.



Injected and cured Light Body.



After clean-up and cut back of the restorations.



Surface modification with Lustre Paint, covered with Heavy Body Enamel HB-E.



Final case finished with Lustre Paint CLF.



Cases: Stephen Lusty, United Kingdom
EXACLEAR, GC Stick and GRADIA PLUS are trademarks of GC



GC EUROPE N.V.
Head Office
Researchpark
Haasrode-Leuven 1240
Interleuvenlaan 33
B-3001 Leuven
Tel. +32.16.74.10.00
Fax. +32.16.40.48.32
info.gce@gc.dental
<http://www.gceurope.com>

GC UNITED KINGDOM Ltd.
Coopers Court
Newport Pagnell
UK-Bucks. MK16 8JS
Tel. +44.1908.218.999
Fax. +44.1908.218.900
info.uk@gc.dental
<http://uk.gceurope.com>