## FREEGENOL TEMPORARY PACK

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**EN**

Prior to use, carefully read the instructions for use.

## FREEGENOL TEMPORARY PACK

EUGENOL FREE TEMPORARY CEMENT

For use only by a dental professional in the recommended indications.

This product is a Type I material according to classification of ISO 3107.

**INDICATIONS FOR USE**  
Cementation of temporary crowns and bridges.

**CONTRAINDICATIONS**  
Avoid use of this product in patients with known allergies to rosin.

**DIRECTIONS FOR USE**  
1. Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad. (The standard Base to Accelerator ratio is 1.0 g / 0.4 g)  
2. Mix with a spatula for about 20-30 seconds until a uniform color is obtained.  
3. Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.  
4. Allow to set (3 minutes from start of mixing) and trim excess.  
Note:  
1. Setting time can be adjusted by using more or less Base (More Base=faster set ; less Base=slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).  
2. If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

**STORAGE**  
Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

**PACKAGES**  
1-1 pkg: 55 g (20 mL) Base, 20 g (20 mL) Accelerator, Cleaner and mixing pad

**CAUTION**  
1. In case of contact with eyes, flush immediately with water and seek medical attention.  
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.  
3. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gceurope.com
http://www.gcamerica.com
They can also be obtained from your supplier.

**CLEANING AND DISINFECTING:**  
**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:  
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en
as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS
United Kingdom

Last revised: 11/2022

**MANUFACTURED BY**  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan  
**RESPONSIBLE MANUFACTURER IN CANADA:**  
**GC AMERICA INC.**  
3737 West 127th Street, Missi, IL 60803 U.S.A.  
**DISTRIBUTED BY**  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan  
**EU : GC EUROPE N.V.**  
Researchpark Hassode-Leuven 1240, Interfeuvestraat 33, B-3001 Leuven, Belgium- TEL: +32 16 74 10 00  
**GC AMERICA INC.**  
3737 West 127th Street, Missi, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
www.gcamerica.com  
**GC SOUTH AMERICA**  
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL  
CEP: 02022-051 - TEL: +55-11-2925-0965  
CNPJ: 08.276.993/0001-61  
RESP: TÊC, Erick de Lima - CRO/SP 100.866  
**GC ASIA DENTAL PTE. LTD.**  
5 Tampines Central 1, #06-01 Tampines Plaza 2, Singapore 529541  
TEL: +65 6546 7588  
**GC AUSTRALASIA DENTAL PTY. LTD.**  
1753 Botany Rd, Bankmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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