30000605-NV-9905
IFU Fuji 1 Capsule EEP

For use only by a dental professional in the indications for use.

GC Fuji™ I CAPSULE

RADIOOPAKE GLASS IONOMER LUTING CEMENT IN CAPSULES

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Placement of metal-based inlays, onlays, crowns, bridges and posts.
- Construction of high strength (zirconia based) all ceramic crowns and bridges.

CONTRAINDICATIONS

- Direct pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement.

COMPOSITION

Powder: Fluoro-alumino-silicate glass, polyacrylic acid, pigment
Liquid: Distilled water, polyacrylic acid, carboxylic acid

DIRECTIONS FOR USE

Powder / Liquid ratio (g/g)	0,330,18
Mixing time (sec.)	10*
Working time (min., sec.) (at 23 °C) (from start of mixing)	2'15"
Net Setting Time (37 °C/99 F)(min., sec.)	2'30"
Finishing time (37 °C/99 F, after seating the restoration)(min., sec.)	4'30"

Glass polyalkenoate cement, Luting, Net Setting time (37°C/99°F) (min., sec.) 2'30"

ISO 9917-1 Glass polyalkenoate cement, Luting

- TOOTH PREPARATION**
 - Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.
 - Clean the prepared teeth with pumice and water.
 - Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. **DO NOT DRY WITH AIR.** Prepare surfaces according to approved moist (glistening).

- RESTORATION PREPARATION**

Make sure that the restoration is pretreated and handled according to the manufacturer's instructions.

- CAPSULE ACTIVATION AND MIXING**
 - Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).
 - To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig. 2).
 - Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
 - The capsule should be activated just before mixing and used immediately.

- Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM) (Fig. 3).

Note:

When using a rotational mixing device, it is recommended not to use the centrifuge option.

- DISPENSING**
 - Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
 - Make two clicks to prime the capsule (Fig. 4). To adjust the direction of the nozzle, hold the applicer with the capsule towards you and turn the capsule body.
 - Insert directly into the restoration.
 - To remove the used capsule, push the applicer release button. Twist the end and pull upwards.

- CEMENTATION**
 - Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 12 minutes 15 seconds from start of mixing at 23° (73.4°F). Higher temperatures will shorten working time.
 - Maintain moderate pressure.
 - Start removing excess cement when the excess cement feels rubbery.
 - Finishing can be started 4 minutes 30 seconds after seating the restoration.

STORAGE
Recommended for optimal performance, store the capsules in the original aluminium foil in a cool and dark place (4-25°C) (39.2- 77.0°F).

SHADE
Light yellow.

PACKAGE
50 capsules
Average contents per capsule: 0.33g powder and 0.18g (0.14mL) liquid. Minimum net volume of mixed cement per capsule: 0.19ml. Option: GC CAPSULE APPLIER (1 piece).

CAUTION
1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.

- In case of contact with eyes, flush immediately with water and seek medical attention.
- This product is not indicated for filling or core build-up.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at <https://www.gc.dental/europe>. They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed/>) or contact us at Regulatory.gcc@gc.dental

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.
Last revised: 06/2024

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8685, Japan

DISTRIBUTED BY:
EU: GC EUROPE NV
Nieuwsteeg 12
1200 Brussels, Belgium
Tel: +32 (0) 20 76 10 10

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