

GC Fuji II LC



1
Clean the preparation and apply Cavity Conditioner (10s) or Dentin Conditioner (20s).



2
Rinse and gently dry, do not desiccate.



3
Shake or tap your GC capsule to loosen powder, then depress plunger.



4
Mix for 10s at high speed. Check IFU for working time.



5
Insert on Capsule Applier. Click twice to prime capsule.



6
Form the contour and place a matrix if required.



7
Light-cure for 20s.



8
Finish under water spray using a superfine diamond bur or a silicone finishing point.



9
Apply GC Fuji Varnish or GC Fuji Coat LC.



10
Light-cure for 10s.

