IFU FUJI IX GP CAPSULE
NW3705_30000635

EN

GC Fuji™ IX GP CAPSULE

RADIOAPAKE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Cone build-up.

CONTRAINDICATIONS

- Direct pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement.

COMPOSITION
Powder: Fluoro-alumino-silicate glass, polyacrylic acid, pigment
Liquid: Water, polyacrylic acid, carboxylic acid

DIRECTIONS FOR USE

Powder / Liquid Ratio(g/g)	0.40 / 0.11
Mixing Time (sec.)	10"
Time of initial extrusion after mixing	10 seconds maximum
Working Time (min., sec.)	2'00"
Net Setting Time (min., sec.)	22'20"
Final Finishing Commencing Time (37°C, after starting mix (min., sec.)	6'00"

Test conditions: Temperature (23 +/-1 °C), Relative humidity (50 +/-5%)

ISO 9917-1: Glass polyalkenoate cement, Restauration

- CAVITY PREPARATION**
Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For OPS capping use calcium hydroxide.
b) Apply GC CAVITY CONDITIONER(10 seconds) or GC DENTIN CONDITIONER(20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).
- c) Rise thoroughly with water. Blot away excess material with a cotton pellet or dry by gently blowing with an air syringe. Do not DESICCATE. Prepared surfaces should appear moist(glistening).
- CAP SQUEEZING AND PLUNGERING**
a) Before activation, shake the capsule or tap it side on a hard surface to loosen the powder(Fig. 2).
b) To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds(Fig. 3).
- c) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
d) The capsule should be activated just before mixing and used immediately.
e) When using it set it into a rotator (an amalgamator) and mix for 10 seconds +/- 4,000 RPM(Fig. 4). Note: When using a rotational mixing device, it is recommended not to use the centrifuge option.

RESTORATIVE TECHNIQUE

- a) Immediately remove the mixed capsule from the mixer and lead it into the GC Capsule Applier.
b) Make two clicks to prime the capsule then syringe (Fig. 5). The working time is 2 minutes from start of mixing at 23°C(73.4°F). Higher temperatures shorten working time.
c) Within 10 seconds of maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 6).
- d) Finish the preliminary contour, and cover with a matrix if required.
e) When set, immediately apply GC Fuji VARNISH (low-dry) or GC Fuji COAT LC(light cure)(Fig. 7). Note: To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
f) To remove the use capsule, push the applicator upwards. Twist the capsule and spin the cap upwards.
- FINISHING**
a) Finishing using under spray using standard techniques can begin approximately 6 minutes after start of mixing(Fig. 8).
b) Apply a final coat of GC Fuji VARNISH(blow-dry) or GC Fuji COAT LC(light cure for 10 sec.) or GC-COAT PLUS(light cure for 30 sec.) (Fig. 9)
- c) Instruct the patient not to apply pressure for 1 hour.

STORAGE

Store in a cool and dark place (4-25°C/39.2-77.0°F).

SHADES

A2, A3, A3.5, B2, B3, C4

Shade numbers according to Vita® shade guide.

Shade is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany

PACKAGES

- Single shade package: 50 capsules.
- Assortment package: 50 capsules (10 each of A2, A3, A3.5, B2, B3).
- Average contents per capsule: 0.40g powder and 0.11g(0.09mL) liquid. Minimum net volume of mixed cement per capsule: 0.14ml.

Option GC CAPSULE APPLIER(1 piece).

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton swab in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

Personal protective equipment(PPE) such as gloves, face masks and safety eyewear should always be worn.

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gsc@gc-dental

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gsc@gc-dental

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar event, please report it to us at this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

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MANUFACTURED BY GC CORPORATION
5-1-1 Hattori, Shiohata-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY GC CORPORATION
Hattori-cho, Shiohata-ku, Tokyo 174-8585, Japan

EU: GC EUROPE NV
Hattori-cho, Shiohata-ku, Tokyo 174-8585, Japan
Tel: +32 (0) 2 747 10 10

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