30000634 - NW3605
Fuji IX GP Capsule WEP

Prior to use, carefully read the instructions for use. **EN**

GC Fuji IX GP CAPSULE

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Corebuild-up.

CONTRAINDICATIONS

- Direct pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to aphysician.

COMPOSITION

Powder: Fluoro-alumino-silicate glass, polyacrylic acid, pigment
Liquid: Water, polyacrylic acid, carboxylic acid

DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0,40 / 0,11
Mixing Time (sec.)*	10"
Time of initial extrusion after mixing	10 seconds maximum
Working Time (min., sec.)*	2'00"
Net Setting Time (min., sec.)*	2'20"
Final Finishing Commencing Time (37°C, after starting mix) (min.,sec.)	6'00"

*Test conditions : Temperature (23 ± 1 °C),
Relative humidity (50 ± 5%)

ISO 9917-1 Glass polyalkenoate cement, Restoration

1. CAVITYPREPARATION

- Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.
 - Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig.1).
 - Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist(glistening).
- CAPSULE ACTIVATION ANDMIXING**
 - Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig.2).
 - To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig.3).
Note:
 - Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder andaniquid.
 - The capsule should be activated just before mixing and usedimmediately.
 - Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (± 4,000 RPM) (Fig. 4).
Note:
When using a rotational mixing device, it is recommended not to use the centrifuge option.
 - RESTORATIVETECHNIQUE**
 - Immediately remove the capsule from the mixer and load it into the GC CapsuleApplier.
 - Make two clicks to prime the capsule then syringe (Fig.4). The working time is 2 minutes from start of mixing at 23°C (73.4°F). Higher temperatures will shorten workingtime.
 - Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig.6).
 - Form the preliminary contour, and cover with a matrix ifrequired.
 - When set, immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) (Fig. 7).
Note:
 - To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsulebody.
 - To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.
 - FINISHING**
 - Final finishing under water spray using standard techniques can begin approximately 6 minutes after start of mixing (Fig.8).
 - Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.for G-COAT PLUS (light cure for 20 sec.) (Fig. 9).
 - Instruct the patient not to apply pressure for 1hour.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°) (39,2-77,0°F).

SHADES

A2, A3, A3.5, B2, B3, C4

Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

- Single shade package : 50 capsules.
- Assortment package : 50 capsules(10 each of A2, A3, A3.5, B2, B3).
Average contents per capsule: 0.40g powder and 0.11g(0.09mL) liquid.
Minimum net volume of mixed cement per capsule: 0.14mL.
Option: GC CAPSULE APPLIER(1 piece).

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton wool.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal protective equipment(PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS.
Always familiarize yourself with the safety data sheets available at: <https://www.gc.dental/europe> or for the Americas <https://www.gc.dental/americas> They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system:

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE NV
Researchpark HasseltLeuven 1240,
Interleuvenlaan 33, B-3001 Leuven, Belgium
TEL.: +32 16 74 10 00

PRINTED IN BELGIUM

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

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