## GC FujiCEM™ 2 Dispenser

For use only by a dental professional in the recommended indications.

**INTENDED USE**  
To dispense material from GC FujiCEM and GC FujiCEM 2 Paste Pak cartridges. (Not intended for use with other GC Paste Pak products.)

### IDENTIFICATION OF PARTS

- (Paste Pak Cartridge)  
1. Kapačka 2. Vnitřní pist 3. Cartridge tips (GC FujiCEM 2 Dispenser)  
4. Body 5. Plunger (Fig. 1)

### DIRECTIONS FOR USE

1. Cartridge loading (Fig. 2)  
Retract the plunger completely into the body of the dispenser. Ensure the mark (▲) on the cartridge lines up with the mark (▲) on the dispenser as illustrated. Turn the cartridge fully to the right position. Ensure that the cartridge is properly attached to the dispenser. After cartridge loading, slide the plunger forward until it stops. Remove the cover.

To remove the cartridge, retract the plunger fully and turn the cartridge back to the marked positions.

2. Dispensing  
Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the pastes in order to prevent the incorporation of air bubbles into the material.

### Handmixing -

Push the center of the plunger to dispense the paste onto a mixing pad (Fig. 3).  
While pushing the plunger, level and cut off the extruded material from the cartridge tips by moving the cartridge and Dispenser to an upright position on the mixing pad. When the mixing is complete, dispensing, residual pastes at the tips are retracted into the cartridge. Replace the cartridge cover.

Note:  
The minimum dispensing amount is indicated in Fig. 4 (real size). The amount less than indicated may yield inappropriate paste / paste ratio.

### Automixing –

- a) Before the cartridge cover.
- b) Before every use, check the two openings of the cartridge tip and confirm the pastes are at the same level to ensure even flow from the cartridge.

Note:  
Before dispensing the first time from a new cartridge or after a long interval in between, bleed the pastes from the cartridge.

- a) Attach the GC FujiCEM 2 Mixing Tip SL on the cartridge. Push the mixing tip completely until the end (Fig. 5-①). Hold the sides of sleeve and slide it forward to lock the mixing tip (Fig. 5-②).
- b) Slowly push the center of the plunger end into the appliance to dispense the material (Fig. 3).

Note:  
1) Make sure the GC FujiCEM 2 Mixing Tip SL is correctly attached to the cartridge. If not, remove the GC FujiCEM 2 Mixing Tip SL and reattach.

- 2) If the material has been refrigerated, leave the cartridge in room temperature for 10 minutes.
- 3) Do not leave the GC FujiCEM 2 Mixing Tip SL on the cartridge for a long period of time.
- 4) Before dispensing the material into the appliances, bleed a small amount of the mixed paste.
- 5) The amount of remaining paste in the cartridge can be confirmed by the position of the inner plunger seen through the cylinders. When only a small amount of paste remains in the cartridge, the paste may not be dispensed from the GC FujiCEM 2 Mixing Tip SL.
- 6) To avoid damage, do not drop the dispenser with the cartridge and mixing tip.

### Cleaning / Sterilization of GC FujiCEM 2 Dispenser

1. Clean the dispenser with water and soap or detergent. The dispenser may be disinfected using alcohol, autoclave or steam autoclave using either of the following conditions.
  - a. 115-118 °C (239-244 °F) for 30 minutes
  - b. 121-124 °C (250-255 °F) for 15 minutes
  - c. 126-132 °C (259-270 °F) for 10 minutes
 Use purified water for autoclave. Do not use dry heat sterilization.
2. Some cleaning agents like glutaraldehyde or phenol based cleaner may cause discoloration of GC FujiCEM 2 Dispenser.
3. Refer to the manufacturer's instructions for use of the cleaning agent or instrument.
4. Autoclave temperature should not exceed 132°C (270°F).
5. After autoclave, let the dispenser cool before use.

### STORAGE

Store away from direct sunlight.

### PACKAGE

1 piece

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:  
<http://www.gc.europa.com>

They can also be obtained from your supplier.

Undesired effects - Reporting

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

As well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)

In this way you will contribute to improve the safety of this product.

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Note:  
Before dispensing the first time from a new cartridge or after a long interval between uses, bleed the pastes from the cartridge.

- a) Attach the GC FujiCEM 2 Mixing Tip SL on the cartridge. Push the mixing tip completely until the end (Fig. 5-①). Hold the sides of sleeve and slide it forward to lock the mixing tip (Fig. 5-②).
  - b) Slowly push the center of the plunger end into the appliance to dispense the material (Fig. 3).
- Note:  
1) Make sure the GC FujiCEM 2 Mixing Tip SL is correctly attached to the cartridge. If not, remove the GC FujiCEM 2 Mixing Tip SL and reattach.

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