Prior to use, carefully read the instructions for use. **EN**

GC Fuji™ COAT LC

LIGHT CURED PROTECTIVE COATING

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

COMPOSITION

Methacrylate, multifunctional monomer, initiator, stabilizer

DIRECTIONS FOR USE

- Place the glass ionomer cement according to the manufacturer's instructions.
- Using a cotton pellet or brush apply GC Fuji COAT LC to all exposed glass ionomer and adjacent surfaces.
- Light cure for 10 seconds using suitable visible light curing device (>500 mW/cm²).
- Close bottle immediately after use.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C) (39.2-77.0°F).

PACKAGE

Bottle of 5 g (5.2 mL)

CAUTION

- GC Fuji COAT LC is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>
or for The Americas

<http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

as well as to our internal vigilance system: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

UK Responsible Person

GC UNITED KINGDOM Ltd.

Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 03/2024

'GC'

Rx Only

MANUFACTURED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

RESPONSIBLE MANUFACTURER IN CANADA:

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