Prior to use, carefully read the instructions for use.

## GC Fuji LINING™ LC

RADIOPAQUE LIGHT CURED GLASS IONOMER LINING CEMENT

For use only by a dental professional in the indications for use.

**INDICATIONS FOR USE**

As a base or liner in prepared cavities.

**CONTRAINDICATIONS**

- Direct pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylate polymer.

**COMPOSITION**

Powder: Fluoro-alumino-silicate glass, initiator, pigment
Liquid: Methacrylate, water, polyacrylic acid, dimethacrylate, carboxylic acid, initiator, stabilizer

**DIRECTIONS FOR USE**

Powder / Liquid Ratio (g/g)	1,4 / 1,0
Mixing Time (sec.)	15"-25"
Working Time (23°C/73°F) (min., sec.)	3'00"
Light Curing Time (sec.)	30"
Depth of Cure (mm)	1,6

ISO 9917-2 Klasse 2, Base or lining
Radio-opaque

**1. POWDER AND LIQUID DISPENSING**

- The standard powder to liquid ratio is 1,4/1,0 g. 1 level scoop of powder to 1 drop of liquid.
  - For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
  - Hold the liquid bottle vertically and squeeze gently.
  - Close bottles immediately after use.

**2. MIXING**
Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into two equal parts. Mix the first portion with all of liquid for 5-10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 10-15 seconds.

**3. RESTORATIVE TECHNIQUE**

- Prepare tooth using standard techniques. Extensive mechanical retention is not necessary.
  - Wash and dry but **DO NOT DESICCATE. CAVITY CONDITIONER** or **DENTIN CONDITIONER** is recommended to remove the smear layer. For pulp capping use calcium hydroxide.
  - Mix the required amount of cement. Working time is 3 minutes from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
  - Remove surface moisture, but **DO NOT DESICCATE.**
  - Transfer cement to the preparation using a syringe or suitable placement instrument, covering dentine up to the dentino-enamel junction.
  - Light cure for 30 seconds using a visible light curing device (LED / Halogen >700 mW/cm²).
- Note:
  - If application exceeds 1,6 mm in depth, use a layering technique.
  - Proceed with standard composite or amalgam restorative techniques.

**STORAGE**

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

**PACKAGES**

- Bottle of 10 g powder with scoop.
- Bottle of 8 g (6,8 mL) liquid.
- 1-1 pkg. : 10 g powder, 8 g (6,8 mL) liquid, powder scoop, plastic spatula, mixing pads.

**CAUTION**

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT** mix powder or liquid with any other glass ionomer components.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
https://www.gc.dental/europe or The Americas
https://www.gc.dental/america
They can also be obtained from your supplier.

**CLEANING AND DISINFECTING:**

**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
**DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper author of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en
as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person

GC UNITED KINGDOM LTD.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 11/2024

**GC** **Only**

MANUFACTURED BY  
**GC CORPORATION**  
76-1 Hisatsuna-cho, Itabashi-ku, Tokyo 174-8585, Japan  
RESPONSIBLE MANUFACTURER IN CANADA:  
**GC AMERICA INC.**  
3737 West 127th Street, Aliso, IL, 60803 U.S.A.  
TEL: +1 708-597-0900

DISTRIBUITO DA  
**GC CORPORATION**  
76-1 Hisatsuna-cho, Itabashi-ku, Tokyo 174-8585, Japan  
**EU** : **GC EUROPE N.V.**  
Researchpark, Hazeleerde-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium. TEL: +32 16 74 10 90

**GC AMERICA INC.**  
3737 West 127th Street, Aliso, IL, 60803 U.S.A.  
TEL: +1 708-597-0900  
www.gc.dental/america

**GC SOUTH AMERICA**  
Rua Heitor de Sá, 399, Santana - São Paulo, SP, BRASIL  
CEP: 02022-051 - TEL: +55-11-2925-0965  
CNPJ: 08.278.999/0001-61  
RESP: YEC- Erick de Lima -CNPJ-590.806

**GC ASIA DENTAL PTE. LTD.**  
5 Tampines Central 1, #06-01 Tampines Plaza 2, Singapore 529541  
TEL: +65 6546 7588

**GC AUSTRALASIA DENTAL PTY. LTD.**  
1753 Botany Rd, Bankmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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