

Prior to use, carefully read  
the instructions for use.

EN

**GC Fuji ORTHO™ BAND  
PASTE PAK**

GLASS IONOMER ORTHODONTIC BAND CEMENT

For use only by a dental professional in the recommended indications.

G.C. FUJI ORTHO BAND PASTE PAK is a glass ionomer orthodontic band cement available in paste form delivery.

G.C. FUJI ORTHO BAND PASTE PAK is available in a cartridge pack called Paste Pak Dispenser.

INDICATIONS FOR USE

1. Cementation of orthodontic bands.

2. Cementation of appliances retained with stainless steel crowns (e.g., Herbst or expansion appliances).

CONTRAINDICATIONS

No known contraindications in patients with known allergies to glass ionomer cement, methacrylate monomer or methylmethacrylate polymer.

**IDENTIFICATION OF PARTS**The following parts are included with this product:  
1. Cover 2. Inner plunger 3. Cartridge tip  
4. Piston 5. Adjusting guide 6. Lever 7. Sliding block  
8. Piston release (Fig. 1)**DIRECTIONS FOR USE**1. Place the Paste Pak Dispenser on a flat surface.  
2. Push the piston forward until it is fully seated in the Paste Pak Dispenser (Fig. 2).

3. Remove the piston release (Fig. 3).

4. Push the piston forward until it is fully seated in the Paste Pak Dispenser (Fig. 4).

5. Remove the cover and piston release (Fig. 5).

6. Press the lever to dispense the required amounts of pastes (Fig. 6).

7. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

8. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

9. Then gently push the piston fully to the right position.

Note: To change the cartridge, repeat the process.

10. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

11. Wear your occlusal splint or Herbst, if applicable.

12. Remove the cartridge (Fig. 5).

13. Remove the piston release (Fig. 6).

14. Replace the lever to disperse the required amounts of pastes (Fig. 7).

15. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

16. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

17. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

18. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

19. Wear your occlusal splint or Herbst, if applicable.

20. Remove the piston release (Fig. 6).

21. Replace the lever to disperse the required amounts of pastes (Fig. 7).

22. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

23. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

24. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

25. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

26. Wear your occlusal splint or Herbst, if applicable.

27. Remove the piston release (Fig. 6).

28. Replace the lever to disperse the required amounts of pastes (Fig. 7).

29. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

30. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

31. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

32. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

33. Wear your occlusal splint or Herbst, if applicable.

34. Remove the piston release (Fig. 6).

35. Replace the lever to disperse the required amounts of pastes (Fig. 7).

36. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

37. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

38. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

39. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

40. Wear your occlusal splint or Herbst, if applicable.

41. Remove the piston release (Fig. 6).

42. Replace the lever to disperse the required amounts of pastes (Fig. 7).

43. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

44. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

45. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

46. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

47. Wear your occlusal splint or Herbst, if applicable.

48. Remove the piston release (Fig. 6).

49. Replace the lever to disperse the required amounts of pastes (Fig. 7).

50. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

51. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

52. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

53. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

54. Wear your occlusal splint or Herbst, if applicable.

55. Remove the piston release (Fig. 6).

56. Replace the lever to disperse the required amounts of pastes (Fig. 7).

57. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

58. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

59. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

60. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

61. Wear your occlusal splint or Herbst, if applicable.

62. Remove the piston release (Fig. 6).

63. Replace the lever to disperse the required amounts of pastes (Fig. 7).

64. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

65. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

66. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

67. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

68. Wear your occlusal splint or Herbst, if applicable.

69. Remove the piston release (Fig. 6).

70. Replace the lever to disperse the required amounts of pastes (Fig. 7).

71. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

72. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

73. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

74. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

75. Wear your occlusal splint or Herbst, if applicable.

76. Remove the piston release (Fig. 6).

77. Replace the lever to disperse the required amounts of pastes (Fig. 7).

78. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

79. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

80. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

81. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

82. Wear your occlusal splint or Herbst, if applicable.

83. Remove the piston release (Fig. 6).

84. Replace the lever to disperse the required amounts of pastes (Fig. 7).

85. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

86. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

87. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

88. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

89. Wear your occlusal splint or Herbst, if applicable.

90. Remove the piston release (Fig. 6).

91. Replace the lever to disperse the required amounts of pastes (Fig. 7).

92. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

93. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

94. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

95. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

96. Wear your occlusal splint or Herbst, if applicable.

97. Remove the piston release (Fig. 6).

98. Replace the lever to disperse the required amounts of pastes (Fig. 7).

99. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

100. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

101. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

102. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

103. Wear your occlusal splint or Herbst, if applicable.

104. Remove the piston release (Fig. 6).

105. Replace the lever to disperse the required amounts of pastes (Fig. 7).

106. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

107. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

108. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

109. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

110. Wear your occlusal splint or Herbst, if applicable.

111. Remove the piston release (Fig. 6).

112. Replace the lever to disperse the required amounts of pastes (Fig. 7).

113. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

114. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

115. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

116. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

117. Wear your occlusal splint or Herbst, if applicable.

118. Remove the piston release (Fig. 6).

119. Replace the lever to disperse the required amounts of pastes (Fig. 7).

120. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

121. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

122. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

123. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

124. Wear your occlusal splint or Herbst, if applicable.

125. Remove the piston release (Fig. 6).

126. Replace the lever to disperse the required amounts of pastes (Fig. 7).

127. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them apart (Fig. 7).

128. Ensure the cartridge index onto the Paste Pak Dispenser (Fig. 3).

129. Then gently push the piston fully to the right position.

Note: To change the piston, repeat the process.

130. After changing the piston, load the Paste Pak Dispenser (Fig. 1).

