IDENTIFICATION OF PARTS
BEZEICHNUNG DER TEILE
IDENTIFICATION DES
PARTIES
COMPONENTI
IDENTIFICACION DE
LAS PARTES
BENOEIING VAN DE
ONDERDELEN
BESKRIVELSE

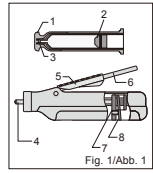
Fig. 1/Abb. 1

DIRECTIONS FOR USE
GEBRAUCHSANWEISUNG
MODE D'EMPLOI
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
GEBRUIKSANWIJZING
BRUGSANVISNING

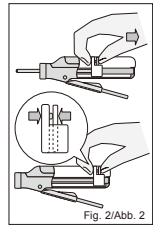
Fig. 2/Abb. 2

Marks
Markierungen
Markieren
Segni
Marcas
Markeringen

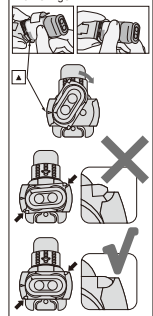
Fig. 3/Abb. 3

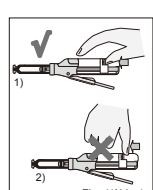
Fig. 4/Abb. 4

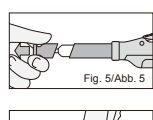
Fig. 5/Abb. 5

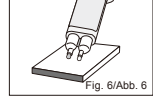
Fig. 6/Abb. 6

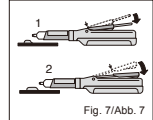
Fig. 7/Abb. 7

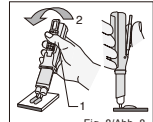
Fig. 8/Abb. 8

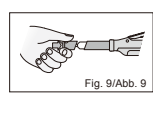
Fig. 9/Abb. 9

Paste Pak Dispenser

For use only by a dental professional in the recommended indications.

Paste Pak Delivery System consists of Paste Pak Dispenser and the special Paste Pak cartridge. Paste Pak Dispenser enables you to dispense material easily and correctly from the cartridge.

INDICATIONS FOR USE

Dispensing material from GC special Paste Pak cartridges.

- IDENTIFICATION OF PARTS**
(Paste Pak Cartridge)
- Cover 2. Inner plunger 3. Cartridge tip (Paste Pak Dispenser)
 - Piston 5. Adjusting guide 6. Lever 7. Sliding block 8. Piston release (Fig. 1)

- Before use**
- Remove the fixing tape placed between the main part and lever of the dispenser.
 - Remove the V shaped board from the underside of the dispenser.

DIRECTIONS FOR USE

- Make sure that the piston is completely retracted into the dispenser. (Fig. 2)
Note:
To retract the piston, push the piston release and at the same time pull the sliding block (on the underside of the dispenser) with fingers.
- Paste Pak Cartridge Loading (Fig. 3)
 - Ensure the ▲ mark on the cartridge lines up with the ▲ mark on the Paste Pak Dispenser as illustrated.
 - Then turn the cartridge fully to the right position.
Note:
To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.
- After cartridge loading, push the piston release forward until it stops. (Fig. 4)
 - Push the piston release forward.
 - Do not push the upper side.
- Remove the cartridge cover. (Fig. 5)
 - Depress the lever to dispense the required amounts of paste onto the mixing pad.
Note:
1) Before dispensing the first time from a new cartridge or after a long interval in between use, bleed the pastes from the cartridge. (Fig. 6)
 - The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed. (Fig. 7)
 - Position of adjusting guide for one inlay.
 - Position of adjusting guide for one crown.
 - While depressing the lever (Fig. 8-1), level and cut off the extruded material from the cartridge tips by moving the cartridge and dispenser to an upright position on the mixing pad (Fig. 8-2).
When the lever is released after dispensing, residual pastes at the cartridge tips are retracted into the cartridge. Clean the cartridge tip and replace the cartridge cover. (Fig. 8)
Note:
1) The amount of residual material in the cartridge can be confirmed by the position of the sliding block on the underside of the dispenser. If the cartridge cylinders are translucent, the residual amount can also be confirmed by the position of the inner plunger seen through the cylinders.
 - Do not drop the dispenser with the cartridge. Otherwise the cartridge may be damaged.

Cleaning / Sterilization of Paste Pak Dispenser

- Clean the dispenser and adjusting guide with gauze soaked in alcohol.
- The dispenser may be steam autoclaved at 121-135°C (250-275°F) for 10 to 30 minutes or disinfected using alcohol. (Please remove the adjusting guide and the cartridge before autoclaving the dispenser). The use of various chemical disinfectants / sterilants may cause damage to the dispenser and therefore is not recommended.
Note:
After autoclaving, the piston release of the dispenser may not easily be moved forwards and backwards by fingers. In such a case, slide the piston release forward until it stops. Apply a very small quantity of petroleum jelly onto the entire surface of the piston.
Do not apply excess petroleum jelly. Then move the piston release back and forth a few times to ensure smooth movement.

PACKAGE

1 piece

CAUTION

- In rare cases metal may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product (nickel-plated aluminum) and refer to a physician.
- Store Paste Pak Dispenser away from chemicals such as iodine tincture containing halogen element. Otherwise the dispenser may discolor.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<http://www.gceurope.com>
or for The Americas
<http://www.gcamerica.com>
They can also be obtained from your supplier.

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM LTD.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS United Kingdom

Last revised: 08/2022

Prior to use, carefully read the instructions for use. **EN**

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