

GC Fuji TRIAGE™



1
Clean the preparation and apply Cavity Conditioner (10s) or Dentin Conditioner (20s).



2
Rinse and gently dry, do not desiccate.



3
Shake or tap your GC capsule to loosen powder, then depress plunger.



4
Mix for 10s at high speed. Check IFU for working time.



5
Insert on Capsule Applicator. Click twice to prime capsule.



6
Proceed to capsule activation, then dispense immediately after (within 10s).



7
Use a brush to directly spread a thin film of GC Fuji Triage.



8
Thermo-cure for 20-40s (only for PINK shade).



9
Finish under water spray using superfine diamond bur or silicone finishing point.



10
Apply GC Fuji Varnish or GC Fuji Coat LC and light-cure for 10s.

