Prior to use, carefully read the instructions for use.

## GC Fuji VARNISH

PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

- Place the glass ionomer cement according to the manufacturer’s instructions.
  - Using a cotton pellet or brush apply GC Fuji VARNISH to all exposed glass ionomer and adjacent surfaces.
  - Dry by gently blowing with air syringe.
  - Maintain moisture isolation for 2-3 minutes.
- Note :  
GC Fuji VARNISH is volatile. Close bottle immediately after use.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).  
(Shelf life: 3 years from date of manufacture)

### PACKAGE

Bottle of 10g (10.4mL)

### CAUTION

- GC Fuji VARNISH is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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