GC Fuji™ VARNISH

PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to acetate esters or acetone.

DIRECTIONS FOR USE

- Place the glass ionomer cement according to the manufacturer's instructions.
- Using a cotton pellet or brush apply GC Fuji VARNISH to all exposed glass ionomer and adjacent surfaces.
- Dry by gently blowing with air syringe.
- Maintain moisture isolation for 2-3 minutes.
Note:
GC Fuji VARNISH is volatile. Close bottle immediately after use.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

PACKAGE

Bottle of 10 g (10.4 mL)

CAUTION

- GC Fuji VARNISH is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas <http://www.gcamerica.com>
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM LTD.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 09/2022

GC

MANUFACTURED BY
GC ASAHİ CORP.
1-3-3 Kachigawa-cho, Katsugai, Aichi 486-0945, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interlieuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1 708-597-0900
www.gcamerica.com
GC SOUTH AMERICA
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TÉCN: Erick de Lima - CRO/SP 100.866

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