G-ænial A’CHORD™ SHADE GUIDE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS
Used for shade selection.

HOW TO USE

Making the personalized shade tabs

This shade guide is comprised of shade tab moulds and their holder, allowing for personalized shade tab creation. The following steps describe how to create your own shade guide.

- Take one shade tab mould from the holder and set it aside
- Take the desired G-aenial A'CHORD shade with an instrument from the syringe or the unitip
- Place the composite in the shade tab mould
- Start to model the composite inside the mould
- Place it against a glass plate to remove excess.
- Light cure through the glass plate for less inhibition layer and faster polishing
- Polish if needed or use a glazing agent such as Optiglaze color in Clear HV or G-Coat Plus to finish in a fast way
- If desired, shade stickers can be placed next to the shade tab to remember the shade you have selected.

Please refer to the technique card for visual guidance.

Making the shade selection

Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Keep the teeth hydrated. Select the appropriate G-ænial A'CHORD shades by referring to the G-ænial A'CHORD shade guide.

Each shade tab has a gradient of thickness starting from 0.8mm to 2.5mm. The middle of the shade tab consists of 1.5mm in thickness.

PACKAGE
1 piece

NOTE

The shade guide can be cleaned with water and mild soap. It should not be sterilized. The use of various chemical disinfectants / sterilants may cause damage to the shade guide.

Last revised: 05/2021

Prior to use, carefully read the instructions for use.

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