2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Take care to avoid ingesting the material.

4. Do not mix with other similar products.

5. Dispensing tips are for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilized in an autoclave or chemically.

6. Wear dental rubber gloves and wear an eye protection to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.

7. Wear a pair of eye glasses during light curing.

8. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.

9. Avoid getting material on clothing.

10. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument.

11. Do not use G-enial Universal Flo in combination with eugenol containing materials as eugenol may cause discoloration of the Universal Flo from setting.

12. Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the material.

13. Dispose of all wastes according to local regulations.

14. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to its MSDS. Always familiarize yourself with the safety data sheets available at: <https://www.gc-dental.eu/en>

Their can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SCP) please see EUAMED database (<https://ec.europa.eu/tools/euamed/>) or contact us at Regulatory@gc-gd.com

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar event, please report by use of this product, including those not listed in this instruction for use, please report them directly through the relevant national regulatory authority or through the authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

as well as to our internal vigilance system: vgilance@gc-dental.com

In this way you will contribute to improve the safety of this product.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS, to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use, please use and label for deterioration. Discard immediately if damaged.

DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants.

Disinfect with a mid-level disinfectant. Do not use disinfection product according to regional/national guidelines.

UK Responsible Person

GC UNITED KINGDOM LTD.

Attn: Regulatory, Court Newsgate, Biddingsham, MK16 8JS United Kingdom

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