



For use only, carefully read the instructions for use.

G-aenial™ Universal Injectable

UNIVERSAL LIGHT-CURE RADIOOPAKE INJECTABLE COMPOSITE
FOR USE WITH A DENTAL PROFESSIONAL IN THE INDICATIONS FOR USE.

INDICATIONS FOR USE:
1. Direct restorative for Class I, II, III, IV and V cavities
2. Fissure sealant
3. Sealing of indirect aesthetic restorations
4. Repair of hypoplastic restorations, temporary crown & bridge, defective crown or bridge
5. Bleaching of undercuts
6.liner of class

7. Restoration of crown & bridges, inlays and veneers using the indirect technique in combination with GRADIA or GRADIA PLUS components (please refer to their dedicated I.D.U.)
8. Sealing of crown & bridge margins with GRADIA PLUS or GRADIA PLUS components (please refer to their dedicated I.D.U.)
9. Follow manufacturer's instructions.

CONTRAINDICATIONS
1. Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.
2. Do not use for pulp capping.

PRODUCT DESCRIPTION
G-aenial Universal Injectable is a light-cure, radio-opaque restorative material to be used intra-orally and classified as a Type I and Class 2 (Group 1) per ISO standard 4049.
This material has a radiopacity equivalent to 2.5-3.0 mm of aluminum (density = 1 mm, enamel = 2 mm).
The particle size of inorganic fill is approximately 0.1-0.15 µm. The total amount of inorganic fill is approximately 46 vol%.

COMPOSITION
Barium glass, dimethacrylate, initiator, pigment, silicon dioxide, stabilizer

DIRECTIONS FOR USE
1. Shade Selection
Select shade from 16 shades of XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, A03, A03.5, A04, A04.5, A05, A05.5, A06, A06.5, A07, A07.5, A08, A08.5, A09, A09.5, A10, A10.5, A11, A11.5, A12, A12.5, A13, A13.5, A14, A14.5, A15, A15.5, A16, A16.5, A17, A17.5, A18, A18.5, A19, A19.5, A20, A20.5, A21, A21.5, A22, A22.5, A23, A23.5, A24, A24.5, A25, A25.5, A26, A26.5, A27, A27.5, A28, A28.5, A29, A29.5, A30, A30.5, A31, A31.5, A32, A32.5, A33, A33.5, A34, A34.5, A35, A35.5, A36, A36.5, A37, A37.5, A38, A38.5, A39, A39.5, A40, A40.5, A41, A41.5, A42, A42.5, A43, A43.5, A44, A44.5, A45, A45.5, A46, A46.5, A47, A47.5, A48, A48.5, A49, A49.5, A50, A50.5, A51, A51.5, A52, A52.5, A53, A53.5, A54, A54.5, A55, A55.5, A56, A56.5, A57, A57.5, A58, A58.5, A59, A59.5, A60, A60.5, A61, A61.5, A62, A62.5, A63, A63.5, A64, A64.5, A65, A65.5, A66, A66.5, A67, A67.5, A68, A68.5, A69, A69.5, A70, A70.5, A71, A71.5, A72, A72.5, A73, A73.5, A74, A74.5, A75, A75.5, A76, A76.5, A77, A77.5, A78, A78.5, A79, A79.5, A80, A80.5, A81, A81.5, A82, A82.5, A83, A83.5, A84, A84.5, A85, A85.5, A86, A86.5, A87, A87.5, A88, A88.5, A89, A89.5, A90, A90.5, A91, A91.5, A92, A92.5, A93, A93.5, A94, A94.5, A95, A95.5, A96, A96.5, A97, A97.5, A98, A98.5, A99, A99.5, A100, A100.5, A101, A101.5, A102, A102.5, A103, A103.5, A104, A104.5, A105, A105.5, A106, A106.5, A107, A107.5, A108, A108.5, A109, A109.5, A110, A110.5, A111, A111.5, A112, A112.5, A113, A113.5, A114, A114.5, A115, A115.5, A116, A116.5, A117, A117.5, A118, A118.5, A119, A119.5, A120, A120.5, A121, A121.5, A122, A122.5, A123, A123.5, A124, A124.5, A125, A125.5, A126, A126.5, A127, A127.5, A128, A128.5, A129, A129.5, A130, A130.5, A131, A131.5, A132, A132.5, A133, A133.5, A134, A134.5, A135, A135.5, A136, A136.5, A137, A137.5, A138, A138.5, A139, A139.5, A140, A140.5, A141, A141.5, A142, A142.5, A143, A143.5, A144, A144.5, A145, A145.5, A146, A146.5, A147, A147.5, A148, A148.5, A149, A149.5, A150, A150.5, A151, A151.5, A152, A152.5, A153, A153.5, A154, A154.5, A155, A155.5, A156, A156.5, A157, A157.5, A158, A158.5, A159, A159.5, A160, A160.5, A161, A161.5, A162, A162.5, A163, A163.5, A164, A164.5, A165, A165.5, A166, A166.5, A167, A167.5, A168, A168.5, A169, A169.5, A170, A170.5, A171, A171.5, A172, A172.5, A173, A173.5, A174, A174.5, A175, A175.5, A176, A176.5, A177, A177.5, A178, A178.5, A179, A179.5, A180, A180.5, A181, A181.5, A182, A182.5, A183, A183.5, A184, A184.5, A185, A185.5, A186, A186.5, A187, A187.5, A188, A188.5, A189, A189.5, A190, A190.5, A191, A191.5, A192, A192.5, A193, A193.5, A194, A194.5, A195, A195.5, A196, A196.5, A197, A197.5, A198, A198.5, A199, A199.5, A200, A200.5, A201, A201.5, A202, A202.5, A203, A203.5, A204, A204.5, A205, A205.5, A206, A206.5, A207, A207.5, A208, A208.5, A209, A209.5, A210, A210.5, A211, A211.5, A212, A212.5, A213, A213.5, A214, A214.5, A215, A215.5, A216, A216.5, A217, A217.5, A218, A218.5, A219, A219.5, A220, A220.5, A221, A221.5, A222, A222.5, A223, A223.5, A224, A224.5, A225, A225.5, A226, A226.5, A227, A227.5, A228, A228.5, A229, A229.5, A230, A230.5, A231, A231.5, A232, A232.5, A233, A233.5, A234, A234.5, A235, A235.5, A236, A236.5, A237, A237.5, A238, A238.5, A239, A239.5, A240, A240.5, A241, A241.5, A242, A242.5, A243, A243.5, A244, A244.5, A245, A245.5, A246, A246.5, A247, A247.5, A248, A248.5, A249, A249.5, A250, A250.5, A251, A251.5, A252, A252.5, A253, A253.5, A254, A254.5, A255, A255.5, A256, A256.5, A257, A257.5, A258, A258.5, A259, A259.5, A260, A260.5, A261, A261.5, A262, A262.5, A263, A263.5, A264, A264.5, A265, A265.5, A266, A266.5, A267, A267.5, A268, A268.5, A269, A269.5, A270, A270.5, A271, A271.5, A272, A272.5, A273, A273.5, A274, A274.5, A275, A275.5, A276, A276.5, A277, A277.5, A278, A278.5, A279, A279.5, A280, A280.5, A281, A281.5, A282, A282.5, A283, A283.5, A284, A284.5, A285, A285.5, A286, A286.5, A287, A287.5, A288, A288.5, A289, A289.5, A290, A290.5, A291, A291.5, A292, A292.5, A293, A293.5, A294, A294.5, A295, A295.5, A296, A296.5, A297, A297.5, A298, A298.5, A299, A299.5, A300, A300.5, A301, A301.5, A302, A302.5, A303, A303.5, A304, A304.5, A305, A305.5, A306, A306.5, A307, A307.5, A308, A308.5, A309, A309.5, A310, A310.5, A311, A311.5, A312, A312.5, A313, A313.5, A314, A314.5, A315, A315.5, A316, A316.5, A317, A317.5, A318, A318.5, A319, A319.5, A320, A320.5, A321, A321.5, A322, A322.5, A323, A323.5, A324, A324.5, A325, A325.5, A326, A326.5, A327, A327.5, A328, A328.5, A329, A329.5, A330, A330.5, A331, A331.5, A332, A332.5, A333, A333.5, A334, A334.5, A335, A335.5, A336, A336.5, A337, A337.5, A338, A338.5, A339, A339.5, A340, A340.5, A341, A341.5, A342, A342.5, A343, A343.5, A344, A344.5, A345, A345.5, A346, A346.5, A347, A347.5, A348, A348.5, A349, A349.5, A350, A350.5, A351, A351.5, A352, A352.5, A353, A353.5, A354, A354.5, A355, A355.5, A356, A356.5, A357, A357.5, A358, A358.5, A359, A359.5, A360, A360.5, A361, A361.5, A362, A362.5, A363, A363.5, A364, A364.5, A365, A365.5, A366, A366.5, A367, A367.5, A368, A368.5, A369, A369.5, A370, A370.5, A371, A371.5, A372, A372.5, A373, A373.5, A374, A374.5, A375, A375.5, A376, A376.5, A377, A377.5, A378, A378.5, A379, A379.5, A380, A380.5, A381, A381.5, A382, A382.5, A383, A383.5, A384, A384.5, A385, A385.5, A386, A386.5, A387, A387.5, A388, A388.5, A389, A389.5, A390, A390.5, A391, A391.5, A392, A392.5, A393, A393.5, A394, A394.5, A395, A395.5, A396, A396.5, A397, A397.5, A398, A398.5, A399, A399.5, A400, A400.5, A401, A401.5, A402, A402.5, A403, A403.5, A404, A404.5, A405, A405.5, A406, A406.5, A407, A407.5, A408, A408.5, A409, A409.5, A410, A410.5, A411, A411.5, A412, A412.5, A413, A413.5, A414, A414.5, A415, A415.5, A416, A416.5, A417, A417.5, A418, A418.5, A419, A419.5, A420, A420.5, A421, A421.5, A422, A422.5, A423, A423.5, A424, A424.5, A425, A425.5, A426, A426.5, A427, A427.5, A428, A428.5, A429, A429.5, A430, A430.5, A431, A431.5, A432, A432.5, A433, A433.5, A434, A434.5, A435, A435.5, A436, A436.5, A437, A437.5, A438, A438.5, A439, A439.5, A440, A440.5, A441, A441.5, A442, A442.5, A443, A443.5, A444, A444.5, A445, A445.5, A446, A446.5, A447, A447.5, A448, A448.5, A449, A449.5, A450, A450.5, A451, A451.5, A452, A452.5, A453, A453.5, A454, A454.5, A455, A455.5, A456, A456.5, A457, A457.5, A458, A458.5, A459, A459.5, A460, A460.5, A461, A461.5, A462, A462.5, A463, A463.5, A464, A464.5, A465, A465.5, A466, A466.5, A467, A467.5, A468, A468.5, A469, A469.5, A470, A470.5, A471, A471.5, A472, A472.5, A473, A473.5, A474, A474.5, A475, A475.5, A476, A476.5, A477, A477.5, A478, A478.5, A479, A479.5, A480, A480.5, A481, A481.5, A482, A482.5, A483, A483.5, A484, A484.5, A485, A485.5, A486, A486.5, A487, A487.5, A488, A488.5, A489, A489.5, A490, A490.5, A491, A491.5, A492, A492.5, A493, A493.5, A494, A494.5, A495, A495.5, A496, A496.5, A497, A497.5, A498, A498.5, A499, A499.5, A500, A500.5, A501, A501.5, A502, A502.5, A503, A503.5, A504, A504.5, A505, A505.5, A506, A506.5, A507, A507.5, A508, A508.5, A509, A509.5, A510, A510.5, A511, A511.5, A512, A512.5, A513, A513.5, A514, A514.5, A515, A515.5, A516, A516.5, A517, A517.5, A518, A518.5, A519, A519.5, A520, A520.5, A521, A521.5, A522, A522.5, A523, A523.5, A524, A524.5, A525, A525.5, A526, A526.5, A527, A527.5, A528, A528.5, A529, A529.5, A530, A530.5, A531, A531.5, A532, A532.5, A533, A533.5, A534, A534.5, A535, A535.5, A536, A536.5, A537, A537.5, A538, A538.5, A539, A539.5, A540, A540.5, A541, A541.5, A542, A542.5, A543, A543.5, A544, A544.5, A545, A545.5, A546, A546.5, A547, A547.5, A548, A548.5, A549, A549.5, A550, A550.5, A551, A551.5, A552, A552.5, A553, A553.5, A554, A554.5, A555, A555.5, A556, A556.5, A557, A557.5, A558, A558.5, A559, A559.5, A560, A560.5, A561, A561.5, A562, A562.5, A563, A563.5, A564, A564.5, A565, A565.5, A566, A566.5, A567, A567.5, A568, A568.5, A569, A569.5, A570, A570.5, A571, A571.5, A572, A572.5, A573, A573.5, A574, A574.5, A575, A575.5, A576, A576.5, A577, A577.5, A578, A578.5, A579, A579.5, A580, A580.5, A581, A581.5, A582, A582.5, A583, A583.5, A584, A584.5, A585, A585.5, A586, A586.5, A587, A587.5, A588, A588.5, A589, A589.5, A590, A590.5, A591, A591.5, A592, A592.5, A593, A593.5, A594, A594.5, A595, A595.5, A596, A596.5, A597, A597.5, A598, A598.5, A599, A599.5, A600, A600.5, A601, A601.5, A602, A602.5, A603, A603.5, A604, A604.5, A605, A605.5, A606, A606.5, A607, A607.5, A608, A608.5, A609, A609.5, A610, A610.5, A611, A611.5, A612, A612.5, A613, A613.5, A614, A614.5, A615, A615.5, A616, A616.5, A617, A617.5, A618, A618.5, A619, A619.5, A620, A620.5, A621, A621.5, A622, A622.5, A623, A623.5, A624, A624.5, A625, A625.5, A626, A626.5, A627, A627.5, A628, A628.5, A629, A629.5, A630, A630.5, A631, A631.5, A632, A632.5, A633, A633.5, A634, A634.5, A635, A635.5, A636, A636.5, A637, A637.5, A638, A638.5, A639, A639.5, A640, A640.5, A641, A641.5, A642, A642.5, A643, A643.5, A644, A644.5, A645, A645.5, A646, A646.5, A647, A647.5, A648, A648.5, A649, A649.5, A650, A650.5, A651, A651.5, A652, A652.5, A653, A653.5, A654, A654.5, A655, A655.5, A656, A656.5, A657, A657.5, A658, A658.5, A659, A659.5, A660, A660.5, A661, A661.5, A662, A662.5, A663, A663.5, A664, A664.5, A665, A665.5, A666, A666.5, A667, A667.5, A668, A668.5, A669, A669.5, A670, A670.5, A671, A671.5, A672, A672.5, A673, A673.5, A674, A674.5, A675, A675.5, A676, A676.5, A677, A677.5, A678, A678.5, A679, A679.5, A680, A680.5, A681, A681.5, A682, A682.5, A683, A683.5, A684, A684.5, A685, A685.5, A686, A686.5, A687, A687.5, A688, A688.5, A689, A689.5, A690, A690.5, A691, A691.5, A692, A692.5, A693, A693.5, A694, A694.5, A695, A695.5, A696, A696.5, A697, A697.5, A698, A698.5, A699, A699.5, A700, A700.5, A701, A701.5, A702, A702.5, A703, A703.5, A704, A704.5, A705, A705.5, A706, A706.5, A707, A707.5, A708, A708.5, A709, A709.5, A710, A710.5, A711, A711.5, A712, A712.5, A713, A713.5, A714, A714.5, A715, A715.5, A716, A716.5, A717, A717.5, A718, A718.5, A719, A719.5, A720, A720.5, A721, A721.5, A722, A722.5, A723, A723.5, A724, A724.5, A725, A725.5, A726, A726.5, A727, A727.5, A728, A728.5, A729, A729.5, A730, A730.5, A731, A731.5, A732, A732.5, A733, A733.5, A734, A734.5, A735, A735.5, A736, A736.5, A737, A737.5, A738, A738.5, A739, A739.5, A740, A740.5, A741, A741.5, A742, A742.5, A743, A743.5, A744, A744.5, A745, A745.5, A746, A746.5, A747, A747.5, A748, A748.5, A749, A749.5, A750, A750.5, A751, A751.5, A752, A752.5, A753, A753.5, A754, A754.5, A755, A755.5, A756, A756.5, A757, A757.5, A758, A758.5, A759, A759.5, A760, A760.5, A761, A761.5, A762, A762.5, A763, A763.5, A764, A764.5, A765, A765.5, A766, A766.5, A767, A767.5, A768, A768.5, A769, A769.5, A770, A770.5, A771, A771.5, A772, A772.5, A773, A773.5, A774, A774.5, A775, A775.5, A776, A776.5, A777, A777.5, A778, A778.5, A779, A779.5, A780, A780.5, A781, A781.5, A782, A782.5, A783, A783.5, A784, A784.5, A785, A785.5, A786, A786.5, A787, A787.5, A788, A788.5, A789, A789.5, A790, A790.5, A791, A791.5, A792, A792.5, A793, A793.5, A794, A794.5, A795, A795.5, A796, A796.5, A797, A797.5, A798, A798.5, A799, A799.5, A800, A800.5, A801, A801.5, A802, A802.5, A803, A803.5, A804, A804.5, A805, A805.5, A806, A806.5, A807, A807.5, A808, A808.5, A809, A809.5, A810, A810.5, A811, A811.5, A812, A812.5, A813, A813.5, A814, A814.5, A815, A815.5, A816, A816.5, A817, A817.5, A818, A818.5, A819, A819.5, A820, A820.5, A821, A821.5, A822, A822.5, A823, A823.5, A824, A824.5, A825, A825.5, A826, A826.5, A827, A827.5, A828, A828.5, A829, A829.5, A830, A830.5, A831, A831.5, A832, A832.5, A833, A833.5, A834, A834.5, A835, A835.5, A836, A836.5, A837, A837.5, A838, A838.5, A839, A839.5, A840, A840.5, A841, A841.5, A842, A842.5, A843, A843.5, A844, A844.5, A845, A845.5, A846, A846.5, A847, A847.5, A848, A848.5, A849, A849.5, A850, A850.5, A851, A851.5, A852, A852.5, A853, A853.5, A854, A854.5, A855, A855.5, A856, A856.5, A857, A857.5, A858, A858.5, A859, A859.5, A860, A860.5, A861, A861.5, A862, A862.5, A863, A863.5, A864, A864.5, A865, A865.5, A866, A866.5, A867, A867.5, A868, A868.5, A869, A869.5, A870, A870.5, A871, A871.5, A872, A872.5, A873, A873.5, A874, A874.5, A875, A875.5, A876, A876.5, A877, A877.5, A878, A878.5, A879, A879.5, A880, A880.5, A881, A881.5, A882, A882.5, A883, A883.5, A884, A884.5, A885, A885.5, A886, A886.5, A887, A887.5, A888, A888.5, A889, A889.5, A890, A890.5, A891, A891.5, A892, A892.5, A893, A893.5, A894, A894.5, A895, A895.5, A896, A896.5, A897, A897.5, A898, A898.5, A899, A899.5, A900, A900.5, A901, A901.5, A902, A902.5, A903, A903.5, A904, A904.5, A905, A905.5, A906, A906.5, A907, A907.5, A908, A908.5, A909, A909.5, A910, A910.5, A911, A911.5, A912, A912.5, A913, A913.5, A914, A914.5, A915, A915.5, A916, A916.5, A917, A917.5, A918, A918.5, A919, A919.5, A920, A920.5, A921, A921.5, A922, A922.5, A923, A923.5, A924, A924.5, A925, A925.5, A926, A926.5, A927, A927.5, A928, A928.5, A929, A929.5, A930, A930.5, A931, A931.5, A932, A932.5, A933, A933.5, A934, A934.5, A935, A935.5, A936, A936.5, A937, A937.5, A938, A938.5, A939, A939.5, A940, A940.5, A941, A941.5, A942, A942.5, A943, A943.5, A944, A944.5, A945, A945.5, A946, A946.5, A947, A947.5, A948, A948.5, A949, A949.5, A950, A950.5, A951, A951.5, A952, A952.5, A953, A953.5, A954, A954.5, A955, A955.5, A956, A956.5, A957, A957.5, A958, A958.5, A959, A959.5, A960, A960.5, A961, A961.5, A962, A962.5, A963, A963.5, A964, A964.5, A965, A965.5, A966, A966.5, A967, A967.5, A968, A968.5, A969, A969.5, A970, A970.5, A971, A971.5, A972, A972.5, A973, A973.5, A974, A974.5, A975, A975.5, A976, A976.5, A977, A977.5, A

