G-CEM™ CAPSULE
SELF ADHESIVE LUTING CEMENT IN CAPSULES

For use only by a dental professional in the indications for use.

- INDICATIONS FOR USE**
1. Cementation of all types of all ceramic, resin and metal-based inlays, onlays, crowns and bridges (max. 2 abutments)
 2. Cementation of metal, ceramic, fiber posts, and cast post and cores.

- CONTRAINDICATIONS**
1. Direct pulp capping.
 2. Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate, polyurethane or acrylic resin polymer.

COMPOSITION
 Powder: Fluoro-alumino-silicate glass, initiator, pigment
 Liquid: Dimethacrylate, 4-MET, water, phosphoric acid ester monomer, silicon dioxide, initiator, stabilizer

DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0.35 / 0.15
Mixing Time (sec.)	10*
Working Time (23°C/73°F) (min., sec.)	2'30"

A. CEMENTATION TECHNIQUE FOR CERAMIC, INDIRECT COMPOSITES AND METAL RESTORATIONS

- 1. TOOTH PREPARATION**
- a) Prepare tooth in usual manner. For pulp capping, use calcium hydroxide cement.
 - b) Clean the prepared tooth with pumice and water.
 - c) Clean thoroughly with water. Dry by gently blowing with oil free air.
- Prepared surfaces should appear dry.
- Preparation Notes:**
1. Do not use substances like H₂O₂ (hydrogen peroxide) and/or potassium persulfate on the tooth.
 2. Special care should also be taken to remove so called amalgam powders very thoroughly with pumice and water.
- 2. RESTORATION PREPARATION**
 Make sure that the restoration is pretreated and handled according manufacturer's instructions.
- 3. CAPSULE ACTIVATION AND MIXING**
- a) Beforehand, make the capsule or cap its side on a hard surface to loosen the powder (Fig. 1).
 - b) To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig. 2).
- 1) Ensure the plunger is fully pressed to avoid the incorrect ratio of mix of powder and liquid.
- 2) The capsule should be activated just before mixing and used immediately.
 - 3) The capsule is set into a mixer (or an amalgamator) and mix for 10 seconds (N: 4.000 RPM) (Fig. 3). More details on mixing times can be found in following chart.
- | Mixing time chart | |
|-----------------------------------|-------------|
| Spindle Speed | Mixing Time |
| Standard mixer (approx 4.000 rpm) | 10 seconds |
| High speed mixer (over 4.500 rpm) | 10 seconds |
| Rotational mixer | 10 seconds |
| Center centrifuge option | 10 seconds |
- Note: Refer to the respective manufacturer's instructions for use.
- 4. DISPENSING**
- a) Immediately remove the mixed capsule from the mixer and load right into the capsule.
 - b) Click twice to prime the capsule (Fig. 4). To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
 - c) Extrude directly into the restoration (Fig. 5).
- Ambient light may shorten the working time. It is recommended to diminish the intensity or turn off your operation light during the cementation procedure.
- d) To remove the used capsule, pull the applicator release button.
 - e) Twist the capsule and pull upwards.
- 5. CEMENTATION**
- a) Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes 30 seconds from start of mixing at 23°C (73°F). Higher temperatures will shorten working time. The working time of G-CEM CAPSULE is recommended to extend the working time.
 - b) Maintain moderate pressure.
 - c) EXCESS CEMENT REMOVAL
- a) To remove excess cement, light cure using a visible light curing unit for 2 minutes.
 - b) Use excess cement to fill the margins of the restoration.
 - c) Light cure each surface for 10 seconds (single surface from occlusal): 10 seconds, any other surfaces: 10 seconds each for 4 minutes after seating of the restoration.
- G-CEM CAPSULE is a dual cure cement. To insure proper curing following cementation of the following restorations, G-CEM CAPSULE must be light cured for 2 minutes for 4 minutes. Metal, metal coping restorations**
- a) Metal coping restorations
 - b) Metal coping restorations
- *Translucent metal coping restorations greater than 2mm in thickness
- d) Finish and polish margins with appropriate instruments.

- B. CEMENTATION TECHNIQUE FOR METAL, FIBERGLASS AND CERAMIC POSTS**
- Prepare the post space according to manufacturer's instructions. The post space should be cleaned, rinsed and dried thoroughly using paper points. The post space must be dry.
- NOTE:**
 Aqueous hypochlorite solution is recommended to use for the chemical cleaning of the post space.
- DO NOT** use H₂O₂ (hydrogen peroxide) to chemically clean the post space as it may lower the bond strength.
- POST PREPARATION**
 Make sure that the post is pretreated and handled according to the manufacturer's instructions.
- 3. CAPSULE ACTIVATION AND MIXING**
- NOTE:**
 Refrigerate G-CEM CAPSULE to extend working time.
- a) Immediately remove the mixed capsule from the mixer and attach it to the capsule Elongation Tip on the capsule nozzle firmly until it clicks. G-CEM Capsule Elogation Tip can only be attached to G-CEM CAPSULE.
 - b) Do not attach the tip on other capsules. Immediately place the capsule with the elongation tip on the capsule nozzle.
 - c) Click twice to prime the capsule (Fig. 4). To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
- 1) Immerse the tip deep in the post space and start extruding the cement within 45 seconds from start of mix. Keep the tip immersed in the cement whilst dispensing to avoid trapping air bubbles.
- NOTE:**
 Ambient light may shorten the working time. It is recommended to diminish the intensity or turn off your operation light during the cementation procedure.
- d) The post should be immediately into the post space within 1 minute after cement application.
- e) The oral temperature accelerates the setting reaction of the cement.
 - f) Do NOT use lenufo spirals to place the cement in the post space as this may shorten working time.
 - f) To remove the used capsule, push the applicator release button.
 - e) Twist the capsule and pull upwards.
- 5. EXCESS CEMENT REMOVAL**
- a) Continue to maintain moderate pressure while removing excess cement with the applicator.
 - b) Light cure all surfaces for 20 seconds or allow the cement to self cure for 4 minutes after seating of the post (Fig. 10).

STORAGE

Refrigerate for optimal performance, store the capsules in the original aluminium foil in a cool and dark place (4-25°C) (39-27,7°F).

SHADES

A2 (Vita® shades), A03 (opaco), B01 (opaco) and Translucent (translucent shade) available in Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGE

1. Single shade package: 50 capsules
 2. Assorted package: 50 capsules (20 each of A2 and Translucent, 10 each of A03 and B01)
 Average contents per capsule: 0.35 g powder and 0.15 g (0.14 mL) liquid

Minimum net volume of mixed cement per capsule: 0.17 mL
 G-CEM CAPSULE APPLIER: 1 piece
 G-CEM CAPSULE Elogation Tip: 50 pieces

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCCA BUTTER can be used to isolate the operation field from the skin or oral tissue.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. If the tissue contacted by the material turns white or forms a blister, advise the patient to leave the affected area undisturbed, until the mark disappears, usually in 1-2 weeks.

4. In case of use G-CEM CAPSULE in combination with eugenol containing materials as eugenol may hinder G-CEM CAPSULE from setting or bonding properly.

5. Use a protective light shield or similar protective eye wear during light cure.

6. This product is not indicated for filling or core build-up.

7. When wiping G-CEM Capsule Elogation Tip with a cotton soaked with alcohol, prevent the alcohol from entering the inside of the tip.

8. Do not use G-CEM CAPSULE for single use only and cannot be autoclaved or chemically.

9. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

9. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarise yourself with the safety data sheets available at:

http://www.gcemurope.com
 http://www.gcemamerica.com
 They can also be obtained from your supplier.

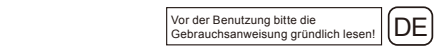
For the Summary of Safety and Clinical Performance (SSCP) please refer to EUAMED database (https://ec.europa.eu/autools/medadm) or contact us at Regulatory.gce@gce.dental

Undesired effects - Reporting:
 If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by contacting the national authority of your country accessible through the following link:

https://ec.europa.eu/growth/basics/medical-devices/contacts_en as well as to our internal vigilance system, vigilance@gce.dental. In this way you will contribute to improve the safety of this product.

This Responsible Person
 GC UNITED KINGDOM Ltd.
 Coopers Court Newport Pagnell, Buckinghamshire, MK16 5AS United Kingdom

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