**GRADIA™ gum shades**

LIGHT-CURED GUM SHADE COMPOSITE SYSTEM

For use only by a dental professional in the indications for use.

**INDICATIONS FOR USE**

Reproduction of gum tissue for crown restorations, such as implant superstructures and removable metal frames.

**CONTRAINDICATIONS**

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

**PRODUCT DESCRIPTION**

Gum shades in a light-cure material. The particle size of inorganic fillers range is 0.01-1.7 µm for GUM OPAQUE / GUM OPAQUE MODIFIER, 0.02-2.9 µm for GUM / GUM MODIFIER / GUM TRANSLUCENT. The total amount of inorganic filler is approximately 45% for GUM OPAQUE / GUM OPAQUE MODIFIER / GUM MODIFIER / GUM TRANSLUCENT, 67 w% for GUM.

**COMPOSITION**

GUM OPAQUE, GUM OPAQUE MODIFIER Dimethacrylate, silicon dioxide, multifunctional monomer, initiator, methacrylate polymer, pigment, stabilizer GUM Dimethacrylate, silicon dioxide, multifunctional monomer, feldspat, initiator, pigment, stabilizer GUM MODIFIER, GUM TRANSLUCENT Dimethacrylate, multifunctional monomer, silicon dioxide, feldspat, initiator, pigment, stabilizer GUM FIBER Pigment

**DIRECTIONS FOR USE**

For the following steps (No.1-5) of the procedure, refer to GRADIA™ instructions for use.

- Production of Metal Primer (G20) for the procedure, refer to:
  - Wax up framework in usual manner.
- Cast in usual manner.
- Using a brush, apply the relevant adhesive in usual manner.
- Treatment of Metal Frame area
  - Sandblast retention area using 50 to 110 micron aluminum oxide.
  - Clean with dry, oil-free air.
- Application of METALPRIMER II / METALPRIMER Z
  - Apply one or two thin coats of METALPRIMER II / METALPRIMER Z on the retention area.

Note:

- After application of the METALPRIMER II / METALPRIMER Z, immediately apply GRADIA OPAQUE to avoid staining the retention area.
- Application and Light curing of GRADIA FOUNDATION OPAQUE
  - Apply a layer of GRADIA FOUNDATION OPAQUE to the retention area to mask the metal colour. Light cure for 1 minute using the LABOLIGHT LV-III / LABOLIGHT DUO.

Note:

- The STEPLIGHT SL- / LABOLIGHT DUO cannot be used for light curing GRADIA FOUNDATION OPAQUE.
- Build-up of veneer crown
  - Apply GRADIA composite and light cure according to GRADIA instructions for use.

6. Application and Pre-light curing of Gum Opaque (GO)

- Apply two thin layers tooth and cast final frame to the part of the metal framework. Temporarily light cure each layer for 1 minute using the LABOLIGHT LV-III / LABOLIGHT DUO.

Note:

- For light curing refer to the following chart of Irradiation time and Depth of cure.

- For colour adjustment, apply Gum Opaque Modifier (GOM51) to the light cured Gum Opaque. Light cure for 1 minute using the LABOLIGHT LV-III /LABOLIGHT DUO.
- The STEPLIGHT SL- /LABOLIGHT DUO cannot be used for light curing Gum Opaque / Gum Modifier.

7. Application, Pre-curing and Final curing of GUM (G)

- Apply GUM (G). Temporarily light cure for 30 seconds using the LABOLIGHT LV-III / LABOLIGHT DUO or for 10 seconds using the STEPLIGHT SL- / LABOLIGHT DUO. Finally light cure for 3 minutes using the LABOLIGHT LV-III / LABOLIGHT DUO.

- If gum application exceeds the depth of cure mentioned in the chart, apply in layers and light cure respectively in the LABOLIGHT LV-III /LABOLIGHT DUO).
- In case of long span restorations, apply in sections to each side of area tooth and cast final frame to the part respectively in order to minimise curing shrinkage.
- Apply Gum Modifier (GM) or Gum Translucent (GT) as necessary.
- If it is desired to reproduce blood vessels in the gum area, incorporate Gum Fiber (GF71) in translucent Gum Modifier (GM30) or material of another shade, mix, apply and light cure.
- When using Gum Fiber, cover with Gum Translucent. Light cure and polish.

8. Removal of the finished layer and Shape adjusting

- Remove air inhibited layer and adjust contour using a carbide bur, diamond point or carbundum point.
- Finish leave the margin thicker than desired and progressively reduce to a rubber point or silicone point.

10. Polish and Buff Polish and buff in usual manner to complete the gum restoration.

Light curing device	Pre-cure	Final cure
LABOLIGHT LV-III, II/DUO	1 min.	3 min.
Halogen / High power LED	20 sec.	30 sec.
Note:		
1) The STEPLIGHT SL- /LABOLIGHT DUO cannot be used to light cure the above materials.		
2) When using a hand held type light curing device*, apply light in all directions for complete polymerization.		

Light curing device	Pre-cure	Final cure
LABOLIGHT LV-III, II/DUO	30 sec.	3 min.
STEPLIGHT SL- /LABOLIGHT DUO	10 sec.	3 min.
Halogen / High power LED	20 sec.	1 min.
Note:		
1) The STEPLIGHT SL- /LABOLIGHT DUO cannot be used for final curing.		
2) When using a hand held type light curing device*, apply light in all directions for complete polymerization.		

**DEPTH OF CURE (using the Labolight LV-III /LABOLIGHT DUO)**

Isolation time	Pre-cure	Pre-cure	Final cure
Shades	30 (sec.)	0.2 (mm)	3 (mm)
Gum Opaque	—	0.2 mm	—
Gum Opaque Modifier	—	0.2 mm	—
Gum	1.1 mm	—	2.5 mm
Gum Modifier	0.8 mm	—	1.5 mm
Gum Translucent	3.0 mm	—	5.0 mm
Note:			
The depths of cure for translucent Gum (G20) and Gum Modifier (GM30) are equivalent to that of Gum Translucent.			

**STORAGE**

Recommended for optimal performance, store in a cool place away from direct sunlight.

**SHADES & PACKAGES** (GRADIA gum shades)

1. Starter Package

- Serlinge of 2.4mL solution - Gum Opaque (G011)
- Serlinge of 2.4mL solution - Gum Opaque (G013)
- Serlinge of 2.9mL paste - Gum (G21)
- Serlinge of 2.9mL paste - Gum (G22)
- Serlinge of 2.9mL paste - Gum (G23)
- Serlinge of 2.9mL paste - Gum (G24)
- Serlinge of 2.4mL gel - Gum Translucent (GT41)

2. Multi Packages

- a. Gum Opaque (G011, G012 or G013)
  - Serlinge of 2.4mL solution
- b. Gum Opaque Modifier (GOM51)
  - Serlinge of 2.4mL solution
- c. Gum (G20, G21, G22, G23 or G24)
  - Serlinge of 2.9mL paste
- d. Gum Modifier (GM30, GM31, GM32, GM33, GM34, GM35 or GM36)
  - Serlinge of 2.4mL gel
- e. Gum Translucent (GT41)
  - Serlinge of 2.4mL gel
- f. Gum Fiber (GF71)
  - Bottle of 0.4g fibers

The SHADE COMBINATION CHART of GRADIA™ gum shades is shown in the manual that is available at <https://www.gdental/europe> or for The Americas <https://www.gdental/america>

**CAUTION**

- In case of contact with oral tissues or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- If material is stored in refrigerator and kept at room temperature for at least 30 minutes before use.

GRADIA™ gum shades are available at <https://www.gdental/europe> or for The Americas <https://www.gdental/america>

**CAUTION**

5. After dispensing material, use as quickly as possible. Be sure to remove any residual material from the tip of the syringe and replace the cap immediately after use.

6. Do not mix with other materials.

- When light curing material, be sure to follow the irradiation time described in the above-mentioned chart. Do not use ultraviolet lights or visible light curing devices that can radiate ultraviolet light.
- When light curing material, do not look directly into the light.
- When adjusting or polishing composite resin, use a dust collector and wear a dust mask to avoid inhaling dust.
- The brush can be cleaned with alcohol after use.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gdental/europe> or for the Americas <https://www.gdental/america>. They can also be obtained from your supplier.

**CLEANING AND DISINFECTING**

**MULTI-USE DELIVERY SYSTEMS:** To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use, inspect device and label for deterioration. Discard device if damaged.

**DO NOT IMMERSE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/euotools/euamed/>) or contact us at Regulatory.gd@gc dental

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental) in which you will contribute to improve the safety of this product.

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