Prior to use, carefully read the instructions for use.

**EN**

**GRADIA®** LIGHT-CURED COMPOSITE FOR CROWN & BRIDGE, INLAYS AND VENEERS

For use only by a dental professional in the recommended indications.

FOR MORE DETAILS READ THE GRADIA USER’S MANUAL.

#### INDICATIONS

1. Crown and Bridgework (with metal backing)
2. Metal free restorations: Jacket Crowns, Inlays, Onlays, Laminated Veneers
3. Implant superstructures

#### CONTRAINDICATIONS

1. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
2. In case of malocclusion, bruxism or clenching.

#### DIRECTIONS FOR USE

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- Coat the clean metal framework with METALPRIMER II.
- (GC) and allow to dry.
- Build up the veneer onto the metal framework in increments in the following order:
  - FOUNDATION OPAQUE, MARGIN OPAQUE, OPAQUE Light cure each layer for 1 minute using LABOLIGHT LV-III, II, DUO (GC).
  - Then INTENSIVE COLOR, OPAQUS DENTIN, DENTIN, MAMELON STAIN, ENAMEL or FLOWABLE COMPOSITE, CERVICAL TRANSLUCENT, ENAMEL INTENSIVE and HALO ENAMEL. Light cure each layer for 30 seconds with LABOLIGHT LV-III, II, DUO (GC) or for 10 seconds with the STEPLIGHT SL-I, LABOLIGHT DUO (GC).
  - Note: FLOWABLE COMPOSITE should be covered by a layer of CERVICAL TRANSLUCENT or ENAMEL INTENSIVE.
  - Coat the veneer surface with GRADIA AIR BARRIER.
  - Final cure for 3 minutes using LABOLIGHT LV-III, II, DUO (GC).
  - Polish with GRADIA DIAPOLISHER on a felt wheel, etc.

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1. Harden the stone tooth surface with GRADIA DIE-HARDNER.
  - Apply a layer of paraffin wax as die spacer, if required.
  - Coat the die surface with GRADIA SEPARATOR.
- Build up the crown onto the stone die in increments in the following order:
  - OPaque (on the labial surface), MARGIn OPAQUE (on the lingual surface) Light cure each layer for 1 minute with LABOLIGHT LV-III, II, DUO (GC).
  - Then SHOULDER DENTIN, OPAQUS DENTIN, DENTIN, MAMELON STAIN, ENAMEL or TRANSLUCENT or FLOWABLE COMPOSITE, CERVICAL TRANSLUCENT, ENAMEL INTENSIVE and HALO ENAMEL (on the labial surface) / MAMELON STAIN, OPAQUS DENTIN INTENSIVE, HALO ENAMEL (on the lingual surface) Light cure each layer for 30 seconds with LABOLIGHT LV-III, II, DUO (GC) or for 10 seconds the STEPLIGHT SL-I, LABOLIGHT DUO (GC).
  - Note: FLOWABLE COMPOSITE should be covered by a layer of other pastes such as ENAMEL or ENAMEL INTENSIVE.
  - Coat the crown surface with GRADIA AIR BARRIER.
  - Final cure for 3 minutes using LABOLIGHT LV-III, II, DUO (GC).
  - Polish with GRADIA DIAPOLISHER on a felt wheel, etc.

##### III. Posterior Inlay

1. a. Harden the stone cavity with GRADIA DIE-HARDNER.
  - Block out undercuts with wax.
  - Apply a layer of paraffin wax as die spacer, if required.
  - Coat the die surface with GRADIA SEPARATOR.
- Build up the inlay onto the stone die in increments in the following order:
  - CERVICAL TRANSLUCENT and ENAMEL INTENSIVE or ENAMEL
  - Photo-polymerize each layer for 30 seconds with LABOLIGHT LV-III, II, DUO (GC) or 10 seconds with STEPLIGHT SL-I, LABOLIGHT DUO (GC).
  - Coat the inlay surface with GRADIA AIR BARRIER.
  - Final cure for 3 minutes using LABOLIGHT LV-III, II, DUO (GC).
  - Polish with GRADIA DIAPOLISHER on a felt wheel, etc.

##### IV.COMPOSITE PRIMER

To be used with light-cured adhesive for additional resin application and repair.

**STORAGE**
Recommended for optimal performance, store in a cool place away from direct sunlight.

#### CAUTION

1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. GRADIA SEPARATOR, GRADIA DIE HARDNER and METALPRIMER II, Z (GC) are highly flammable liquid and vapour. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight.
3. When adjusting or polishing the veneer surface, use a dust collector and wear a dust mask to avoid inhaling dust.
4. In case of contact with oral tissues or skin, immediately remove with cotton. Flush with water.
5. In case of contact with eyes, immediately wash the eye with copious amounts of water and seek medical attention.
6. Dispose of all waste according to local regulations.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at <http://www.gecorpoe.com>. They can also be obtained from your supplier.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@ge.dental](mailto:vigilance@ge.dental)
In this way you will contribute to improve the safety of this product.

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- Apply a layer of paraffin wax as die spacer, if required.
- Coat the die surface with GRADIA SEPARATOR.

**2. Build up the crown onto the stone die in increments in the following order:**

- OPaque (on the labial surface), MARGIn OPAQUE (on the lingual surface) Light cure each layer for 1 minute with LABOLIGHT LV-III, II, DUO (GC).
- Then SHOULDER DENTIN, OPAQUS DENTIN, DENTIN, MAMELON STAIN, ENAMEL or TRANSLUCENT or FLOWABLE COMPOSITE, CERVICAL TRANSLUCENT, ENAMEL INTENSIVE and HALO ENAMEL (on the labial surface) / MAMELON STAIN, OPAQUS DENTIN INTENSIVE, HALO ENAMEL (on the lingual surface) Light cure each layer for 30 seconds with LABOLIGHT LV-III, II, DUO (GC) or for 10 seconds the STEPLIGHT SL-I, LABOLIGHT DUO (GC).
- Note: FLOWABLE COMPOSITE should be covered by a layer of other pastes such as ENAMEL or ENAMEL INTENSIVE.
- Coat the crown surface with GRADIA AIR BARRIER.
- Final cure for 3 minutes using LABOLIGHT LV-III, II, DUO (GC).
- Polish with GRADIA DIAPOLISHER on a felt wheel, etc.

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- Block out undercuts with wax.
- Apply a layer of paraffin wax as die spacer, if required.
- Coat the die surface with GRADIA SEPARATOR.

- Build up the inlay onto the stone die in increments in the following order:
- CERVICAL TRANSLUCENT and ENAMEL INTENSIVE or ENAMEL
- Photo-polymerize each layer for 30 seconds with LABOLIGHT LV-III, II, DUO (GC) or 10 seconds with STEPLIGHT SL-I, LABOLIGHT DUO (GC).
- Coat the inlay surface with GRADIA AIR BARRIER.
- Final cure for 3 minutes using LABOLIGHT LV-III, II, DUO (GC).
- Polish with GRADIA DIAPOLISHER on a felt wheel, etc.

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**CAUTION**
1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. GRADIA SEPARATOR, GRADIA DIE HARDNER and METALPRIMER II, Z (GC) are highly flammable liquid and vapour. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight.
3. When adjusting or polishing the veneer surface, use a dust collector and wear a dust mask to avoid inhaling dust.
4. In case of contact with oral tissues or skin, immediately remove with cotton. Flush with water.
5. In case of contact with eyes, immediately wash the eye with copious amounts of water and seek medical attention.
6. Dispose of all waste according to local regulations.

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