GRADIA™ gum shades

LIGHT-CURED GUM SHADE COMPOSITE SYSTEM

Use only by a dental professional in the indications for use.

INDICATIONS FOR USE
Restoration of gum tissue for crown restorations, such as implant superstructures and removable metal frames.

CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

PRODUCT DESCRIPTION
GRADIA gum shades is a light-cure material.

The shade class of Inorganic filler ranges from 0.1-1.7 µm for GUM OPAQUE / GUM OPAQUE MODIFIER, 0.02-2.9 µm for GUM / GUM MODIFIER / GUM TRANSLUCENT.

The total amount of inorganic filler is approximately 45 wt% for GUM OPAQUE / GUM OPAQUE MODIFIER / GUM TRANSLUCENT, 47 wt% for GUM / GUM MODIFIER / GUM TRANSLUCENT, 67 wt% for GUM TRANSLUCENT.

COMPOSITION
GUM OPAQUE: GUM OPAQUE MODIFIER
Dimethacrylate, silicon dioxide, multifunctional monomer, initiator, photoinitiator, pigment, stabilizer
GUM MODIFIER: GUM MODIFIER
Dimethacrylate, silicon dioxide, multifunctional monomer, feldspar, initiator, pigment, stabilizer
GUM TRANSLUCENT: GUM TRANSLUCENT
Dimethacrylate, multifunctional monomer, silicon dioxide, feldspar, initiator, pigment, stabilizer
GUM FIBER: GUM FIBER
Pigment

DIRECTIONS FOR USE
For the following steps (N-1-5) of the procedure, refer to GRADIA Instructions for use.

1. Preparation of Metal Framework
a) Wax up framework in usual manner.
b) Use in usual manner.

2. Using a carbide bur, adjust casting in usual manner.
3. Sandblast retention area using 50 to 100 micron aluminum oxide.

4. Application of METALPRIMER 1 / METALPRIMER 2
Apply one or two coats of METALPRIMER 1 / METALPRIMER 2 to the retention area.

After application of the METALPRIMER 1 / METALPRIMER 2, immediately apply using GRADIA OPAQUE to avoid contamination of the retention area.

4. Application and Light curing of GRADIA FOUNDATION OPAQUE
Apply a layer of GRADIA FOUNDATION OPAQUE to the retention area to mask the metal colour. Light cure for 1 minute using the LABOLIGHT LV. / LABOLIGHT DUO (step mode) cannot be used for light curing GRADIA FOUNDATION OPAQUE.

5. Build-up crown cover
Apply GRADIA composite and light cure according to GRADIA Instructions for use.

6. Application and Light curing of GUM OPAQUE (G01)
Apply two thin layers of GUM OPAQUE (G01) to the gum part of the metal framework. Temporarily light cure each layer for 1 minute using LABOLIGHT LV. / LABOLIGHT DUO (full mode).

Note:
1) For light curing refer to the following chart of irradiation time and Depth of cure.

2) For colour adjustment, apply GUM OPAQUE MODIFIER (G05) to the light cured GUM OPAQUE. Light cure for 1 minute using the LABOLIGHT LV. / LABOLIGHT DUO (full mode).

3) The STEPLIGHT SL / LABOLIGHT DUO (step mode) cannot be used for light curing GUM OPAQUE and GUM OPAQUE MODIFIER.

4. Application and Light curing of GUM MODIFIER (G02) and GUM MODIFIER (G03)
Apply GUM (G02) temporarily light cure for 30 seconds using the LABOLIGHT LV. / LABOLIGHT DUO (step mode) or 10 seconds using the STEPLIGHT SL / LABOLIGHT DUO (step mode). Finally light cure the entire area for 1 minute using the LABOLIGHT LV. / LABOLIGHT DUO (full mode).

Note:
1) If light curing exceeds the depth of cure mentioned in the chart, apply in layers and light cure respectively (in the LABOLIGHT LV. / LABOLIGHT DUO (step mode)).

2) In case of long span restorations, apply in sections to each side of one tooth and carry out light cure respectively in order to minimize curl shrinkage.

3) If it is desired to reproduce blood vessels in the gum area, incorporate GUM FIBER (F7) in translucent GUM MODIFIER (G03) or material of another shade, mix, apply and light cure.

4) When using GUM FIBER, cover with GUM TRANSLUCENT. Light cure only the GUM FIBER.

5. Removal of wax inhibited layer and Shade adjustment
Remove wax inhibited layer and adjust contour using a carbide bur. Blameed zone or defect will be removed.

6. Finish
Apply a margin brush than desired and progressively finish using a polish point or silicone point.

7. Polish
Polish and buff in usual manner to complete the gum restoration.

Irradiation time for GUM OPAQUE, GUM OPAQUE MODIFIER

Light curing device Pre-cure Final cure
LABOLIGHT LV. 1 / LABOLIGHT DUO (full mode) 1 min. 3 min.
Halogén / High power LED 20 sec.

Note:
1) The STEPLIGHT SL / LABOLIGHT DUO (step mode) cannot be used to light cure the above material.

2) The STEPLIGHT SL / LABOLIGHT DUO (step mode) cannot be used in all directions for complete polymerization.

Irradiation time for GUM, GUM MODIFIER, GUM TRANSLUCENT

Light curing device Pre-cure Final cure
LABOLIGHT LV. 1 / LABOLIGHT DUO (full mode) 1 min. 3 min.
Halogén / High power LED 20 sec. 3 min.

Note:
1) The STEPLIGHT SL / LABOLIGHT DUO (step mode) cannot be used for final curing.

2) Apply a hand held type light curing device, apply light in all directions for complete polymerization.

Depth of cure (using the LABOLIGHT LV. / LABOLIGHT DUO (full mode))

Shades Irradiation time (30 sec.) Final cure (30 sec.)
GUM OPAQUE 0.2 mm 1.1 mm
GUM OPAQUE MODIFIER 1.1 mm 2.5 mm
GUM MODIFIER 0.8 mm 1.5 mm
GUM TRANSLUCENT 3.0 mm 5.0 mm

SHADE COMBINATION CHART OF GRADIA gum shades is shown in the manual that is available at <https://www.gc.dental/europe>

CAUTION
1. In case of contact with oral tissues or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Take care to avoid ingestion of the material. Do not use ultraviolet lights or visible light curing devices that can radiate an invisible light.

4. When light curing material, do not look directly into the light source.
5. When adjusting or polishing composite resin, use a dust collector and wear a dust mask to avoid inhaling dust.

6. The brush can be cleaned with alcohol after use.
7. In all rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue use of the product and refer to a physician.

8. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

9. Some products referenced in the present file may be classified as hazardous according to GHS. Always refer yourself with the safety data sheets available at <https://www.gc.dental/europe>.

10. The data were obtained from our supplier.

CLEANING AND DISINFECTING.
1. To clean and disinfect systems to avoid cross contamination between patients, this device requires mid-level disinfection. Immediately after use inspect the device for any visible contamination.
2. Do NOT IMMERS. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare grade infection control disinfectant to registered guidelines.
3. For the Summary of Safety and Clinical Performance (SSCP) please see the SHARED database (<https://www.gc.dental/europe>) or contact us at Regulatory.gcc@gc-dental.com

Undeclared effects: Reporting
This device becomes aware of any kind of undesired effect, reaction or similar events experienced by the user of this product, including those not listed in this instruction for use.
If you have any information regarding an undesired effect, reaction or similar event, reporting the proper authority of your country accessible through the following link: <https://www.gc.dental/europe>

As well as to our internal vigilance system, vigilance@gcc-dental.com
In any way you will contribute to improve the safety of our products.

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